

# ACCLEAN®



## BEST PRACTICES FOR YOUR NEW ULTRASONIC SCALER

### DISINFECTING & CLEANING THE UNIT

**Turn the device off and unplug from power before performing disinfection and cleaning on the device.**

- To clean the surface of the unit, tubing, or sheath, use rubbing alcohol on a towel and discard towel in a bio-hazard bin
- When removing the handpiece sheath, hold the handpiece base in place and only twist and pull on the sheath to remove it
- **Do not** clean surface of unit, tubing, or sheath with cold sterile or cavi wipes
- **Do not** submerge the handpiece assembly or sheath in disinfectant solution
- **Do not** spray liquids directly onto the device surface. Always spray them onto a towel and then discard the towel into a bio-hazard bin

### OPERATING THE UNIT

- Only operate the unit in boost mode, as needed. Boost mode is when the red arrow lights up
- At the end of the day, purge the water line and system of all water, using the air quick disconnect
- When disconnecting the foot pedal, make sure to unplug with your fingers on the connector with the up and down arrows
- **Do not** pull on the cord, to prevent damaging internal connections
- **Do not** use damaged or bent inserts with the unit
- **Do not** remove or operate the unit without an inline water filter
- **Do not** prime the system using the pedal, only use the purge function



### MAINTENANCE

**Change the water supply line filter every 3 months or when the filter becomes discolored.**

**ACCLEAN® WATER FILTER REPLACEMENT KIT**  
4-Pack, (570-0396)

Everyday Preventives for Healthy Patients

[www.HenryScheinDental.com/ACCLEAN](http://www.HenryScheinDental.com/ACCLEAN)