

ACCLEAN®



ULTRASONIC SCALER FAQ

SOLUTIONS FOR YOU

My unit is continuously beeping when it is plugged into power and switched on. How do I fix this?

Solution: Make sure the pedal is plugged in and connected to the back of the unit before the unit is plugged in and powered on.

When I am purging/priming my unit, the unit is beeping (beeps twice, then pauses, then beeps again three times when the pedal is pressed). What should I do?

Solution: This indicates there is no insert in the handpiece. The unit cannot be purged/primed with the pedal. Use the purge function on the unit (see IFU page 4 for instructions on the purge function).

The unit is beeping.

Solution: See the audible alerts chart below.

USER OPERATION	ALERT DESCRIPTION	SOUND
Power Switch	Power ON	Beep x2
Cruise Button	Cruise Mode ON	Beep x1
Cruise Button	Cruise Mode OFF	Beep x2
Purge Button	Purge Mode ON	Beep x1
N/A	Purge cycle finished	Beep x2
Purge Button	Purge Mode OFF	Beep x2
Foot Pedal	Handpiece without insert	Beep x2, Pause, Beep x3
Foot Pedal	Foot Pedal does not connect with main unit	Beep x3

Everyday Preventives for Healthy Patients

www.HenryScheinDental.com/ACCLEAN