

ACCLEAN®

ULTRASONIC SCALER TIPS AND TRICKS FOR YOUR NEW PURCHASE

A GUIDE TO THE LIGHTS



1. RED ARROW
Boost Mode is ON

2. BLUE
Cruise control is ON

WHITE ARROW
Standard Power Mode
turns ON when pedal is initiated

3. GREEN
Purge function is ON

THE WATER GUIDE

Make sure that there is properly treated water being used through the unit and that the unit is flushed of all water at the end of use.

Remember to change the water line filter every 3 months or when the filter becomes discolored.

**ACCLEAN® WATER FILTER
REPLACEMENT KIT**
4-Pack, (570-0396)

AUDIBLE ALERTS: KNOW YOUR BEEPS

USER OPERATION	ALERT DESCRIPTION	SOUND
Power Switch	Power ON	Beep x2
Cruise Button	Cruise Mode ON	Beep x1
Cruise Button	Cruise Mode OFF	Beep x2
Purge Button	Purge Mode ON	Beep x1
N/A	Purge cycle finished	Beep x2
Purge Button	Purge Mode OFF	Beep x2
Foot Pedal	Handpiece without insert	Beep x2, Pause, Beep x3
Foot Pedal	Foot Pedal does not connect with main unit	Beep x3

Everyday Preventives for Healthy Patients

www.HenryScheinDental.com/ACCLEAN