Read before use. Keep in a safe place for future reference.

Caution: Federal (US) law restricts this device to sale by or on the order of a licensed professional.

Caution

- For professional use only.
- Sterilization is required before first use.
- This handpiece uses only disposable prophy angles.
- Before each use, check for vibration, noise, and overheating. If any abnormalities are observed, stop using immediately and contact Henry Schein ProRepair.
- Do not drop, hit or subject to excessive shock.

- The user is responsible for correct operation, maintenance, and inspection.
- Do not wipe or immerse in acidic solutions.
- Never operate the handpiece without a disposable prophy angle installed.
- Do not exceed the recommended air pressure. Excessive air pressure may cause premature failure.
- Do not disassemble or alter handpiece.

Specifications

Item	Speed range	Drive air pressure
Acclean hygienist handpiece (112-5544)	0 - 5,000 rpm	35 psi

Motor Connection

Caution

- Air supply must be dry, clean, and uncontaminated. Moist, contaminated air leads to wear and premature failure.
- Damaged or missing connection gasket can cause water and air leaks resulting in poor performance.

Connection to air supply

Insert 4-hole backend into tubing, then hand-tighten union nut clockwise.

Caution

- Improper connection to tubing may result in handpiece disconnecting while in use. Before each use, ensure handpiece is securely fastened to tubing.

Inserting / Removing Disposable Prophy Angle (Fig. 1)

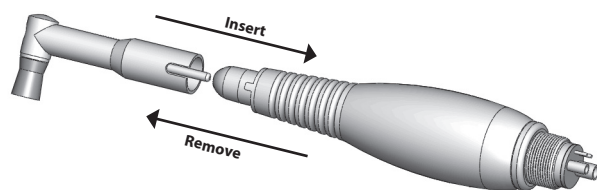
Fig. 1

Inserting

- Align notch on prophy angle with alignment pin on handpiece and push the prophy angle on until it stops.

Removing

- To remove prophy angle, pull off by hand.

Caution

- Make sure the prophy angle sits flush on handpiece. If not, repeat inserting process.
- Do not install or remove the prophy angle until motor has stopped completely.
- Loose prophy angles cause excessive noise and vibration and may result in patient injury.
- After the prophy angle is seated, pull on it slightly to make sure it is locked into place.
- Do not re-use disposable prophy angles as they could break and cause injury.

Maintenance

The Henry Schein Acclean handpiece DOES NOT require lubrication.

1. Disconnect handpiece from air supply tubing.
2. Dispense handpiece cleaner into drive air tube (Fig. 2).
3. Reconnect handpiece and run for 10 seconds or until handpiece runs smoothly.

Fig. 2

Sterilization

1. Wipe handpiece with damp cloth to remove debris.
2. Place handpiece in sterilization pouch and seal. Bag handpiece individually.
3. Steam autoclave for 25 minutes at 270° F (132° C).
4. Allow a minimum of 20 minutes for the drying cycle.

Caution

- Sterilization is required before first use.
- Sterilize by steam autoclave after each patient use.
- Wear appropriate personal protective equipment during sterilization process. This should include eye protection and gloves.
- Do not use ultrasonic bath, boiling water or chemicals to clean the handpiece.
- Do not autoclave at temperatures that exceed 275° F (135° C).
- Do not autoclave together with other instruments.
- Do not use dry heat sterilization.
- Keep in sterilization pouch until ready to use.

Troubleshooting

Problem	Solution
Sluggish handpiece	1. Check air pressure. 2. Check for faulty or missing gasket. 3. Perform motor cleaning procedure.
Handpiece does not turn on	1. Check air pressure. 2. Check for faulty or missing gasket.
Air leak at connector	1. Check air pressure. 2. Check for faulty or missing gasket.
Handpiece runs but disposable prophy angle does not rotate	Faulty prophy angle. Install new one.
Disposable prophy angle vibrates excessively	Faulty prophy angle. Install new one.

If problems still occur after following these instructions, contact Henry Schein ProRepair at 1-800-367-3674.

Service

For items that require service, including warranty service, please send to Henry Schein ProRepair.

Henry Schein ProRepair
 620 S. Placentia Ave.
 Placentia, CA 92870 USA
 Phone: +1-714-579-0175
 Fax: +1-714-579-0186
 Email: prrepair@henryschein.com

Warranty

These products are designed for use in a dental office and this warranty is not applicable to other uses. The products are warranted against manufacturing defects in materials and workmanship for one (1) year from date of purchase. Henry Schein® will repair or replace products at its sole discretion. This warranty shall be limited to replacement or repair of the products or their parts and shall not extend to any other claims, including but not limited to loss of profit, cost of removal or replacement, incidental or consequential damages or other similar claims arising from use of this product. Damage to products resulting from acts of God, faulty installation, misuse, tampering, accident, abuse, negligence, alterations, unauthorized repairs or problems unrelated to materials and workmanship are not covered by this warranty.

DISTRIBUTED BY:
 Distribuido por: Distribuído por:
 HENRY SCHEIN, INC.
 Melville, NY 11747 USA

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