# ACCLEAN™

REF

570-2861  
570-2855

## HYGIENE HANDPIECE 2

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INSTRUCCIONES DE USO  
MODE D'EMPLOI

by  HENRY SCHEIN®

- For professional use only.
- The user is responsible for correct operation, maintenance, and inspection.
- Sterilization is required before first use.
- Do not wipe or immerse in acidic solutions.
- This handpiece uses only disposable prophylaxis angles.
- Never operate the handpiece without a disposable prophylaxis angle installed.
- Before each use, check for vibration, noise, and overheating. If any abnormalities are observed, stop using immediately and contact Henry Schein ProRepair.
- Do not exceed the recommended air pressure. Excessive air pressure may cause premature failure.
- Do not drop, hit or subject to excessive shock.
- Do not disassemble, modify or alter handpiece.
- Never attempt to remove disposable prophylaxis angle while handpiece is running.

## Specifications

Description	Acclean Hygiene Handpiece 2	Acclean Hygiene Handpiece 2S
Item Code	570-2861	570-2855
Drive Air Pressure	36 psi	36 psi
Operating Speed	0 - 3,000 rpm	0 - 3,000 rpm
Connection Type	4-hole	Swivel Coupler
Weight	89 g	86 g

## Operation

### Caution

- Air supply must be dry, clean, and uncontaminated. Moist, contaminated air leads to wear and premature failure.
- Exceeding recommended air pressure may result in premature failure.
- Damaged or missing connection gasket can cause water and air leaks resulting in poor performance.
- Improper connection to tubing may result in handpiece disconnecting while in use. Before each use, ensure handpiece is securely fastened to tubing.

## ACCLEAN HYGIENE HANDPIECE 2

### Connecting Handpiece (Fig. 1)

1. Align tubes in handpiece with delivery system tubing.
2. Thread tubing nut to bottom of handpiece and finger tighten.

### Removing Handpiece (Fig. 1)

1. Unthread tubing nut and pull handpiece straight off.

Fig. 1

## ACCLEAN HYGIENE HANDPIECE 2S

### Connecting Coupler (Fig. 2)

1. Align tubes and pins on coupler with holes in delivery system tubing.
2. Thread tubing nut to bottom of coupler.
3. Hold coupler and finger tighten nut.

### Connecting Handpiece (Fig. 3)

1. Slide handpiece onto coupler.
2. Press firmly until it snaps into place.

**Note:** Handpiece should swivel freely with slight tension and sit flush on coupler.

### Removing Handpiece (Fig. 3)

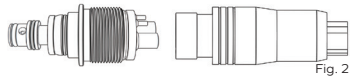
1. Hold ring on coupler and pull handpiece straight off.

**Note:** Do not hold or pull on hose.

### Changing Color Bands

**Note:** Color bands are autoclavable.

1. Use an explorer to gently remove old color band.
2. Slide on new color band using only fingers to not cause damage.

## CHANGING DISPOSABLE PROPHY ANGLE (FIG. 4)

### Caution

- Make sure the prophy angle sits flush on handpiece. If not, repeat inserting process.
- Do not install or remove the prophy angle until motor has stopped completely.
- Loose prophy angles cause excessive noise and vibration and may result in patient injury.
- After the prophy angle is seated, pull on it slightly to make sure it is locked into place.
- Do not re-use disposable prophy angles as they could break and cause injury.

### INSTALLATION

1. Align notch on prophy angle with alignment pin on handpiece and push the prophy angle on until it stops.

### REMOVAL

1. To remove prophy angle, pull off by hand.

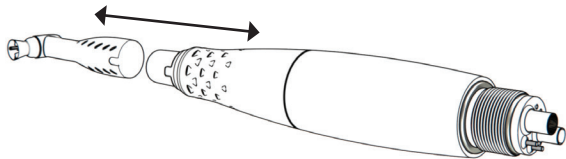
Fig. 4

## Maintenance

**NOTE: The Acclean Hygiene Handpiece 2 & 2S DO NOT require lubrication.**

1. Disconnect handpiece and coupler from air supply tubing Leave the coupler in the handpiece..
2. Dispense handpiece cleaner into drive air tube of handpiece or coupler (Fig. 5).
3. Reconnect handpiece and run for 10 seconds or until handpiece runs smoothly.

### CHANGING COUPLER O-RINGS

- Improper care of o-rings may cause handpiece to malfunction.
- When changing/replacing coupler o-rings, always replace complete set of 2 pieces.

**Note:** Coupler o-rings may be lubricated with Henry Schein Maxima HP Spray.

1. Use an explorer to gently remove old/worn o-rings.
2. Slide on new o-rings using only fingers to not cause damage.

## Sterilization

### Warning

- Sterilize with autoclave after each patient use.
- Sterilization is required before first use.

### Caution

- Wear appropriate personal protective equipment during sterilization and lubrication process. This should include eye protection and gloves.
  - Do not use ultrasonic bath or cold sterile/disinfectant chemicals to clean handpiece.
  - Do not autoclave handpieces together with other instruments.
  - Do not use dry heat sterilization.
  - Keep handpiece in sterilization pouch until ready to use.
  - Do not cool handpiece manually as damage may occur.
  - Do not autoclave at temperatures that exceed 275° F (135° C).
1. Remove bur, then wipe external surface with dry paper towel to remove any excess lubricant and debris. Place paper towel in bio-hazard bin when finished.
  2. Place handpiece in sterilization pouch and seal. Bag handpieces individually; no more than 1 per pouch

3. Sterilize in a gravity type autoclave for a minimum of 15 minutes or in a pre-vacuum autoclave for a minimum of 4 minutes at 270° F (132° C). Let autoclave cycle completely. Do not remove before handpiece is completely cool and pouch is dry.

## Replacement Parts & Accessories

570-2854	Acclean Hygiene Handpiece 2S Coupler	570-2857	Acclean Hygiene HP 2 Bands - Red
570-2856	Acclean Hygiene HP 2 Bands - Pink	570-2859	Acclean Hygiene HP 2 Bands - Blue
570-2858	Acclean Hygiene HP 2 Bands - Orange	112-6928	Maxima HP Spray
570-2860	Acclean Hygiene HP 2 Bands - Green		

## Service

For items that require service, including warranty service, please send to Henry Schein® ProRepair.

Henry Schein ProRepair  
620 S. Placentia Ave.  
Placentia, CA 92870  
714-579-0175

## Warranty

These products are designed for use in a dental office and this warranty is not applicable to other uses. The products are warranted against manufacturing defects in materials and workmanship for 1 year from date of purchase.

Henry Schein® will repair or replace products at its sole discretion. This warranty shall be limited to replacement or repair of the products or their parts and shall not extend to any other claims, including but not limited to loss of profit, cost of removal or replacement, incidental or consequential damages or other similar claims arising from use of this product. Damage to products resulting from acts of God, faulty installation, misuse, tampering, accident, abuse, negligence, alterations, unauthorized repairs or problems unrelated to materials and workmanship are not covered by this warranty.

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**Henry Schein Inc.**