REF 570-3444

ACCLEAN™

PIEZO ULTRASONIC SCALER

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by/de  HENRY SCHEIN®

INDICATION FOR USE

1. Removing supra and sub gingival calculus deposits and stains from teeth
2. Periodontal pocket lavage with simultaneous ultrasonic tip movement

Caution

- The Acclean Piezo Ultrasonic Scaler should not be used for restorative dental procedures involving the condensation of amalgam.
- Persons with cardiac pacemakers, defibrillators and other active implanted medical devices are cautioned that some types of electronic equipment might interfere with the operation of the device. Dental professionals should contact the device manufacturer or the patient's physician for specific recommendations.
- Do not use in the presence of flammable anesthetic mixture with air, oxygen or nitrous oxide.
- Do not modify the Acclean Piezo Ultrasonic Scaler. Modifications will invalidate the warranty and may endanger the patient and/or operator.
- This equipment is not to be used on a patient with Hemophilia.
- Be cautious when using this equipment on patients who have heart disease, who are pregnant, or who are children.

GENERAL OPERATION/PRECAUTIONS

Caution

- For professional use only.
- Use personal protective wear during use.
- Sterilize handpiece, wrench and tips prior to first use.
- Never use with tips that show excessive wear.
- Continuous water flow must be maintained at all times during operation to prevent overheating.
- Keep away from heat and direct sunlight.
- If overheating or any abnormalities occur, stop using immediately and contact Henry Schein Handpiece & Small Equipment Solutions at 800-235-6337.
- If excessive wear of tip is observed, discard and replace tip immediately.
- Bent, damaged or reshaped tips are susceptible to breakage during use, discard these tips immediately and replace.
- To prevent contact with the ultrasonic tip, be sure to retract patient's lips, cheeks, and tongue.

PARTS AND CONTROLS

IN THE BOX

1. Main Unit
2. Handpiece
3. External power supply
4. Water supply tube with filter & 2 quick disconnects
5. Foot control pedal

TIP KIT

1. Torque Wrench
2. Universal Tip (3)
3. Heavy Scaling (1)
4. Slim Tip (1)

BACK OF UNIT

1. Water flow adjustment
2. Water supply line connection
3. Foot pedal connection
4. Power supply connection
5. Power on/off switch

SETUP

1. Place the Accclean Piezo Ultrasonic Scaler main unit on a level surface.
2. Confirm the power switch on the back of the main unit is in the OFF position.
3. Completely insert water supply tube into water supply line connection on the main unit (Fig. 1) until it snaps into place and then connect the quick disconnect (Fig. 2) to the water source. Confirm there are no leaks. Note: Only connect to a clean, filtered water supply capable of supplying 25 – 40 psi that has a manual shut-off valve so that water can be completely shut off when the office is unoccupied.
4. Firmly plug the foot control pedal connector to the foot pedal connection on the back of the unit.
5. Connect handpiece assembly at the front of the unit and place handpiece in holder.
6. Connect the power cord from the external power supply into the back of the main unit then plug into power outlet.

Fig. 1

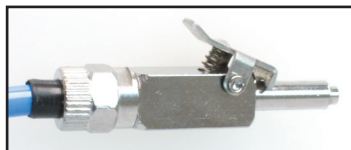
Fig. 2

OPERATION

Caution

- Use personal protective wear during use.
- Do not use ultrasonic scaler if water is not continuously flowing to the insert tip.

DAILY PREPARATION

1. Open manual shut-off valve on office water supply.
2. Set power setting to minimum setting.
3. Turn the system ON using the ON/OFF power switch at back of unit. Verify light on power control is on.
4. Turn water flow adjustment dial to maximum setting.
5. Hold handpiece (without a tip installed) over sink or drain. Activate foot pedal to flush water line for 30 seconds.

SCALING

Fig. 3

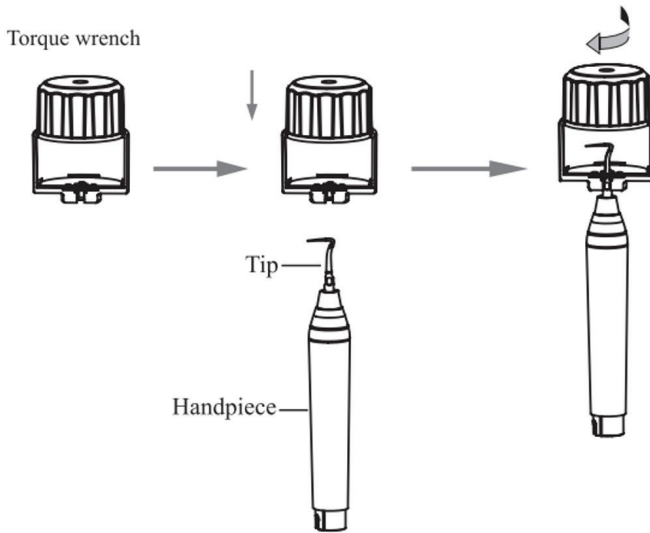
Fig. 4

1. Slide sterilized autoclavable handpiece until secure (Fig 3).
2. Insert tip in torque wrench, place on top of handpiece thread and turn clockwise (Fig. 4).
3. Set power and water flow dials to desired settings. Hold handpiece over sink and activate system by depressing foot control halfway. Verify water is flowing to end of insert tip.
4. Adjust water flow to ensure adequate cooling and irrigation for selected power setting.
5. It is recommended to begin procedure using minimum power and to increase as necessary. Use a light touch while scaling and move handpiece in a circular or wave pattern.

CRUISE CONTROL

- Once desired power and water flow have been selected press the cruise control button to activate cruise control mode which eliminates the need to hold the foot pedal down. While activated "Cruise" will be lit up.
- To deactivate cruise control mode either press the cruise control button or tap the foot pedal.

MAINTENANCE / STERILIZATION

Caution

- Sterilize tips and handpiece by autoclave before first use and after each patient.
- Do not immerse handpiece or sheath in liquid disinfectant.
- The device is not protected against penetration of liquids. Do not spray liquids directly onto the surface of the device.
- Do not clean handpiece with an ultrasonic cleaner

CLEANING & STERILIZATION

1. Before first use and after each procedure, remove tip and sterilize according to manufacturer instructions.
2. Set water flow adjustment dial to maximum setting, hold handpiece over sink or drain, and activate foot pedal for 30 seconds to purge water line.
3. Remove handpiece and tip (Fig 3).

- Clean work area surfaces and Acclean Piezo Ultrasonic Scaler (including controls, cables, foot pedal, and tubing) with a non-immersion disinfecting solution. When cleaning the Acclean Piezo Ultrasonic Scaler, moisten towel with solution and wipe down unit. Discard towel in biohazard bin. CAUTION: Do not apply disinfecting solution directly to Acclean Piezo Ultrasonic Scaler.
- Place handpiece in a sterilization pouch and seal.
- Steam autoclave at 273 °F (134 °C). Highest sterilization temperature is 280.4 °F (138 °C). Sterilization time should be at least 4 mins at a pressure of 29 psi (2.0 bar) ~ 33 psi (2.3 bars). Maximum allowable sterilization time is 20 mins at 273 °F (134 °C)

WATER LINE FILTER MAINTENANCE

NOTE: It is recommended that you replace water filter every 3 months or when filter becomes discolored.

Though your system comes with a water filter, it is recommended that you also use a filter in your dental water system supply line.

- Confirm power switch on back of main unit is in the OFF position.
- Close manual shut-off valve on office water supply.
- Disconnect water supply tube from water source.
- Grasp sleeve clamp on both sides of filter and twist counterclockwise to disconnect filter (Fig. 4).
- Remove filter from both sides of water supply tube.
- Install replacement filter onto water supply tube as shown in Figure 5.
- Grasp sleeve clamp on both sides of filter and twist clockwise to hand tighten (Fig 4). Do not over tighten.
- Reconnect water supply tube and follow instructions in Daily Preparation section to bleed air and test for leaks.

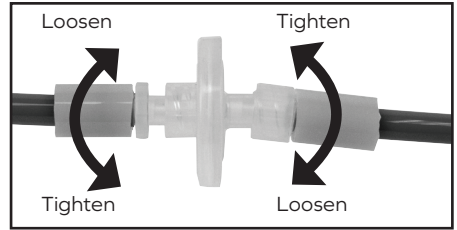
Fig. 4

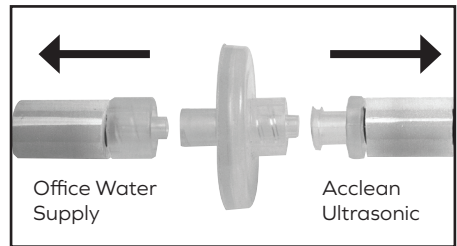
Fig. 5

STORAGE

- The equipment should be handled carefully and lightly. Be sure that it is far from the vibration, and is installed or kept in a cool, dry and ventilated place.
- Don't store the machine together with the articles that are combustible poisonous, caustic, or explosive.
- This equipment should be stored in a room where the relative humidity is 10% ~ 93%, atmospheric pressure is 10 psi (70kPa) to 15 psi (106kPa), and the temperature is -4°F (-20°C) ~ 131°F (55°C).
- Please turn off the power switch and pull out the power plug when the equipment is not used. If the machine is not used for a long time, please make it get through to the power and water once per month for five minutes.

ACCESSORIES

570-3469	Optic Handpiece	570-1143	Non-Optic Handpiece
570-0397	Water Supply Tube with Filter and Quick Disconnect	570-3472	Torque Wrench
570-1144	Foot Pedal	570-3471	Power Cord
570-0396	Replacement Water Filters (4 pack)		

ACCLEAN ULTRASONIC TIPS

570-1145	Universal Scaling Tip
570-1146	Heavy Scaling Tip
570-1148	Slim Tip

TROUBLESHOOTING

Problem	Cause	Solution
The scaling tip doesn't vibrate and there is no water flowing out when stepping on the foot switch.	Power plug is not securely connected.	Check the power plug connection at back of unit to confirm it's fully sealed.
	Foot pedal is not securely connected.	Check the foot pedal connection at back of unit to confirm it's fully sealed.
	The fuse in the adapter or main unit is broken.	Contact Henry Schein Handpiece & Small Equipment Solutions.
The scaling tip doesn't vibrate but there is water flowing out when stepping on the switch.	The tip is not securely connected.	Screw the tip on the handpiece tightly.
	The connection between the handpiece and the circuit board is not secure.	Contact Henry Schein Handpiece & Small Equipment Solutions.
	Something wrong with the handpiece.	Contact Henry Schein Handpiece & Small Equipment Solutions.
	Something wrong with the cable.	Contact Henry Schein Handpiece & Small Equipment Solutions.
The scaling tip vibrates but there is no spray when stepping on the foot switch.	The water control is not opened.	Turn the water control knob to open [1].
The vibration of the tip becomes weak.	The tip is not securely connected.	Screw the tip on the handpiece tightly.
	The coupling between the handpiece and the cable isn't dry.	Dry the area.
	The tip is damaged.	Replace the damaged tip.
There is water seeping from the coupling between the handpiece and cable.	The O-ring is damaged.	Replace the O-ring.
There is water flow after the power is turned off.	There is impurity in the solenoid valve.	Contact Henry Schein Handpiece & Small Equipment Solutions.
The handpiece generates heat.	Insufficient water flow.	Turn the water control switch increase water flow [1].
	The potentiometer is damaged.	Contact Henry Schein Handpiece & Small Equipment Solutions.
The amount of water flow is insufficient.	The water control valve is closed or not opened enough.	Turn the water control switch increase water flow [1].
	The water pressure is too low.	Increase the external water pressure.
	The water tubing is jammed.	Clean water tubing with multi-function syringe.
The digital handpiece control has seized.	The potentiometer is damaged.	Contact Henry Schein Handpiece & Small Equipment Solutions.
The u-file doesn't vibrate.	The screw is loose.	Tighten screw.
LED light doesn't work	Poor contact	Contact tightly
	Something wrong with LED light	Change LED light

If the problem still can't be solved, please contact Handpiece & Small Equipment Solution.

Troubleshooting Notes

1. The water control knob can adjust the water flow in the direction of the accompanying symbol on the unit.
2. If the scaling tip has been screwed on tightly and there is an acceptable amount of water spraying, the following phenomena show that the scaling tip is damaged:
 - a. The vibrating intensity and the amount of pulverization weakens.
 - b. A buzzing noise emits during operation.

SPECIFICATIONS

Input Power: 110V~ 50Hz/60Hz

Current: 1.3 A Max

Main Unit Input: 24V~ 50Hz/60Hz

Output Power: 20 W Max

Weight: 1.43 lbs

Tip Frequency: 28 ±3 kHz

Water Pressure: 0.01 MPa to 0.5 MPa

Dimensions: 7.68" x 7.09" x 2.91"

SERVICE

For items that require service, including warranty service, please send to Handpiece & Small Equipment Solutions

Handpiece & Small Equipment Solutions

620 S. Placentia Ave.

Placentia, CA 92870 USA

Phone # 800-235-6337

WARRANTY

This product is designed for use in a dental office and this warranty is not applicable to other uses. This product is warranted against manufacturing defects in materials and workmanship for 1 year from date of purchase.

Henry Schein® will repair or replace product at its sole discretion. This warranty shall be limited to replacement or repair of the product or its parts and shall not extend to any other claims, including but not limited to loss of profit, cost of removal or replacement, incidental or consequential damages or other similar claims arising from use of this product. Damage to product resulting from acts of God, faulty installation, misuse, tampering, accident, abuse, negligence, alterations, unauthorized repairs or problems unrelated to materials and workmanship are not covered by this warranty.

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Henry Schein Inc.