# ACCLEAN<sup>®</sup> SONIC SCALER

Swivel - 900-4590  
4-hole - 900-4012  
2-Hole - 900-4011

## INSTRUCTIONS FOR USE

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**CAUTION:** Federal (US) law restricts this device to sale by or on the order of a licensed professional

Thank you for purchasing a Henry Schein<sup>®</sup> Sonic Scaler.  
To keep its performance, please read this instruction manual carefully and thoroughly before use.  
Mis-handling of this instrument may cause mechanical trouble and accidents during treatment.  
Keep this instruction manual for future reference.  
Specifications in this manual may be changed for improvement and modification without prior notice.  
This instrument must be used only by dental professionals and only for dental treatment.  
Sonic Scaler is referred to as "handpiece" throughout this instruction manual.

## 1. PREFACE

Please begin by familiarizing yourself with the following indicators. These indicators, which will be preceded by this symbol: , will appear throughout the manual and will indicate situations which could, potentially, harm a patient or damage the handpiece.

- WARNING**  
Indicates situation when the incorrect operation could cause serious damage or injury.
- CAUTION**  
Indicates situation when the incorrect operation could cause slight damage or minor injury.
- IMPORTANT**  
Indicates situation when the incorrect operation or maintenance could cause damage or malfunction of the device.

## 2. CAUTIONS FOR USE

### CAUTION

- This product must be used only by qualified dental professionals and only for dental treatment.
- Do not modify this product.
- In case of malfunction, send to Henry Schein<sup>®</sup> ProRepair for repair. Do not attempt to repair on your own.
- In case of deterioration, do not use this product.
- Do not use this product if it is not maintained or checked properly.
- Keep the instructions for use for correct and safe use.
- Do not use a scaling tip with edge (No.2 tip) at sites such as subgingival contacting soft tissues. The soft tissues may be damaged.

## 3. INTENDED USE


This product is intended to remove dental plaque, calculus and stain from tooth surface and tooth root by dental professionals during dental treatment.

## 4. CHARACTERISTICS


- The handpiece can be used with much less stimulus on patients who cannot bear the irritation of the electric vibration of an ultrasonic scaler.
- The scaling tips generate less heat than ultrasonic scalers and require less cooling spray. This allows a clearer vision of the treatment site.
- The three-dimensional movement of the tips achieves efficient scaling and affects the tooth structure much less than ultrasonic scalers.
- The handpiece is compact and handy, directly attachable to dental unit.
- Driven by compressed air and water from the dental unit, the handpiece neither affects other medical devices nor is it affected by electromagnetic waves.

## 5. CONTENTS

The following components are included in the package. Upon opening, confirm whether all the components are included in the package. If something is missing, contact your dealer immediately.

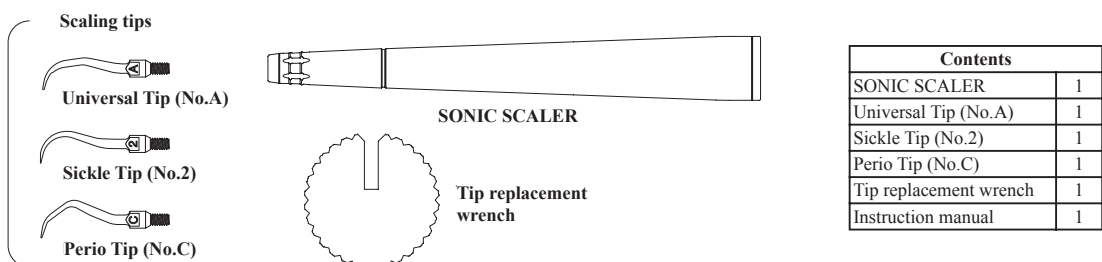## 6. PART DESCRIPTION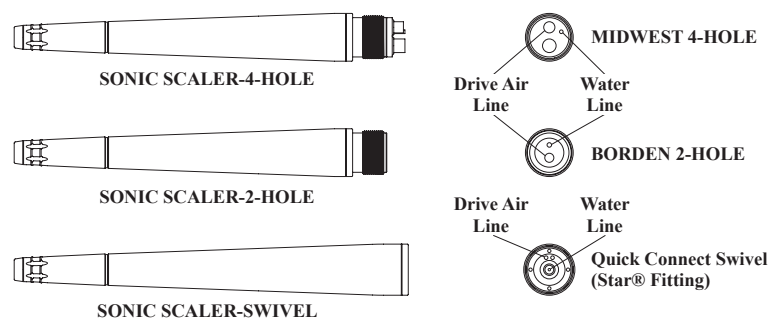

## 7. HOW TO USE

### 7.1 Before use:

#### CAUTION

- This product should only be used for the prescribed purpose as outlined in section 3.
- When using other products which are compatible (such as peripheral devices and instruments) with this product, observe instructions for use of such devices or instruments.

- Opening the package:**  
When unpacking the handpiece for the first time, confirm that all items in 5. DELIVERY PACKAGE is included in the package. If something is missing, contact your dealer immediately.
- Lubrication (before first use and in case the handpiece is not used for more than one week):**  
Lubricate the handpiece according to the instructions in 8.3 Lubrication.
- Sterilization (sterilize before each patient use):**  
According to instructions in 9. CLEANING AND STERILIZATION, clean the handpiece, scaling tips and replacement wrench with alcohol and sterilize in an autoclave.

#### WARNING


The handpiece, scaling tips and replacement wrench are cleaned, but not sterilized, before packaging. Clean them with alcohol and sterilize in an autoclave before first use.

- Checking your air compressor:**  
To supply the handpiece with clean and dry air, drain air filter of the compressor in your dental office. Do not supply with compressed air containing moisture. For drainage and regular maintenance of your compressor, please refer to your air compressor's instruction manual. Confirm the compressor is maintained according to its instructions for use.

### 7.2 Setting air pressure (first use):

- Preparation of handpiece:**  
Confirm the handpiece is disconnected from the dental unit hose. If connected, disconnect from the dental unit hose first.

- Attaching a pressure gauge:**
  - Attach a pressure gauge to your dental unit hose. Confirm the gauge is firmly inserted.
  - Attach the handpiece to the coupling of the pressure gauge. Confirm the handpiece is firmly inserted.

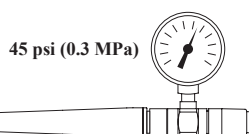- Setting air pressure:**
  - Turn off the water supply of your dental unit. Turn the handpiece to your cuspidor and depress your foot control.
  - Adjust the air pressure of the unit until the gauge shows 45 psi (0.3 MPa, 3.0 bar, 3.0kgf/cm<sup>2</sup>).
  - Detach the pressure gauge and handpiece.

#### WARNING

- Always connect the handpiece to your hose connector firmly. If the handpiece is improperly attached it may be forced off the connector by the compressed air and cause an accident.
- While the handpiece is in operation, never turn the ring nut of your dental unit hose. The handpiece may be forced off the connector by the pressure of compressed air and cause an accident.
- Use this handpiece at working air pressure not exceeding 45 psi (0.3MPa, 3.0 bar, 3.0kgf/cm<sup>2</sup>). Use at an excessive air pressure level causes damage to the tooth surface, patient discomfort, and breakage to the instrument.

### 7.3 Attaching scaling tips:

#### 1) Scaling tips:

Three scaling tips are supplied as standard accessories. Each scaling tip has its specific application, as detailed below.


-  **Universal Tip (No.A):**  
For curved surfaces, anterior exposed root surfaces, posterior teeth with lingual inclination.
-  **Sickle Tip (No.2):**  
For mandibular anterior surfaces, lingual surfaces, in case thick hard band calculus adheres firmly.
-  **Perio Tip (No.C):**  
For subgingival and supragingival, effective for tar adherence, etc.

#### CAUTION

Do not use a scaling tip with edge (No.2 tip) at sites such as subgingival contacting soft tissues. The soft tissues may be damaged.

#### 2) Attaching scaling tip:

- Use sterilized scaling tips and replacement wrench only. Wear protective gloves when attaching.
- The handpiece must be disconnected from the dental unit hose when attaching and detaching a scaling tip. If connected, disconnect the handpiece from the dental unit hose first.
- Put a scaling tip onto the screw thread at the handpiece's nose. Turn the tip in the direction of the arrow in the illustration with fingers to screw in.
- Tighten firmly using smaller opening of the supplied wrench until fully seated. Do not apply excessive force.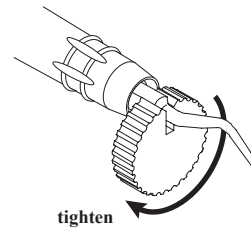

#### WARNING

- Use only original scaling tips. Use with non-original scaling tips can result in improper insertion or irregular vibration of the scaling tip, which will cause accidents induced by breakage of scaling tip.
- Attach instruments to handpiece firmly using the replacement wrench which is delivered with the handpiece. Confirm firm attachment for each patient. Loose attachment may cause accidents during treatment.
- When attaching and detaching instruments, take special care not to injure your hands and finger in order to prevent cross contamination.


#### CAUTION

- Do not screw in the scaling tips too tightly.
- Depending on the instruments used and degree of their wear, usage conditions such as power setting, the handpiece may generate louder noise and larger amplitude. Too large amplitude may cause damage on the tooth and gingiva.
- If the handpiece generates louder noise than usual, do not use since this may affect the hearing of the user and patient. Send to Henry Schein<sup>®</sup> ProRepair to be repaired.

### 7.4 Connecting to dental unit:

#### 1) SWIVEL Model:

Use a coupling compatible with the backend of this handpiece. For attachment of the coupling, follow the coupling's instructions for use. If the coupling has a light source, turn off the light on the coupling.

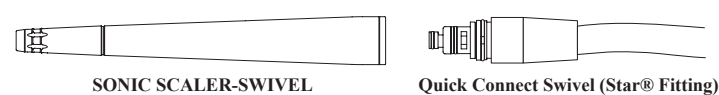#### 2) 4-HOLE & 2-HOLE Models:

Confirm the coupling of the handpiece and your dental unit hose is the same (4-HOLE & 2-HOLE).

- Align the backend of the handpiece and connector holes of your dental unit hose.
- Rotate the ring nut to attach the handpiece until fixed in the connector completely.
- Confirm the handpiece is firmly attached.

#### WARNING

- Use a coupling compatible with the backend of this handpiece. If an incompatible coupling is used, it may damage the handpiece and the coupling. Also, the handpiece may be forced off the coupling by the pressure of compressed air and cause accidents (For Quick Connect Swivel coupling).
- Always connect the handpiece to your hose connector firmly. If the handpiece is not properly connected it may be forced off the connector by the compressed air and cause accidents.
- While the handpiece is in operation, never turn the ring nut of your dental unit hose. The handpiece may be forced off the connector by the pressure of compressed air and cause accidents.

#### IMPORTANT

When using this handpiece, turn off the light of the coupling, otherwise the service life of the light source (bulb) in the coupling may be shortened.

### 7.5 Adjustment:

#### 1) Adjustment of water volume:

Set water supply volume from the dental unit sufficient to cool the scaling tip and the tooth being treated. For adjustment of water supply volume, refer to the instruction manual of your dental unit. The figure "GOOD" shows the best water supply condition.

#### WARNING

Supply sufficient water to cool the tooth surface and gingiva. Insufficient water will damage the tooth surface and gingiva as well as shorten the life of the instruments.

### 7.6 Treatment:

To avoid damaging the patient's tooth and the scaling tip, scaling should be done lightly. Select a scaling tip suitable to the treatment site. (See 7.3 step 1 Scaling tips)

#### CAUTION

- Do not use a scaling tip with edge (No.2 tip) at sites such as subgingival contacting soft tissues. The soft tissues may be damaged.
- Avoid applying a scaling tip at the same site for a long time.
- When applying a scaling tip to a root surface, apply in an angle parallel to the axis of the tooth as much as possible.
- Confirm no damage, no cracks, no breakage or no major corrosion is on the product. Verify safe operation before every use.

#### WARNING

- To avoid infection, wear suitable protective equipment such as mask, gloves and eyewear when using.
- To prevent unexpected accidents such as accidental swallowing of dropped or broken instrument, take precautionary measures such as rubber dam isolation.
- The amplitude of scaling tip may exceed 200 μ depending on the use environment such as which scaling tip is used, degree of wear or power setting. Take special care not to damage the tooth surface or gingival.
- To avoid inserting a scaling tip deeper than necessary and injuring bottom of periodontal pocket, grasp the depth of the pocket before treatment.
- Apply plenty of water to create an irrigation effect and avoid damaging tooth root surfaces.
- When activating the handpiece in periodontal pocket, enter the scaling tip into the pocket and grasp the depth of the pocket in advance.

### 7.7 After use:


#### 1) Detaching from dental unit hose:

- SWIVEL Model:**  
For detachment of the coupling, follow the coupling's instructions for use.
- 4-HOLE & 2-HOLE Models:**  
Detach the handpiece from the dental unit hose by rotating the ring nut.

#### 2) Detaching scaling tip:

Using the supplied wrench, turn the tip counter-clockwise to detach.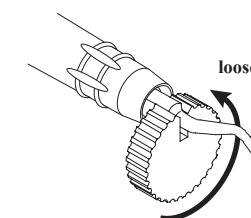

#### WARNING

When attaching and detaching instruments, take special care not to injure your hands or fingers in order to prevent cross contamination.

## 8. MAINTENANCE AND CHECKS

### CAUTION

- 1) This product has to be maintained before every sterilization and after clinic hours according to this instruction without fail.
- 2) To use safely, check and maintain the product after clinic hours according to these instructions.

#### 8.1 Cleaning:


- 1) Detaching handpiece and scaling tip:  
Detach the handpiece and scaling tip according to 7.7 After use.
- 2) Cleaning:  
Wipe off saliva and blood remaining on the handpiece and scaling tip with alcohol.

### CAUTION

Do not use ultrasonic cleaners to clean the handpiece. This will cause damage to the handpiece.

#### 8.2 Draining:

Drain remaining water from the water line of the handpiece by blowing compressed air with a three-way syringe in the backend of the handpiece as shown on the right.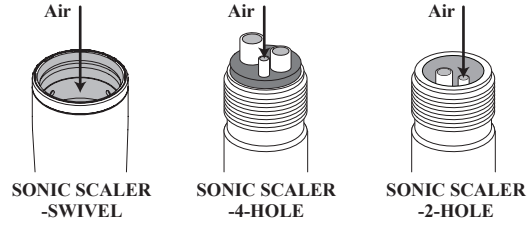

### IMPORTANT

Drain the handpiece before sterilizing and after clinic hours.

#### 8.3 Lubrication:


- 1) When to lubricate:  
Use lubrication spray (Henry Schein® Spray & Clean - 102-8721) for high speed air turbine handpiece. Lubrication must be done in the following cases:  
-- before first use  
-- before sterilization in autoclave  
-- twice a day after morning and afternoon clinic hours  
-- if the handpiece is not used more than one week

### CAUTION

The handpiece has to be lubricated according to the instructions in this section. Insufficient lubrication will cause mechanical troubles or shorter life of the handpiece.

- 2) Before lubrication:  
Detach the handpiece and scaling tip according to 7.7 After use.

- 3) Lubrication:  
a) SWIVEL Model:  
Attach a nozzle compatible with the scaler backend to the lubricant spray. Insert the nozzle of the spray into the handpiece backend. Spray for one second holding the handpiece firmly.  
b) 4-HOLE & 2-HOLE Models:  
When using a lubrication spray, insert the nozzle into the drive air line at the handpiece backend. Spray for about one second. Hold the handpiece and spray can tightly.

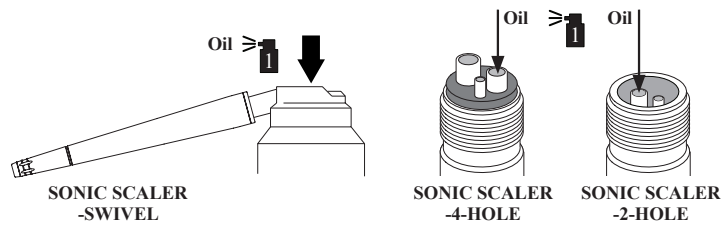### CAUTION

- 1) When lubricating with a lubrication spray, hold the spray can upwards. If the can is held downwards, the spray spouts only the gas and no proper lubrication will be done and may damage the handpiece and shorten its life.
- 2) For lubrication with spray, use a spray nozzle which is compatible to the backend of your handpiece only. Use of incompatible nozzle will cause improper or insufficient lubrication and will cause trouble or shorter life of the handpiece.
- 3) Lubricating sprays are flammable products in which high-pressure propellant gas is used. When using a lubricating spray, follow the instructions of the spray manufacturer.
- 4) By the nature of the material, the vibrator in the handpiece is subject to rust. Insufficient or incomplete lubrication and supplying moistened air, due to insufficient or incomplete draining of your compressor (see 7.1 step 4 Checking your air compressor), will cause deterioration due to rusting.

- 4) After lubrication:  
Connect the handpiece to your dental unit hose according to 7.4 Connecting to dental unit and run for one minute to distribute the lubricant. Wipe off excess lubricant with clean gauze or tissue paper.

#### 8.4 Drying scaling tip:

- 1) Detaching scaling tip:  
Detach scaling tip from the handpiece according to 7.7 step 2 Detaching scaling tip.
- 2) Drying:  
The following parts must be dried with special care using air from three-way syringe:  
-- Scaling tip surface and spray hole,  
-- Screw thread of handpiece nose.

### WARNING

- 1) To prevent rusting, remove scaling tip from the handpiece and dry thoroughly with air of three-way syringe after use.
- 2) The scaling tips are made from rustable metal. Insufficient drying will cause rust and make the scaling tips easily breakable and may cause accidents.

#### 8.5 Checking air compressor:

Drain the air filter of the air compressor in your dental clinic regularly to supply the handpiece with clean and dry air. Do not supply with compressed air containing moisture. For drainage and regular maintenance of your compressor, please refer to the air compressor's instruction manual. Confirm your compressor is maintained according to its instruction manual.

## 9. CLEANING AND STERILIZATION

#### 9.1 Cleaning:

- 1) Cleaning:  
Clean the handpiece and scaling tips according to 8.1 Cleaning.
- 2) Draining:  
Drain the water line of the handpiece according to 8.2 Draining.
- 3) Lubrication:  
Lubricate the handpiece according to 8.3 Lubrication.

### WARNING

- 1) To avoid infection, wear suitable protective wears such as mask, gloves and eyewear when using.
- 2) Sterilize by autoclave after each patient use.

#### 9.2 Sterilization:

- 1) The handpiece, scaling tips, and replacement wrench are autoclavable:
- 2) Sterilization:  
Put the handpiece, scaling tips, and replacement wrench into non-broken sterilization bags (not included in the delivery package) separately and seal. Place the sealed instruments on a sterilization tray to avoid direct contact with sterilizer's inner chamber surface, and put them into the autoclave. Steam sterilize for 3 minutes at 273°F (134°C) in an autoclave. The handpiece, scaling tips, and replacement wrench are autoclavable up to 135°C (275°F). Follow your local regulation for sterilization.

### CAUTION

- 1) The maximum sterilization temperature is 135°C (275°F). Steam sterilize for 3 minutes at 273°F (134°C) in an autoclave. Skip dry cycle. Follow your local regulations for sterilization.
- 2) Do not sterilize the product with instruments on which some medicine or chemical agent remains.
- 3) Do not immerse the handpiece in disinfectant liquids.
- 4) Do not use alkaline disinfectant and oxidized water (hard acid water or super acid water) for disinfection.
- 5) Dry heat sterilizers generate high heat that exceeds the maximum sterilization temperature of the product. Do not use in order to avoid mechanical trouble or damage.

- 3) After sterilization:  
Keep the handpiece in sterilization bag until ready for use.

#### 9.3 Storage:

After sterilization, dry the handpiece, scaling tips, and replacement wrench naturally and completely. Keep them in the sealed sterilization bags.

### IMPORTANT

- 1) The handpiece may become discolored depending on the sterilization method or sterilizer's condition.
- 2) Before storage, dry the handpiece, scaling tips, and replacement wrench thoroughly. Remaining water due to insufficient drying will cause rusting or discoloration.
- 3) Store and take care of the product properly to keep it out of reach of persons other than dental professionals.
- 4) Do not store the product in a place where it may be exposed to water.
- 5) Keep the product in a place where it is not adversely affected by barometric pressure, temperature, humidity, ventilation, strong sunshine, dust, salt or sulphurous air.
- 6) Keep the product in places without vibration or shock.
- 7) Do not store the product in a place where chemical agents are kept or some gas exists.

## 10. REPLACEMENT OF CONSUMABLE PARTS, DISPOSAL

#### 10.1 Replacement of scaling tip:

- 1) When to replace scaling tips:  
Scaling tips will be worn through normal use, and it will affect the handpiece's working frequency and efficiency. Check scaling tip's wear regularly.

### WARNING

Scaling tips in the following conditions will break easily and may cause unexpected accidents or injuries. Do not use such tips and replace with a new tip immediately.

- Tips with edge worn through scaling.
- Corroded or rusted tips.
- Tips deformed on impact, e.g. dropped on the floor.
- Sharpened, altered, or deformed tips.

#### 2) How to check scaling tip wear:

- a) Detach the scaling tip from the handpiece.
- b) Put the scaling tip on the drawing to check the wear.
- c) If the tip is worn shorter than the marked line, it must be replaced. Use a new scaling tip.

#### 3) How to replace scaling tip:

- a) Detach the scaling tip from the handpiece according to 7.7 step 2 Detaching scaling tip.
- b) Check scaling tip wear according to 10.1 step 2 How to check scaling tip wear.
- c) Attach a new tip according to 7.3 step 2 Attaching scaling tip.

#### 10.2 Disposal of handpiece and accessories:

Dispose of the handpiece and accessories according to your local rules and regulations.

## 11. TROUBLESHOOTING

If this product does not operate as stated in this manual, check your problem according to the list below. If your issue is not corrected, or in case of a problem that is not listed below, contact Henry Schein® ProRepair.

### CAUTION


In case of trouble, the product must not be repaired by the user. Contact Henry Schein® ProRepair for repair.

SYMPTOMS	POSSIBLE CAUSES	CHECK PROCEDURES AND CORRECTIONS
1) No vibration	1. No air supply. 2. Handpiece air line is blocked. 3. Air supply system of the dental unit is not working. 4. Vibration source is broken. 5. Right after lubrication.	1. Check air pressure of your dental unit. 2. Ask your dealer for repair. 3. Check with other handpiece to see if the air supply system works correctly or not. 4. Ask your dealer for repair. 5. Run one minute after lubrication.
2) Weak power	1. The attached instrument is loosened. 2. Low air pressure. 3. Broken or worn scaling tip. 4. Vibration source is broken. 5. Vibration source inside is unclean.	1. Tighten the attached instrument firmly with supplied replacement wrench. 2. Check the pressure with a gauge and adjust to the proper setting (45 psi, 0.3 MPa, 3.0 bar, 3.0 kgf/cm <sup>2</sup> ). 3. Replace the scaling tip with new one. 4. Ask your dealer for repair. 5. Lubricate the handpiece according to 8.3 Lubrication.
3) No water supply	1. Water shut off. 2. Water doesn't reach dental unit hose. 3. Water port of scaling tip is clogged. 4. Handpiece water line clogged.	1. Check the water supply control system and water inlet of your dental unit. 2. Check the water supply control system and water inlet of your dental unit. 3. Blow air into the water port of the scaling tip and clean with a three-way syringe. 4. Ask your dealer for repair.
4) Loud noise	1. Loose scaling tip. 2. Broken or worn scaling tip. 3. Vibration source is broken. 4. Right after lubrication.	1. Tighten scaling tip firmly with supplied replacement wrench. 2. Replace the scaling tip with new one. 3. Ask your dealer for repair. 4. Run one minute after every lubrication.
5) Water and/or air leaks from handpiece connection	1. Seal between tubing and water port of handpiece is imperfect. 2. Scaling tip is not fixed firmly.	1. Tighten coupling firmly. If the leak continues, replace the gasket. 2. Tighten scaling tip firmly with the supplied replacement wrench.
6) Scaling tips cannot be attached	1. Use of non-original scaling tip.	1. Use original scaling tip.

## 12. SPECIFICATIONS

Specifications			
Product name	Henry Schein® SONIC SCALER		
Model	SONIC SCALER-SWIVEL	SONIC SCALER-4-HOLE	SONIC SCALER-2-HOLE
Backend configuration	Quick Connect Swivel (Star® Fitting)	MIDWEST 4-HOLE	BORDEN 2-HOLE
Frequency range	about 6,000 Hz		
Working air pressure	45 psi (0.3 MPa, 3.0 bar, 3.0 kgf/cm <sup>2</sup> )		
Air consumption volume	24 L / min		
Cooling water volume	25 cc / min (Water pressure 29 psi (0.2 MPa))		
Weight (without scaling tip)	2.2 oz (60 g)	2.1 oz (59 g)	
Total length (without scaling tip)	5.2 inch (131 mm)	5.1 inch (129 mm)	4.8 inch (123 mm)

## 13. COMPONENT PARTS

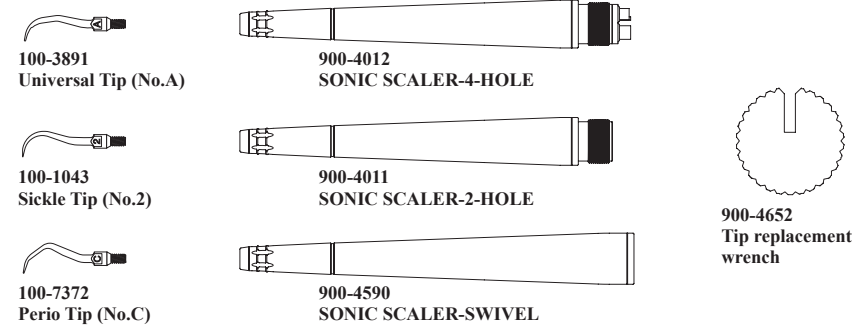## 14. SERVICE

For items that require service, including warranty service, please send to Henry Schein® ProRepair.

Henry Schein® ProRepair  
620 S. Placentia Ave.  
Placentia, CA 92870 USA  
Phone # 714-579-0175  
Fax # 714-579-0186

## 15. WARRANTY

This product is designed for use in a dental office and this warranty is not applicable to other uses. This product is warranted against manufacturing defects in materials and workmanship for six (6) months from date of purchase (excluding scaling tips).

Henry Schein® will repair or replace product at its sole discretion. This warranty shall be limited to replacement or repair of the product or its parts and shall not extend to any other claims, including but not limited to loss of profit, cost of removal or replacement, incidental or consequential damages or other similar claims arising from use of this product. Damage to product resulting from acts of God, faulty installation, misuse, tampering, accident, abuse, negligence, alterations, unauthorized repairs or problems unrelated to materials and workmanship are not covered by this warranty.

Distributed by (in US only)  
**HENRY SCHEIN INC.**  
(manufacturer)  
135 DURYEA ROAD  
MELVILLE, NY 11747 USA

EC REP

**HENRY SCHEIN U.K. HOLDINGS LTD**  
MEDCARE HOUSE, CENTURION CLOSE  
GILLINGHAM BUSINESS PARK  
GILLINGHAM, ME8 0SB U.K.

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Swivel - 900-4590  
4-hole - 900-4012  
2-Hole - 900-4011

## INSTRUCTIONS FOR USE

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Read before use. Keep in a safe place for future reference.

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