PRÉPARATION

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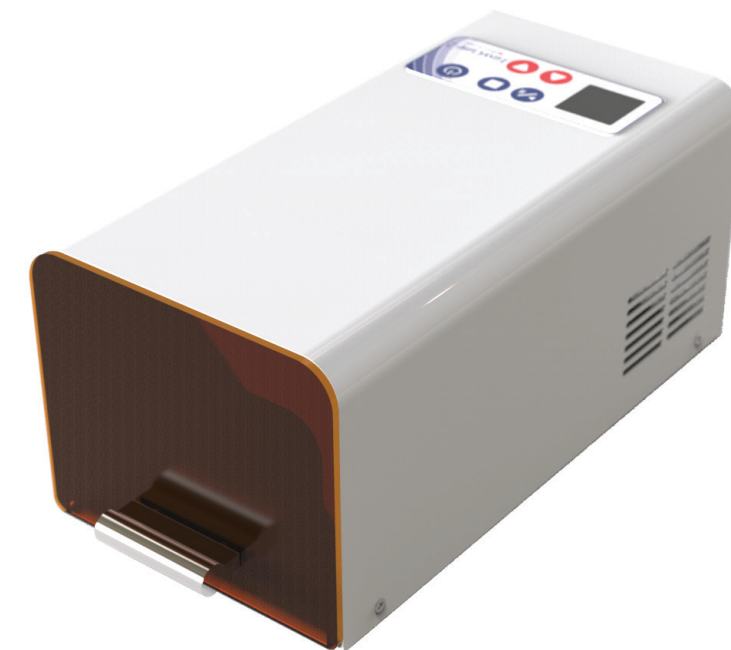
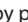
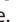
Made in Taiwan
Fabricado en Taiwan/Fabriqué aux Taiwan

SET UP

1. Unpack curing unit, place on a firm, flat surface.
2. Remove protective film on both sides of door and packaging foam from curing chamber.
3. Attach power cord and plug in unit.

OPERATION

1. Press the  button to power unit on. Curing time (in minutes) will display. Curing time will display 5 minutes for first use.
2. Adjust curing time in increments of 10 seconds on control panel by pressing the  or  button. The maximum curing time allowed is 20 minutes.
3. Insert composite/resin trays and close chamber door.
NOTE: Unit will not function with chamber door ajar and  will appear on the display. To optimize curing location, place tray insert inside curing chamber as necessary.
4. Press  button on control panel to begin curing cycle.
5. To interrupt curing cycle, press the  button. Pressing  again will continue the curing cycle. Pressing  will reset the curing time to the original selection. The unit will “beep” for the last 5 seconds of curing time.
6. After use turn off power. Last used curing time will be stored in memory and will display on next use. The unit will power down after 3 minutes on standby.

SPECIFICATIONS	
Model Number:	570-2721
Voltage:	110 V 50/60 Hz
Lamp:	3W, 4 UV LED & 2 Blue LED
Wavelength Peak:	UV: 360~420 nm (Peak 390~400 nm) Blue LED: 420~490 nm (Peak 455~465 nm)
Chamber Size:	6.5 in (l) x 4.5 in (w) x 3.5 in (h)
Weight:	2.7 kg

TROUBLESHOOTING

Warning	Problem	Solution
	The chamber door is open.	Close door completely.
	The LED lamp is not on.	Contact Henry Schein ProRepair.
	The LED lamp overheated.	The unit will automatically turn off all lights until cooled down. When “Er” and time alternately flash on the display, you can press  or  button to remove the warning. You may now resume curing.
N/A	Unit does not power on.	Confirm power cord connection.

SERVICE

For items that require service, including warranty service, please send to Henry Schein ProRepair.

Henry Schein ProRepair
620 S. Placentia Ave.
Placentia, CA 92870 USA
Phone: +1-714-579-0175
Fax : +1-714-579-0186

WARRANTY

This product is designed for use in a dental laboratory / dental office and this warranty is not applicable to other uses. The unit is warranted for twelve (12) months from date of purchase against manufacturing defects in materials and workmanship; Henry Schein® will repair or replace the unit at its sole discretion. This warranty shall be limited to replacement or repair of the unit or its parts and shall not extend to any other claims, including but not limited to loss of profit, cost of removal or replacement, incidental or consequential damages or other similar claims arising from the use of this product. Damages to the product resulting from acts of God, faulty installation, misuse, tampering, accident, abuse, negligence or unauthorized repairs or alternations unrelated to problems with materials and workmanship are not covered by this warranty.

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