Essentials

PRO 2 HIGH-SPEED HANDPIECE

INSTRUCTIONS FOR USE

Read before use. Keep in a safe place for future reference.

CAUTION: Federal (US) law restricts this device to sale by or on the order of a licensed professional

 **Caution**

- Do not run handpiece without a bur locked in chuck.
- Do not disassemble, modify or alter handpiece.
- Do not press push-button end cap during use since this could result in OVERHEATING of handpiece head. Special caution must be exercised during use to keep cheek tissue away from push-button end cap. Contact with cheek tissue may cause push-button to depress and may result in burn injury to patient.
- Never attempt to remove bur while handpiece is running.
- If any abnormalities are found, stop using immediately and contact Henry Schein® ProRepair.
- Exceeding recommended air pressure may result in premature cartridge failure.

SPECIFICATIONS

Description	Pro 2	Pro 2M	Pro 2S	Pro 2SM
Item Code	570-1431	570-1432	570-1433	570-1434
Air Pressure	32-35 psi / 2.2-2.5 bar		2.6-3.0 bar	2.6-3.0 bar
Speed	300,000-350,000 rpm	350,000-400,000 rpm	300,000-350,000 rpm	350,000-400,000 rpm
Connection	4-Hole Fixed		NSK QD-J coupler	
Power	21 W	18 W	21W	18W
Weight	60 g	57.5 g	62.1g	59.6g
Head Height	12.4 mm	12.1 mm	12.4 mm	12.1 mm
Head Diameter	12 mm	10.5 mm	12 mm	10.5 mm

OPERATION

Attaching Handpiece

Fixed 4-hole

1. Align tubes on handpiece with holes in delivery system tubing (Fig. 1).
2. Thread tubing nut to bottom of handpiece and finger tighten (Fig. 2).

Swivel Coupler

1. Align Tubes on coupler with holes in delivery system tubing (Fig 3).
2. Thread tubing nut to bottom of coupler and finger tighten (Fig 4).
3. Pull back coupler ring and slide handpiece onto coupler. Release ring to secure.

Removing Handpiece

Fixed 4-hole

1. Unscrew tubing nut from bottom of handpiece (Fig. 2).

Note: Do not hold or pull on hose.

Swivel Coupler

1. Pull back on coupler ring.
2. Pull handpiece straight off.

Note: Do not hold or pull on hose.

Fig. 1

Fig. 2

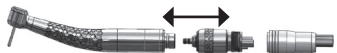
Fig. 3

Coupler Ring

Fig. 4

Changing Burs (Fig. 5)

Caution

- Only use ISO burs with shank diameter of .0626" - .0630" (1.59-1.60 mm).
- Do not use bent, worn or damaged burs.
- Always keep burs clean. Dirt and debris on shank may cause clogging/sticking, loss of grip or premature wear of chuck.

Note: For use with standard and short shank burs.

Removal

1. Press push-button on end cap with thumb.
2. Pull to remove bur.

Installation

1. Press push-button and insert bur into head until it stops.
2. Release pressure on button; bur should now be secure.
3. Firmly tug on bur to ensure that it is properly secured.

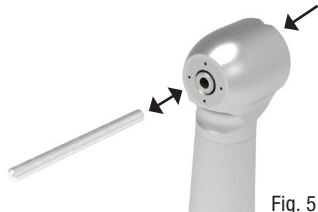
Fig. 5

MAINTENANCE

- Wear appropriate personal protective equipment during maintenance process. This should include eye protection and gloves.

Note: For best performance and extended handpiece life, use the Henry Schein® Maxima XTEND Handpiece Maintenance Machine to care for your handpieces. Cleaning and lubrication are required before each sterilization cycle.

Cleaning and Lubrication with Maxima XTEND Handpiece Maintenance Machine

Handpiece maintenance

1. Remove bur.
2. Detach handpiece from delivery system tubing.
3. Attach handpiece onto appropriate connector. Run maintenance cycle. See Maxima XTEND instructions for further information.
4. Remove handpiece from Maxima XTEND Maintenance Machine after cycle completes.
5. Wipe excess cleaner/lubricant from handpiece with dry, lint-free cloth.

Push-button Chuck Maintenance

Note: It is recommended that push-button chuck be cleaned every 3 - 4 maintenance cycles.

1. Remove bur.
2. Place chuck opening against tapered end of chuck cleaning nozzle.
3. Press start. See Maxima XTEND instructions for additional information.

**Manual Cleaning and Lubrication with Henry Schein® Spray & Clean (102-8721)
Handpiece Maintenance (Fig. 6)**

Fig. 6

1. Disconnect handpiece from air supply.
2. Insert nozzle into handpiece drive air tube and spray for approximately 3 seconds.
3. Attach the handpiece to an air supply or purge station and run for approximately 20 seconds to remove debris and excess lubricant. A spray cover or paper towel should be used to contain expelled lubrication.

Push-button Chuck Maintenance (Fig. 7)

Note: It is recommended that push-button chuck be cleaned every 3 - 4 maintenance cycles.

1. Remove bur.
2. Detach handpiece from air supply.
3. Insert spray nozzle tip of Henry Schein Spray & Clean into chuck and dispense spray for 1 - 2 seconds.
4. Work chuck mechanism to distribute cleaner/lubricant by repeatedly pressing button.
5. With push button depressed, insert and remove bur several times to break-up remaining debris.
6. Wipe off excess cleaner/lubricant with a dry, lint-free cloth.

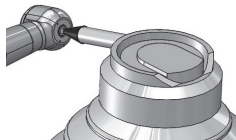
Fig. 7

Cleaning Water Spray Ports (Fig. 8)

Note: It is recommended that spray ports be periodically cleaned.

1. Carefully clean ports using cleaning wire supplied with handpiece.
2. Connect handpiece and run with water flow activated to confirm ports have been fully cleared of debris.
3. Repeat steps 1 and 2 as necessary.

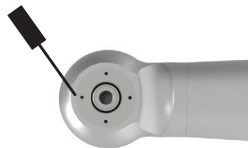
Fig. 8

STERILIZATION

WARNING

- Sterilize by autoclave after each patient use.
- Sterilization is required before first use.

Caution

- Wear appropriate personal protective equipment during sterilization and lubrication process. This should include eye protection and gloves.
 - Do not use ultrasonic bath or cold sterile/disinfectant chemicals to clean handpiece.
 - Do not autoclave handpieces together with other instruments.
 - Do not use dry heat sterilization.
 - Keep handpiece in sterilization pouch until ready to use.
 - Do not cool handpiece manually as damage may occur.
1. Remove bur, then wipe external surface with dry paper towel to remove any excess lubricant and debris. Place paper towel in biohazard bin when finished.
 2. Place handpiece into sterilization pouch and seal. Bag handpieces individually; no more than 1 per pouch.

3. Steam autoclave for a minimum of 15 minutes at 270° F (132° C). Let autoclave cycle completely. Do not remove before hand-piece is completely cool and pouch is dry.

ACCESSORIES

570-0072 Maxima XTEND Handpiece Maintenance Machine

102-8721 Henry Schein® Spray & Clean

112-7372 Essentials Pro Non-Optic 4-Hole Coupler

SERVICE

For items that require service, including warranty service, please send to Henry Schein® ProRepair®.

Henry Schein ProRepair

620 S. Placentia Ave.

Placentia, CA 92870 USA

Phone # 714-579-0175

Fax # 714-579-0186

WARRANTY

These products are designed for use in a dental office and this warranty is not applicable to other uses. The products are warranted against manufacturing defects in materials and workmanship for 1 year from date of purchase.

Henry Schein® will repair or replace products at its sole discretion. This warranty shall be limited to replacement or repair of the products or their parts and shall not extend to any other claims, including but not limited to loss of profit, cost of removal or replacement, incidental or consequential damages or other similar claims arising from use of this product. Damage to products resulting from acts of God, faulty installation, misuse, tampering, accident, abuse, negligence, alterations, unauthorized repairs or problems unrelated to materials and workmanship are not covered by this warranty.

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