
Distributed by:

Henry Schein Inc.
135 Duryea Road
Melville, NY 11747 USA

MADE IN TAIWAN

REV. 2023/06

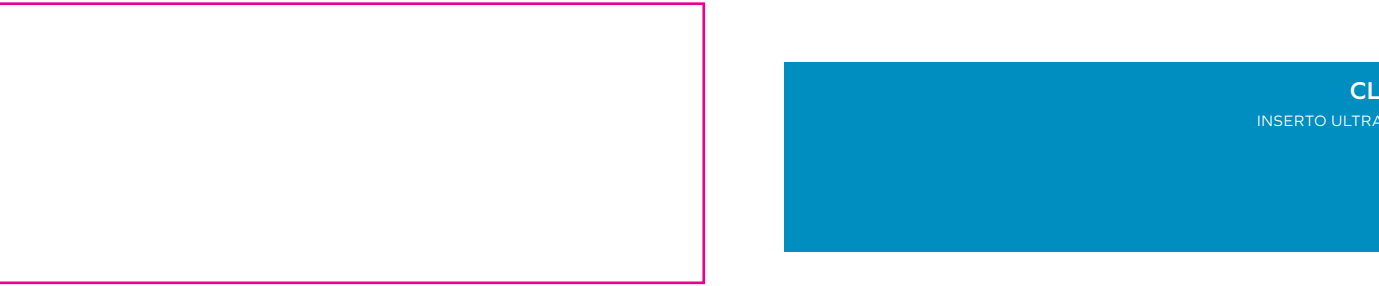ACCLEAN[®]

CLASSIC ULTRASONIC INSERT

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Flujo externo
Flux externe

WEAR GUIDE INSTRUCTION

Match the insert to the appropriate graphic. Hold the top of the grip flush against the edge of the wear guide. Rotate the insert so the tip is flat against the card. Evaluate the insert shape against the graphic. The blue line designates where the insert has lost 25% of its scaling efficiency. It is recommended the insert be disposed of when the tip touches the red line; at which point the insert has lost 50% of its scaling efficiency.

The use of worn tips can require excessive pressure at the treatment site, which results in discomfort for both the clinician and patient during the procedure. This guide should not be used for inserts that have been subjected to improper use or whose tips have been bent or reshaped as it will not provide reliable information.

CAUTION: Federal (US) law restricts this device to sale by or on the order of a licensed professional.

INDICATIONS FOR USE

This device is intended to be used by dental professionals for dental cleaning and periodontal (gum) therapy to remove calculus from the teeth.

CONTRAINDICATIONS

Acclean Ultrasonic Inserts should not be used for restorative dental procedures involving condensation of amalgam.

WARNINGS

- Acclean Ultrasonic Inserts with high usage which exceed the warranty period should be disposed to avoid possible in-use breakage and injury.
- Handle insert with care. Improper handling of insert, may result in injury and/or cross contamination, especially the tip.

- Failure to properly follow validated reprocessing method and techniques of Acclean Ultrasonic Inserts may lead to in cross contamination.
- Acclean Ultrasonic Inserts are intended to be used with Magnetostrictive Ultrasonic Scalers.

PRECAUTIONS

It is the responsibility of the Dental Healthcare Professional to:

- Check water spray to verify fluid is reaching the working end of tip. Adjust the water lavage control until the water flows with a rapid drip or small. Higher water flow settings provide cooler irrigation.
- Do not bend or reshape. Inserts should be handled with care at all times. Ultrasonic Insert tips that have been bent, reshaped or otherwise damaged are susceptible to in-use breakage and should be disposed and replaced immediately.

OPERATING INSTRUCTIONS

1. Fill entire handpiece with water.
2. Lubricate O-ring with water.
3. Gently twist the insert into the handpiece until the insert is fully seated in place.
4. Use the lowest effective power setting.
5. Adjust the water control until the water flows with a rapid drip or small spray.
6. Use a light but stable intraoral or extraoral fulcrum.
7. Adapt the insert so the point is not directed into root surface.
8. Use a light, smooth gentle touch.
9. Keep the insert tip in motion at all times, using overlapping horizontal, vertical or oblique brush-like strokes.

INFECTION CONTROL PROCEDURES

WARNINGS

All Inserts must be cleaned, disinfected and sterilized before initial use and after each patient use to avoid cross-contamination.

LIMITATIONS ON PROCESSING

- Repeated processing has minimal effect; end of life is normally determined by wear and damage due to use.
- Always use a pH neutral instrument cleaning solution / disinfecting detergent or solution. Product can be damaged by alkaline and acidic detergents.
- Failure to follow the cleaning solution recommended concentration rates and contact times may result in rapid deterioration of the material.
- Cold liquid disinfection/sterilization, chemical vapor sterilization, and dry heat sterilization methods that have not been tested or validated for efficacy are not recommended for use.

INSTRUCTIONS

- Disassemble ultrasonic insert from handpiece.
- Do not allow residue or any form of contamination to dry on the device. Remove excess soil/gross debris by rinsing under running water.
- After each use, device should be reprocessed as soon as is reasonably practical.

MANUAL CLEANING

1. Disconnect the foot pedal and the connection cable of the handpiece from the control box.
2. Disassemble the insert from the handpiece.
3. Use a clean lint-free cloth that has been dampened with cold utility (tap) water to wipe the insert to remove any visible soil.
4. Prepare a detergent bath using neutral detergent following the vendor's recommendation of ½ oz per gallon using lukewarm utility (tap) water.
5. Dip a clean lint free cloth in detergent solution. Use the dampened lint free cloth to wipe the

insert thoroughly to remove excess soil. Pay special attention to cracks, crevices, seams, and hard to reach areas.

6. Prepare a container of hypochlorous acid solution, 200 ppm.
7. Wipe the surface of the insert using a clean lint free cloth dampened with hypochlorous acid solution. Pay special attention to cracks, crevices, seams, and hard to reach areas.
8. Visually inspect the insert, if soil or contaminant is still present, please repeat steps 3 to 5.
9. Dry the insert by allowing it to air dry, or by using clean lint free cloth(s).

MAINTENANCE, INSPECTION AND TESTING

- Visually inspect to ensure all contamination is removed.
- Visually inspect all O-rings for wear. Replace worn O-rings. Lubricate O-rings with water before use.
- Check for distortion, damage and wear. Discard damaged, worn or corroded instruments.

STERILIZATION

Place bagged instruments into the steam sterilizer as following parameter (per manufacturer's instructions). Only use steam sterilization pouches that comply with the ISO 11607-1 standard.

- Handle insert with care. Improper handling of insert, may result in injury and/or cross contamination, especially the tip.
- Failure to properly follow validated reprocessing method and techniques of Acclean Ultrasonic Inserts may lead to in cross contamination.
- Acclean Ultrasonic Inserts are intended to be used with Magnetostrictive Ultrasonic Scalers.

Sterilization Mode	Sterilizing Temperature	Cycle Time
Gravity Displacement	250°F / 121°C	30 minutes

DRYING

Recommend 30 minutes drying time after sterilization cycle.

STORAGE

To maintain sterility, instruments should remain bagged until their next use.

WARRANTY

This product is designed for use in a dental office and this warranty is not applicable to other uses.

This product is warranted against manufacturing defects in materials and workmanship for 90 days from date of purchase.

Henry Schein® will repair or replace product at its sole discretion. This warranty shall be limited to replacement or repair of the product or its parts and shall not extend to any other claims, including but not limited to loss of profit, cost of removal or replacement, incidental or consequential damages or other similar claims arising from use of this product. Damage to product resulting from acts of God, faulty installation, misuse, tampering, accident, abuse, negligence, alterations, unauthorized repairs or problems unrelated to materials and workmanship are not covered by this warranty.

DISPOSAL

Dispose of inserts in accordance with state and local laws.

ACCLEAN CLASSIC ULTRASONIC INSERT	
Item	Use
570-1624	Supragingival Tooth Surfaces
570-1729	Supragingival Tooth Surfaces
570-1873	Supragingival Areas
570-1874	Supragingival Areas

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