MASTER 3

ⓔ EN Highspeed Handpiece

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INSTRUCTIONS FOR USE

Read before use. Keep in a safe place for future reference.

CAUTION: Federal (US) law restricts this device to sale by or on the order of a licensed professional

 **Caution**


- Do not run handpiece without a bur locked in chuck.
- Do not disassemble, modify or alter handpiece.
- Do not press push-button end cap during use since this could result in OVERHEATING of handpiece head. Special caution must be exercised during use to keep cheek tissue away from push-button end cap. Contact with cheek tissue may cause push-button to depress and may result in burn injury to patient.
- Never attempt to remove bur while handpiece is running.
- If any abnormalities are found, stop using immediately and contact Henry Schein® ProRepair.
- Exceeding recommended air pressure may result in premature cartridge failure.

SPECIFICATIONS

Description	Master 3	Master 3L	Master 3S	Master 3W	Master 3 Mini	Master 3W Mini
Item Code	570-0486	570-0488	570-0490	570-0487	570-0489	570-0496
Air Pressure	38-42 psi / 2.6-2.9 bar					
Speed	360,000-380,000 rpm				400,000-420,000 rpm	
Connection	4-Hole Fixed	5-Hole Fixed	Swivel	4-Hole Fixed		
Power	18 W				15 W	
Weight	54 g		60 g	54 g	53 g	
Head Height	13.5 mm			12.7 mm	11.6 mm	10.8 mm
Head Diameter	11.1 mm					

OPERATION

Attaching Handpiece

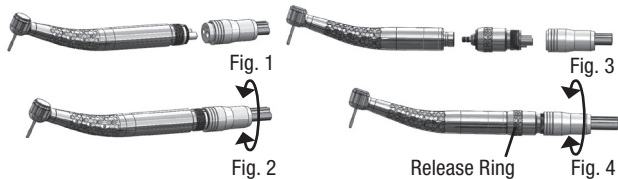Fixed Backend

1. Align tubes on handpiece with holes in delivery system tubing (Fig. 1).
2. Thread tubing nut to bottom of handpiece and finger tighten (Fig. 2).

Swivel Coupler

1. Align tubes on swivel coupler with holes in delivery system tubing (Fig. 3).
2. Thread tubing nut to bottom of handpiece and finger tighten (Fig. 4).
3. Pull back release ring on swivel coupler and slide handpiece onto coupler. Release ring (Fig. 4).

Removing Handpiece

Fixed Backend

1. Unscrew tubing nut from bottom of handpiece (Fig. 2).

Swivel Coupler (Fig. 4)

1. Pull back release ring.
2. Pull handpiece off.

Note: Do not hold or pull on hose.

Changing Burs

Caution

- Only use ISO burs with shank diameter of .0626" - .0630" (1.59-1.60 mm).
- Do not use bent, worn or damaged burs.
- Always keep burs clean. Dirt and debris on shank may cause clogging/sticking, loss of grip or premature wear of chuck.

Note: For use with standard and short shank burs.


Push-Button (Fig. 5)

Removal

1. Press push-button on end cap with thumb.
2. Pull to remove bur.

Installation


1. Press push-button and insert bur into head until it stops.
2. Release pressure on button; bur should now be secure.
3. Firmly tug on bur to ensure that it is properly secured.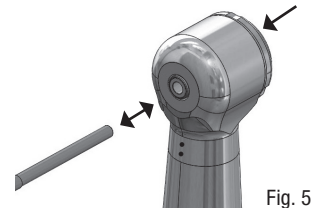

Manual Chuck (Fig. 6)

Removal

1. Mount bur tool as shown with knob pulled out.
2. Push knob into head and make a 1/4 counter-clockwise turn.
Do not loosen chuck more than necessary to remove bur.
3. Remove bur.

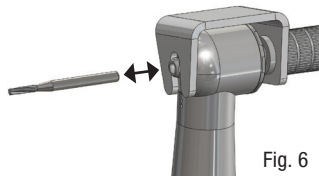Installation

1. Insert bur into chuck until it stops.
2. Push knob into head and turn clockwise.

Note: Do not over tighten. Excessive tightening may cause damage to chuck and/or bur tool.

MAINTENANCE

- Wear appropriate personal protective equipment during maintenance process. This should include eye protection and gloves.

Note: For best performance and extended handpiece life, use the Henry Schein® Maxima XTEND Handpiece Maintenance Machine to care for your handpieces. Cleaning and lubrication are required before each sterilization cycle.

Cleaning and Lubrication with Maxima XTEND Handpiece Maintenance Machine

Handpiece maintenance

1. Remove bur.
2. Detach handpiece from delivery system tubing.
3. Attach handpiece onto appropriate connector. Run maintenance cycle. See Maxima XTEND instructions for further information.

4. Remove handpiece from Maxima XTEND Maintenance Machine after cycle completes.
5. Wipe excess cleaner/lubricant from handpiece with dry, lint-free cloth.

Push-button Chuck Maintenance

Note: It is recommended that push-button chuck be cleaned every 3 - 4 maintenance cycles.

1. Remove bur.
2. Place chuck opening against tapered end of chuck cleaning nozzle.
3. Press start. See Maxima XTEND instructions for additional information.

Manual Cleaning and Lubrication with Henry Schein® Spray & Clean (102-8721) Handpiece Maintenance (Fig. 7)

1. Disconnect handpiece from air supply or coupler.
2. Insert appropriate nozzle into handpiece drive air tube and spray for approximately 3 seconds.
3. Attach the handpiece to an air supply or purge station and run for approximately 20 seconds to remove debris and excess lubricant. A spray cover or paper towel should be used to contain expelled lubrication.

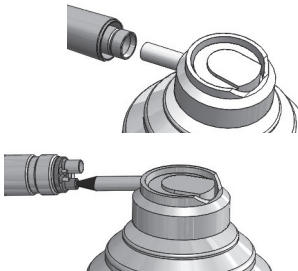
Fig. 7

Push-button Chuck Maintenance (Fig. 8)

Note: It is recommended that push-button chuck be cleaned every 3 - 4 maintenance cycles.

1. Remove bur.
2. Detach handpiece from air supply.
3. Insert spray nozzle tip of Henry Schein Spray & Clean into chuck and dispense spray for 1 - 2 seconds.
4. Work chuck mechanism to distribute cleaner/lubricant by repeatedly pressing button.
5. With push button depressed, insert and remove bur several times to break-up remaining debris.
6. Wipe off excess cleaner/lubricant with a dry, lint-free cloth.

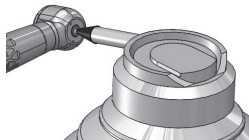
Fig. 8

Cleaning Water Spray Ports (Fig. 9)

Note: It is recommended that spray ports be periodically cleaned.

1. Carefully clean ports using cleaning wire supplied with handpiece.
2. Connect handpiece and run with water flow activated to confirm ports have been fully cleared of debris.
3. Repeat steps 1 and 2 as necessary.

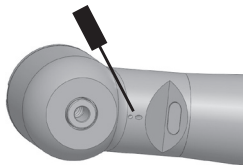
Fig. 9

Cleaning Optics - Master 3L (Fig. 10)

1. Clear optic surfaces of particulates by spraying with air syringe.
2. Wipe optic surfaces with a soft cloth or cotton swab moist with alcohol.

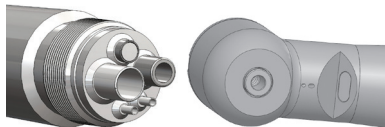
Fig. 10

STERILIZATION

⚠ WARNING

- Sterilize by autoclave after each patient use.
- Sterilization is required before first use.

⚠ Caution

- Wear appropriate personal protective equipment during sterilization and lubrication process. This should include eye protection and gloves.
- Do not use ultrasonic bath or cold sterile/disinfectant chemicals to clean handpiece.
- Do not autoclave handpieces together with other instruments.

- Do not use dry heat sterilization.
- Keep handpiece in sterilization pouch until ready to use.
- Do not cool handpiece manually as damage may occur.

1. Remove bur, then wipe external surface with dry paper towel to remove any excess lubricant and debris. Place paper towel in biohazard bin when finished.
2. Place handpiece into sterilization pouch and seal. Bag handpieces individually; no more than 1 per pouch.
3. Steam autoclave for a minimum of 15 minutes at 270° F (132° C). Let autoclave cycle completely. Do not remove before handpiece is completely cool and pouch is dry.

REPLACEMENT PARTS & ACCESSORIES

570-0574 Master 3 Mini PB Turbine

570-0577 Master 3W Turbine

570-0576 Master 3 PB Head Cap

570-0580 Master 3W Chuck

570-0581 Master 3W Bur Wrench

570-0072 Maxima XTEND Handpiece Maintenance Machine

570-0575 Master 3 PB Turbine

570-0578 Master 3W Mini Turbine

570-0579 Master 3W Head Cap

570-0573 Master 3 Swivel Coupler

102-8721 Henry Schein® Spray & Clean

SERVICE

For items that require service, including warranty service, please send to Henry Schein® ProRepair®.

Henry Schein ProRepair
620 S. Placentia Ave.
Placentia, CA 92870 USA
Phone # 714-579-0175
Fax # 714-579-0186

WARRANTY

These products are designed for use in a dental office and this warranty is not applicable to other uses. The products are warranted against manufacturing defects in materials and workmanship for 1 year from date of purchase.

Henry Schein® will repair or replace products at its sole discretion. This warranty shall be limited to replacement or repair of the products or their parts and shall not extend to any other claims, including but not limited to loss of profit, cost of removal or replacement, incidental or consequential damages or other similar claims arising from use of this product. Damage to products resulting from acts of God, faulty installation, misuse, tampering, accident, abuse, negligence, alterations, unauthorized repairs or problems unrelated to materials and workmanship are not covered by this warranty.

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Henry Schein Inc.