MASTER L35

LABORATORY HANDPIECE

by  HENRY SCHEIN®

INSTRUCTIONS FOR USE

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Read before use. Keep in a safe place for future reference.

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ELECTRICAL SAFETY

WARNING



- Never operate the unit if it has a damaged or loose cord or plug.
- Never allow the power cord to be pulled, twisted or severely bent.
- Keep unit away from water or other liquid. If the unit falls into water, unplug it immediately; do not reach into the water or liquid. Send to Henry Schein® ProRepair for evaluation and repair.
- Do not use outdoors or in an extremely humid or damp environment.
- Do not use an extension cord to operate.
- This unit has a grounded plug as a safety feature. Never bypass.
- When unplugging the unit, grasp plug with a dry hand. Never jerk cord.
- Keep unit away from excessive heat sources.
- Only use 115 V ac 60Hz input power.

GENERAL OPERATION / PRECAUTIONS

Caution


- To avoid damage to the handpiece, do not drop or subject it to excessive shock. If the handpiece has been dropped, inspect the bur/instrument for damage.
- If any abnormalities are found, stop using immediately and contact Henry Schein® ProRepair.
- Do not operate the handpiece without a bur/instrument in the chuck.
- Turn off unit before unplugging.
- Do not open the chuck while operating.
- Do not set handpiece down until it has stopped.
- Only use accessories approved for use with this product.
- Wear appropriate personal protective equipment during use.
- Never use in the presence of flammable liquids, aerosols or oxygen.
- Keep workspace clean and free of debris.

SPECIFICATIONS

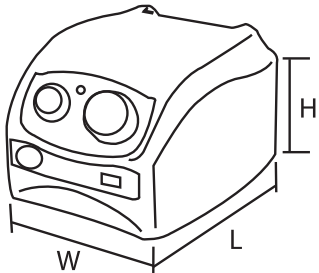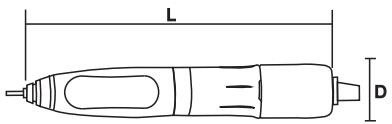	
CONTROL UNIT	HANDPIECE
Input Power: 115 V ac 60 Hz	Output: 0 – 32 V dc
Maximum Load: 2 A	Maximum Torque: 4.4 N · cm
Fuse: 2 A (T2AL250V)	Non-load RPM: 35,000 rpm
Weight: 1.8 kg	Weight: 240 g
Dimensions: 130 mm (W) x 152 mm (L) x 103 mm (H)	Dimensions: 156 mm (L) x 27 mm (D)

PARTS AND CONTROLS

- Control unit
- Handpiece
- Variable speed foot pedal
- Handpiece stand
- Power switch
- Speed control dial
- Forward/reverse switch
- Handpiece connection
- Overload warning light
- Hand/foot control switch
- Foot pedal connection
- Fuse holder
- Power cord
- Chuck wrenches

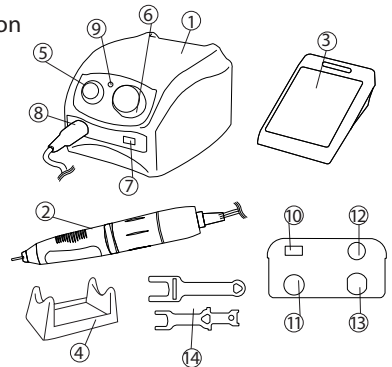SET UP

- Confirm hand/foot control switch is off and speed control dial is set to "0."
- Plug in control unit.
- Connect handpiece to front of control unit.
- Connect foot pedal to back of control unit.

OPERATION

- Never operate handpiece without bur/instrument inserted securely in chuck.
- Do not overload the handpiece while working.
- Change speed gradually while in operation.

- Choose hand or foot pedal operating mode.
- Select forward or reverse direction. Be sure to turn off control unit power before changing direction.
- While holding handpiece, turn on power switch.
- Gradually increase handpiece speed to ensure that handpiece is running normally.


NOTE: The Master L35 system is equipped with an overload protection circuit breaker to prevent overheating and/or damage to the unit. When an overload condition occurs, the warning light on the control unit will turn on, a tone will sound, and the handpiece will shut down automatically. To reset, turn off control unit power switch, turn speed control dial to 0, then turn power back on.

BUR/INSTRUMENT TYPE

⚠ Caution

- Only use ISO standard burs with shank diameter of 2.35 mm (ISO 77852-2 Type 2).
- Never exceed maximum rpm ranges as detailed in Chart A. Failure to comply may result in serious injury or damage to the unit.
- Always follow bur/instrument manufacturer user guidelines.
- Do not use bent, irregularly worn or damaged burs/instruments.
- Always keep burs/instruments clean. Dirt and debris on the shank may cause clogging of the chuck, loss of grip or premature wear of the chuck.

Reference Chart A for maximum running speed for various bur/instrument types.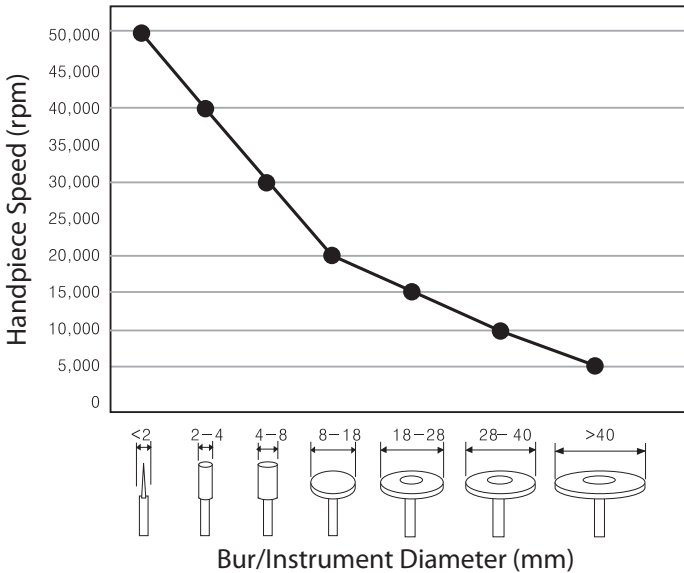

CHANGING BURS/INSTRUMENTS

⚠ Caution

- Turn power switch off before changing bur/instrument.
- Never run handpiece without a bur/instrument.
- Do not run handpiece with bur/instrument not fully inserted or securely locked in place.
- Minimum shank insertion depth is 10 mm.
- Maximum bur/instrument extension is 35 mm.

1. To open chuck, rotate upper handpiece housing into open position (Fig. 1) until it clicks.
2. Install or remove bur/instrument.
3. To close, rotate upper handpiece housing in opposite direction into the lock position until it clicks.

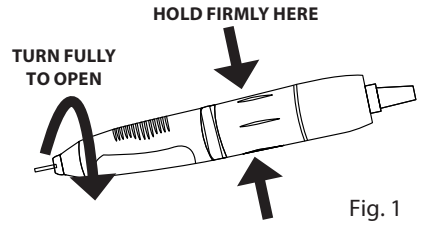
Fig. 1

MAINTENANCE

Cleaning

Caution

- Ensure that power is off.
- Do not wash with water.
- Do not lubricate.

- Regularly use a soft, dry cloth to remove dust from handpiece, control unit, and foot pedal.
- Use caution when cleaning with compressed air to avoid damaging the system.
- Isopropyl alcohol may be used to clean handpiece only.

Changing chuck (Fig. 2)

Caution

- Ensure that power is off and handpiece is disconnected from control unit.
- Always insert a bur into chuck before removal.


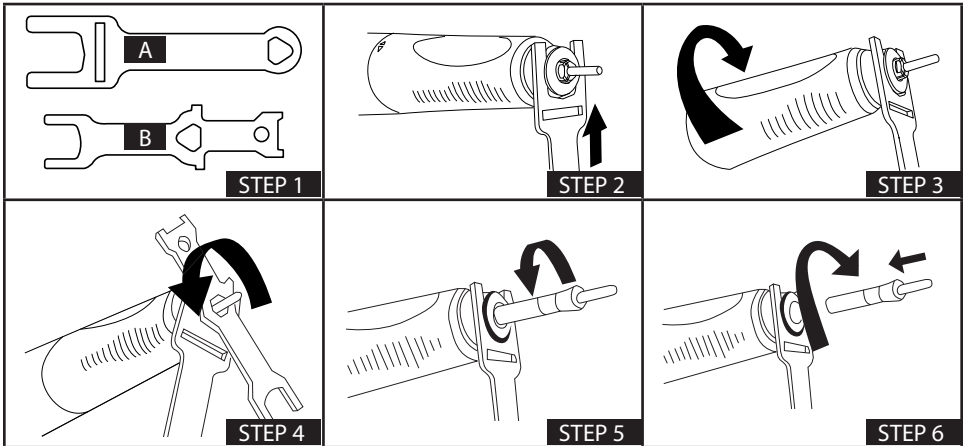
Fig. 2

1. Locate wrenches provided with system.
2. Slide wrench A into dual slots on nose piece.
3. Rotate upper handpiece housing into open position.
4. Slide triangle portion of wrench B over chuck and rotate counterclockwise until it loosens.
5. Unscrew and remove chuck by hand.
6. Insert replacement chuck and finger-tighten clockwise. Before securing with a wrench, ensure that a bur, instrument or blank bur is inserted in the chuck, and then tighten.

Changing carbon brushes (Fig. 3)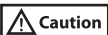

Caution

- Ensure that power is off and handpiece is disconnected from control unit.
- To avoid damage to handpiece motor, periodically check carbon brushes and change as necessary.

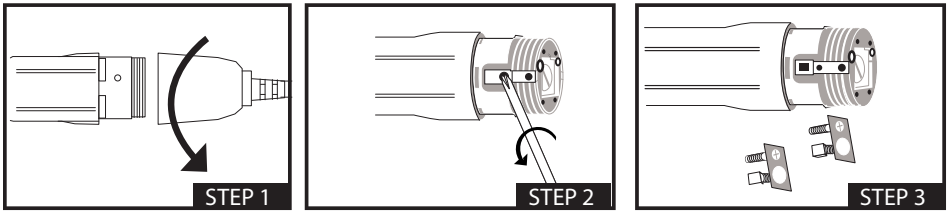
Fig. 3

1. To access brushes, unscrew motor end cap/cord housing counterclockwise.
2. Use a small Phillips head screw driver to remove brush holder assemblies (2).
3. Replace brushes and tighten into place.
4. Install and tighten motor end cap/cord housing.
5. Reconnect handpiece and run at low speed under no load to ensure that brushes seat properly.

Changing fuse

- Ensure that power is off and control unit is unplugged.

1. Locate fuse holder on back of control unit.
2. Rotate fuse holder cap counterclockwise and remove.
3. Remove fuse and inspect.
4. Replace if necessary.
5. Install and rotate fuse holder cap clockwise until secure.

REPLACEMENT PARTS

- Handpiece (112-5642)
- Chuck (112-5643)
- Foot pedal (contact Henry Schein® ProRepair)
- Carbon brush (contact Henry Schein® ProRepair)

TROUBLESHOOTING

PROBLEM	CHECK	SOLUTION
Power light not illuminating	Check power switch is in on position	Turn power switch on
	Inspect fuse	Replace fuse
Handpiece does not run and power light is illuminated	Check if overload light is illuminated	Switch power off then on
	Check handpiece connection to control unit	Reconnect handpiece to control unit
	Check foot pedal connection	Reconnect foot pedal to control unit
	Check handpiece and foot pedal cords for damage	Contact Henry Schein ProRepair
	Check carbon brushes for excessive wear	Change carbon brushes

Handpiece runs but speed control not working properly	Check if problem occurs in foot pedal mode	Replace foot pedal
	Check if problem occurs in hand control mode	Contact Henry Schein ProRepair
Handpiece gets warm during operation	None	Contact Henry Schein ProRepair
Handpiece generates excessive noise and/or vibration during use	Check for bent or damaged bur/instrument	Change bur/instrument
	Inspect chuck	Replace if necessary
	Previous solutions do not fix problem	Contact Henry Schein ProRepair

SERVICE

For items that require service, including warranty service, please send to Henry Schein ProRepair.

Henry Schein ProRepair
620 S. Placentia Ave.
Placentia, CA 92870 USA
Phone: +1-714-579-0175
Fax : +1-714-579-0186

WARRANTY

This product is designed for use in a dental laboratory/dental office and this warranty is not applicable to other uses. The unit is warranted for twelve (12) months from date of purchase against manufacturing defects in materials and workmanship; Henry Schein® will repair or replace the unit at its sole discretion. This warranty shall be limited to replacement or repair of the unit or its parts and shall not extend to any other claims, including but not limited to loss of profit, cost of removal or replacement, incidental or consequential damages or other similar claims arising from the use of this product. Damages to the product resulting from acts of God, faulty installation, misuse, tampering, accident, abuse, negligence or unauthorized repairs or alternations unrelated to problems with materials and workmanship are not covered by this warranty.

Distributed by:
Distribuido por: Distribué par:
HENRY SCHEIN, INC.
Melville, NY 11747 USA

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