# MASTER TORQUE 3

① EN Lowspeed System

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## INSTRUCTIONS FOR USE

Read before use. Keep in a safe place for future reference.

**CAUTION:** Federal (US) law restricts this device to sale by or on the order of a licensed professional

**Caution**

- Do not exceed the recommended air pressure. Excessive air pressure may cause premature failure. Before each use, check for vibration, noise, and overheating. If any abnormalities are observed, stop using immediately and contact Henry Schein ProRepair.
- Do not drop, hit or subject to excessive shock.
- Sterilization is required before first use.
- Do not wipe or immerse in acidic solutions. The user is responsible for correct operation, maintenance, and inspection.
- Do not disassemble or alter the motor, attachments or heads.
- Never operate handpiece without a bur.

**SPECIFICATIONS****Attachments**


<b>ATTACHMENT</b>	<b>GEAR RATIO</b>	<b>SPEED</b>	<b>BURS</b>
Straight Attachment 570-0497	1:1	20,000 rpm max.	Nosecone (HP) Type 2
Contra Angle Attachment 570-0498	1:1	20,000 rpm max.	N/A

4:1 Prophy Attachment 570-0502	4:1	5,000 rpm max.	Disposable prophy angles
Slide Latch Head 570-0499	1:1	20,000 rpm max. input	Latch-type (RA) Type 1
Push-Button Latch Head 570-0500	1:1	20,000 rpm max. input	Latch-type (RA) Type 1
Push-Button Friction Grip Head 570-0501	1:1	20,000 rpm max. input	Friction grip (FG) Type 3

### Motors

MOTOR	SPEED RANGE	DRIVE AIR PRESSURE
20,000 rpm motor 570-0494	0 – 20,000 rpm	35 psi (2.4 bar)
5,000 rpm motor 570-0495	0 – 5,000 rpm	35 psi (2.4 bar)

## MOTOR CONNECTION

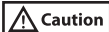- Air supply must be dry, clean, and uncontaminated. Moist, contaminated air leads to wear and premature failure.
- Damaged or missing connection gasket can cause water and air leaks resulting in poor performance.

### Connection to Air Supply

- Improper connection to tubing may result in motor disconnecting while in use. Before each use, ensure motor is securely fastened to tubing.

1. Insert 4-hole backend into tubing.
2. Hand-tighten union nut clockwise.

## Connecting / Disconnecting Attachments (Fig. 1)


**Caution**

- Make sure the attachment sits flush on motor. If not, repeat connecting process.
- Do not connect or disconnect attachments until drive motor has stopped completely.

### Connecting

1. Slide the attachment over the nose portion of the motor until it clicks.
2. Attachment is secure.

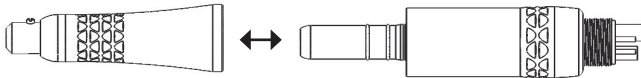
*Fig. 1*

### Disconnecting

1. Pull attachment away from motor to separate.

## Attaching / Removing Contra Angle Heads (Fig. 2)

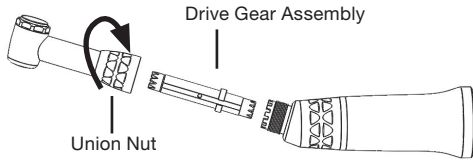**Fig. 2**

### Attaching

1. Insert drive gear assembly into sheath. Be sure it seats properly.
2. Slide head union nut up to expose castellated fitting.
3. Lower head onto drive assembly and align head and sheath fittings. Make sure fittings interlock completely.
4. Securely tighten union nut and check for any looseness.


### Removing

1. Loosen union nut and remove head from sheath.
2. Remove drive gear assembly.

## Motor Operation


### Forward Rotation

Turn the forward/reverse change ring to the F position.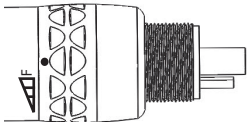


### Reverse Rotation

Turn the forward/reverse change ring to the R position.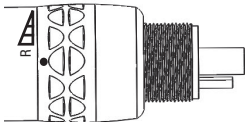

### Stop Position

When the forward/reverse change ring is between the F and R positions, the drive air is interrupted and the motor will not rotate.

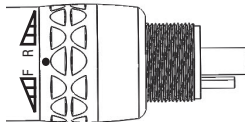
## Inserting and Removing Bur Straight Attachment (Fig. 3)

- Make sure the black dots are aligned after inserting the bur. If the black dots are not aligned, bur may dislodge or overheat and result in burns.
- After bur is seated, pull on bur to make sure it is locked into place.
- Always insert bur completely into chuck to prevent bur walk out.
- Never operate straight attachment without a bur.
- Do not use bent, worn or damaged burs as they could break and cause injury.
- Always keep bur shanks clean. Debris in chuck can cause poor bur concentricity or low retention force.

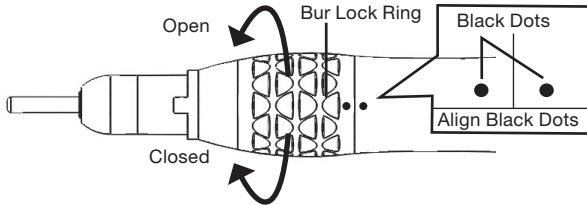*Fig. 3*

### **Inserting Bur**

1. Twist bottom section in the OPEN direction until it clicks.
2. Insert bur.
3. Turn bottom section until it clicks and black dots align.

### **Removing Bur**

1. Twist the bottom section in the OPEN direction until it clicks.
2. Remove bur.

## Slide and Push-Button Latch Heads (Fig. 4)

### Inserting bur

1. Press push button or slide latch upwards to open chuck.
2. Insert notched end of latch bur.
3. Slightly rotate bur until it engages with latch mechanism.
4. Release push button or latch to lock in bur.

### Removing bur

1. Press push button or push latch upwards to open chuck.
2. Remove bur from head.
3. Release latch or push button.

## Push-Button Friction Grip Head

### Inserting bur

1. Press push-button and insert bur into head until it stops.
2. Release pressure on button; bur should now be secure.
3. Firmly tug on bur to ensure that it is properly secured.

### Removing bur

1. Press push-button on end cap with thumb.
2. Pull to remove bur.


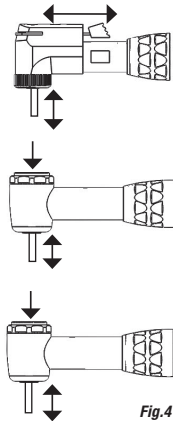
Fig.4

## Inserting / Removing Disposable Prophy Angles (Fig. 5)

**Caution**

- Do not re-use disposable prophy angles as they could break and cause injury.

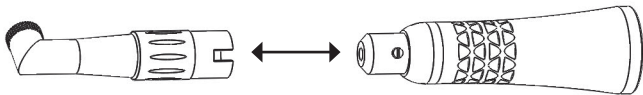*Fig. 5*

### **Insertion**

1. Align notch on disposable prophy angle with alignment screw.
2. Insert angle until it stops.

### **Removal**

1. Pull disposable prophy angle off of handpiece.

## **MAINTENANCE**

### **Contra Angle, Straight Attachments and Motors**

**NOTE:** These items **DO NOT** require lubrication.

## STERILIZATION

- Sterilization is required before first use.
- Sterilize by steam autoclave after each patient use.
- Do not use ultrasonic bath, boiling water or chemicals to clean.
- Do not autoclave at temperatures that exceed 275° F (135° C).
- Do not autoclave together with other instruments.
- Do not use dry heat sterilization.
- Wear appropriate personal protective equipment during sterilization and maintenance process. This should include eye protection and gloves.
- Keep in sterilization pouch until ready to use.

1. Wipe handpiece with damp cloth.
2. Place handpiece in sterilization pouch and seal. Bag handpieces individually.
3. Steam autoclave for a minimum of 15 minutes at 270° F (132° C). Let autoclave cycle completely. Do not remove before

handpiece is completely cool and pouch is dry.

### **SERVICE**

For items that require service, including warranty service, please send to Henry Schein ProRepair.

Henry Schein ProRepair

620 S. Placentia Ave.

Placentia, CA 92870 USA

Phone: +1-714-579-0175

Fax: +1-714-579-0186

Email: [prorepair@henryschein.com](mailto:prorepair@henryschein.com)

### **WARRANTY**

These products are designed for use in a dental office and this warranty is not applicable to other uses. The products are warranted against manufacturing defects in materials and workmanship for 1 year from date of purchase.

Henry Schein® will repair or replace products at its sole discretion. This warranty shall be limited to replacement or repair of the products or their parts and shall not extend to any other claims, including but not limited to loss of profit, cost of removal or replacement, incidental or consequential damages or other similar claims arising from use of this product. Damage to products resulting from acts of God, faulty installation, misuse, tampering, accident, abuse, negligence, alterations, unauthorized repairs or problems unrelated to materials and workmanship are not covered by this warranty.

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**Henry Schein Inc.**