CAPSULE MIXER

EN Capsule Mixer

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FR Malaxeur de Capsules

INSTRUCTIONS FOR USE

Read before use. Keep in safe place for future reference.

 **Caution:** Federal (US) law restricts this device to sale by or on the order of a licensed professional.

Warning

- Do not operate in the presence of flammable anesthetic mixture with air or with nitrous oxide.

Caution

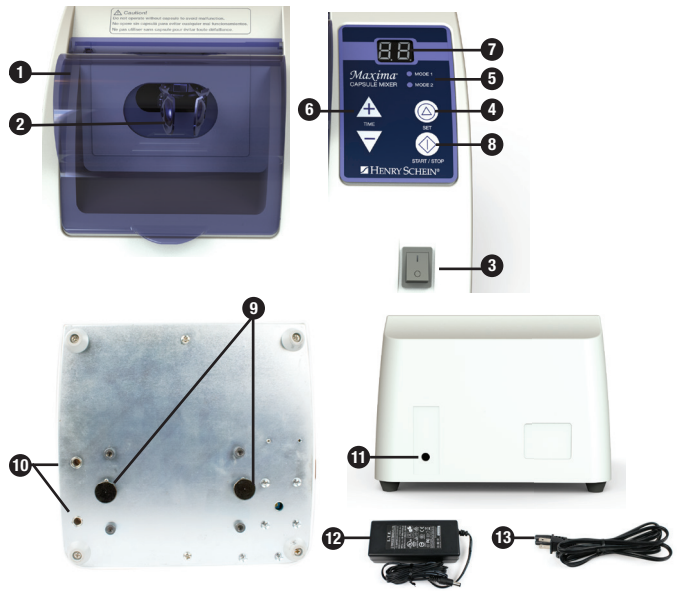
- Never operate unit if cord is damaged or loose.
- Never allow power cord to be pulled, twisted or severely bent.
- Use only with adequate ventilation
- Keep unit away from water or other liquid. If unit falls into water, unplug it immediately; do not reach into the water or liquid. Send to Henry Schein® ProRepair for evaluation and repair.
- To avoid risk of electrical shock, equipment damage or fire during an electrical storm, turn machine off and do not touch machine or cord.
- Do not use extension cord to operate.
- When unplugging unit, turn off unit first then grasp plug with a dry hand. Never jerk cord.
- Keep unit away from excessive heat sources.
- Do not use outdoors or in an extremely humid or damp environment.

INDICATION FOR USE

The Maxima® Capsule Mixer is a dental amalgamator used for the mixing of glass ionomers, amalgam, cements, and other encapsulated dental materials.

PARTS AND CONTROLS

1. Protective Shield
2. Capsule Holding Arms
3. Power Button
4. SET Button
5. Mode Indicator
6. Timer Setting Buttons
7. Display Screen
8. START/STOP Button
9. Shipping Bolts
10. Storage Holes
11. Power Cord Connection
12. AC Power Adapter
13. AC Power Cord

SETUP

Caution

- Do not operate the Maxima® Capsule Mixer with the Shipping Bolts installed.
- Maxima® Capsule Mixer should be positioned on a firm, flat, vibration-free surface.

1. To remove Shipping Bolts, tip unit forward and unscrew both bolts at the same time.
2. Insert Shipping Bolts in storage holes in bottom of unit and then place unit flat on surface.
3. Insert black-ringed DC tip connector into Power Cord Connection on back of unit.
4. Insert AC Power Cord into connection on AC Power Adapter and then plug into wall.

OPERATION

Caution

- Do not operate the Maxima® Capsule Mixer without a capsule seated in the Capsule Holding Arms.

NOTE: If necessary, mixing operation can be halted by opening the Protective Shield or pressing the START/STOP button.

If capsule leaks, stop mixing operation and wipe all surfaces clean.

1. Ensure Protective Shield is closed.
2. Press Power Button to turn unit on. Display Screen will illuminate.
3. Set mixing time according to suggestion by encapsulated material manufacturer.
 - A. With Protective Shield closed, press SET Button to select Mode 1 or Mode 2. Default setting for both modes is 10 seconds.
 - B. To change mixing time, press either time setting buttons (▲ or ▼) once so that display begins to flash.
 - C. Press (▲ or ▼) until desired mixing time appears in the display. Mixing time can be set from 1 – 30 seconds.
 - D. Press SET button to save mixing time. Display will stop flashing.
4. Lift Protective Shield.
5. If capsule has a nozzle, hold it with nozzle facing the right and turned up.

6. Seat capsule into Capsule Holding Arms.
7. Close Protective Shield.
NOTE: Maxima® Capsule Mixer will not run if Protective Shield is open. Opening the Protective Shield during operation will cause the mixer to stop running.
8. Press START/STOP. Display Screen will count down time while unit is operating.
9. When mixing time is complete, open Protective Shield and remove capsule from Capsule Holding Arms. If capsule has a nozzle, be sure to hold it with nozzle turned up.

CLEANING

- Do not use hot water, solvents, chemicals or auto-sterilizer for cleaning unit.
- Wear appropriate personal protective equipment while cleaning unit.

NOTE: It is recommended that the capsule mixer be wiped down every evening.

1. Turn off power and unplug unit from wall.
2. Wipe down all internal and external surfaces with a soft damp cloth or soft damp paper towel.
3. Discard cloth or paper towel in biohazard bin.

TROUBLESHOOTING

Problem	Cause	Solution
Power switch in on position but display is off.	Unit not powered on.	Check power cord connections at outlet and back of unit. Make sure power outlet is live.
Unit doesn't start mixing when START/STOP button pressed.	Protective Shield is not completely closed. Mixing time not set (flashing).	Close Protective Shield completely. Press SET button to save mixing time.
Capsule Mixer moves during use.	Device is on an uneven surface.	Place unit on a firm, flat, vibration-free surface.
Capsule Mixer is extremely noisy during operation.	Shipping Bolts have not been removed.	Unscrew Shipping Bolts from bottom of unit and store in Storage Holes.

SPECIFICATIONS

Input Power	AC 100 V ~ 240 V, 50-60 Hz 2 A
Output Power	DC 24 V, 2.7 A
Weight	6.8 lbs / 3.1 kg (with power adapter)
Dimensions	Unit: 8 1/4" x 7 7/8" x 5 7/8" (210 x 200 x 150 mm)
Adapter	2 1/3" x 4 3/4" x 1 9/16" (60 x 120 x 40 mm)
Operating Temperature	50° ~ 104° F (10° ~ 40° C)

DISPOSAL OF MEDICAL DEVICES

Any medical devices which could possibly be contaminated must first be decontaminated by the responsible doctor or medical institution and then be disposed of by an agent licensed and qualified to handle medical and industrial waste.

SERVICE

For items that require service, including warranty service, please send to Henry Schein ProRepair.

Henry Schein ProRepair
620 S. Placentia Ave.
Placentia, CA 92870 USA
Phone # 714-579-0175
Fax # 714-579-0186

WARRANTY

This product is designed for use in a dental office and this warranty is not applicable to other uses. This product is warranted against manufacturing defects in materials and workmanship for 1 year from date of purchase. Henry Schein® will repair or replace product at its sole discretion. This warranty shall be limited to replacement or repair of the product or its parts and shall not extend to any other claims, including but not limited to loss of profit, cost of removal or replacement, incidental or consequential damages or other similar claims arising from use of this product. Damage to product resulting from acts of God, faulty installation, misuse, tampering, accident, abuse, negligence, alterations, unauthorized repairs or problems unrelated to materials and workmanship are not covered by this warranty.

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