Curing Light Meter

(EN) Curing Light Meter

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Caution

- Never look directly into the light source without eye protection.
- Keep away from heat and direct sunlight.
- Sterilize light guides prior to using with the meter.
- Keep away from water and other liquids.
- Do not disassemble.

SPECIFICATIONS

Length: 115 mm	Light intensity range: 0~2050 mW/cm ²
Width: 62 mm	Light intensity tolerance: ±50 mW/cm ²
Height: 16 mm	Temperature range: 0~99 °C / 0~210 °F
Weight: 85 g	Temperature tolerance: ±1.5 °C / ±2.5 °F
Light guide sizes: 8 mm or 10 mm	Batteries: AAA 1.2V x 3

PARTS AND CONTROLS (Fig. 1)

1. LCM Display
2. Power Switch
3. Temperature Scale Select ($^{\circ}\text{C}$ / $^{\circ}\text{F}$)
4. Light Intensity Sensor (accepts 8 mm and 10 mm light guides)
5. AAA Batteries (3)

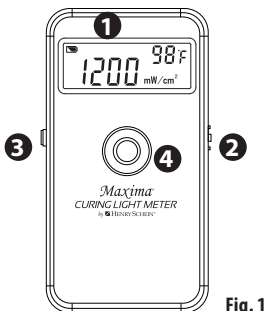
Fig. 1

SETUP (Fig. 2)

1. Remove battery cover from back of unit
2. Install batteries (3) in the correct orientation and replace battery cover.

Caution

If the temperature measurement exceeds 115°F / 46°C , stop using curing light and send to Henry Schein ProRepair

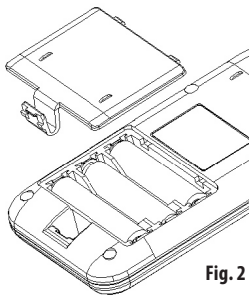
Fig. 2

OPERATION (Fig. 3)

1. Place meter on a clean, level surface.
2. Slide power switch into "ON" position.
3. Press temperature selection button to change scales ($^{\circ}\text{C}$ or $^{\circ}\text{F}$)
4. Activate curing light and position light guide tip horizontally over the sensor (do not apply too much pressure).

5. Read light intensity measurement from the display screen.

Note: The display will change in real time. If curing light is in ramp-up mode, continue to measure output until light reaches full power.

6. Read temperature display after holding light to the sensor for about 15 seconds. If temperature measurement exceeds 115°F / 46°C , stop using curing light immediately and send to Henry

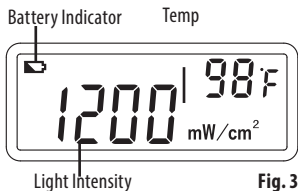
Fig. 3

Schein ProRepair for evaluation and repair.


7. After taking measurements, slide power switch into "OFF" position to preserve battery life.

CLEANING AND MAINTENANCE

1. Replace batteries when low battery indicator appears and display flashes "-Lo-". (Fig. 4)

Note: For optimum performance, replace all three batteries at the same time.

2. Clean meter and sensor surface with a soft cloth using alcohol or a mild detergent mixture.

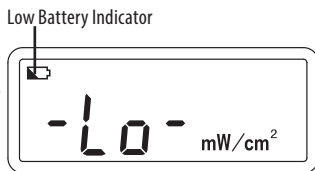TROUBLESHOOTING

Fig. 4

PROBLEM	CAUSE	SOLUTION
Display is blank after switching meter ON	Batteries are dead	Replace batteries
	Damaged power switch	Contact Henry Schein ProRepair
Flickering or discolored display	Low battery level	Replace batteries
	Damaged display	Contact Henry Schein ProRepair
Unit does not toggle between °C and °F temperature scales	Stuck temperature selection switch	Make sure switch is clean and free of debris
	Damaged temperature selection switch	Contact Henry Schein ProRepair

SERVICE

For items that require service, including warranty service, please send to Henry Schein® ProRepair.

Henry Schein® ProRepair 620 S. Placentia Ave. Placentia, CA 92870
USA Phone # 714-579-0175
Fax # 714-579-0186

WARRANTY

This product is designed for use in a dental office and this warranty is not applicable to other uses. This product is warranted against manufacturing defects in materials and workmanship for one (1) year from date of purchase.

Henry Schein® will repair or replace product at its sole discretion. This warranty shall be limited to replacement or repair of the product or its parts and shall not extend to any other claims, including but not limited to loss of profit, cost of removal or replacement, incidental or consequential damages or other similar claims arising from use of this product. Damage to product resulting from acts of God, faulty installation, misuse, tampering, accident, abuse, negligence, alterations, unauthorized repairs or problems unrelated to materials and workmanship are not covered by this warranty.

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