HENRY SCHEIN®

Maxima HP Spray

HANDPIECE SPRAY

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ANGER

EXTREMELY FLAMMABLE, CONTENTS UNDER PRESSURE

- Do not expose to heat or flame
- Use only with adequate ventilation
- Do not puncture or burn container
- Do not store at temperatures exceeding 120° F
- KEEP OUT OF REACH OF CHILDREN

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500 mL

Item Code: 112-6928

Cancer and Reproductive Harm.
www.Prop65Warnings.ca.gov

CAUTION: Federal (US) law restricts this device to sale by or on the order of a licensed professional
ADVERTENCIA: La ley federal de los Estados Unidos restringe el uso de este dispositivo a profesionales con licencia
AVERTISSEMENT: La loi fédérale américaine restreint l'usage de ce dispositif à des professionnels diplômés

INSTRUCTIONS

Use Maxima HP Spray prior to each sterilization cycle and weekly for chuck cleaning. Wear appropriate personal protective equipment during the sterilization and lubrication process. This should include eye protection and gloves.

1. Select correct spray adapter for your handpiece and connect to can nozzle.
2. Cover head of handpiece with paper towel.
3. With can upright, insert spray adapter into chuck or handpiece.
4. Spray Maxima HP Spray for 1 - 2 seconds.
5. Operate handpiece to purge debris and excess cleaner/lubricant.
6. Wipe down handpiece and sterilize according to manufacturer instructions.

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
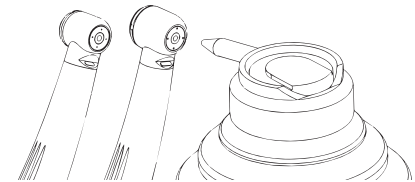
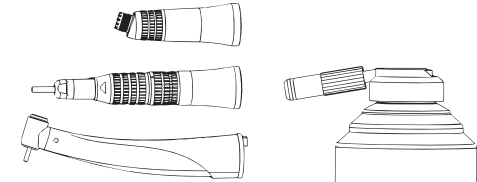
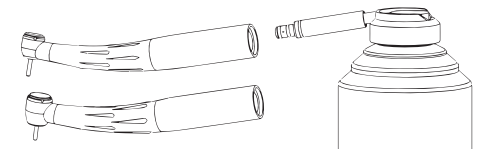
ACCESSORIES / ACCESORIOS / ACCESSOIRES:

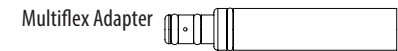112-6934 Maxima HP Spray Nozzle Adapter Kit

(manufacturer) Distributed by (in US only)

HENRY SCHEIN INC.
135 DURYEA ROAD
MELVILLE, NY 11747 USA
HENRY SCHEIN U.K. HOLDINGS LTD
MEDCARE HOUSE, CENTURION CLOSE
GILLINGHAM BUSINESS PARK
GILLINGHAM, ME8 0SB U.K.

L0070 2020/12