Read before use. Keep in safe place for future reference.

**CAUTION:** Federal (US) law restricts this device to sale by or on the order of a licensed professional

**Caution**

- For professional use only.
- Sterilization is required before first use.
- Do not disassemble or alter attachments.
- Before each use, check for vibration, noise and overheating. If any abnormalities are observed, stop using immediately and contact Henry Schein ProRepair.
- Do not drop, hit or subject to excessive shock.
- The user is responsible for correct operation, maintenance, and inspection.
- Do not wipe or immerse in acidic solutions.
- Never operate the straight attachment without a bur.
- Do not exceed the recommended air pressure. Excessive air pressure may cause premature failure.

**INTENDED USE**

The Henry Schein Maxima<sup>®</sup>-N Straight & Contra Angle Attachments are intended to be used on NSK<sup>®</sup> E-Type and any other standard E-Type motors.

**SPECIFICATIONS Attachments**

Item	Gear ratio	Speed	Burs
Straight attachment (112-6017) 	1:1	20,000 rpm max.	Ø 2.35 (ISO1797-1)
Contra angle sheath (112-6015) 	1:1	20,000 rpm max.	n/a

**CONNECTING / DISCONNECTING ATTACHMENTS (Fig. 1)**

**Connecting**

To connect the attachments to the motor, slide the attachment onto nose of E-type motor until it clicks in place.

**Disconnecting**

To remove attachments, pull to remove attachment from motor.

**Caution**

- Make sure the attachment sits flush on top of the motor. If not, repeat connecting process.
- Do not connect or disconnect attachments until the drive motor has stopped completely.

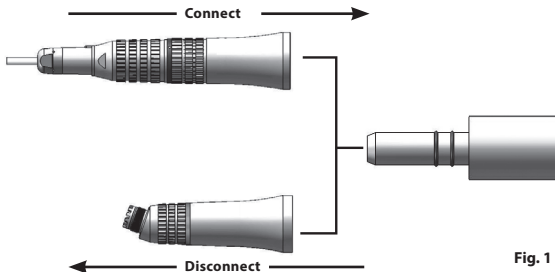
Fig. 1

**ATTACHMENT/REMOVAL OF CONTRA ANGLE HEADS (Fig. 2)**

**Attachment**

1. Insert the drive gear assembly into the sheath. Be sure it seats properly.
2. Slide the head union nut up to expose the castellated fitting.
3. Lower the head down over the drive assembly and align the head and sheath fittings. Make sure the fittings interlock completely.

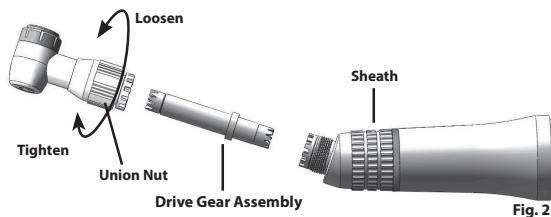
Fig. 2

**Removal**

1. Loosen the union nut and remove the head from the sheath.
2. Pull out the drive gear assembly.

**Caution**

- Loose and / or improper attachment causes excessive noise and vibration and may result in patient injury.

**INSERTING AND REMOVING THE BUR**

**Straight Attachment (Fig. 3)**

**Inserting the Bur**

1. Twist the bottom section in the OPEN direction until it clicks.
2. Insert bur.
3. Turn the bottom section back until it clicks and the black dots align.

**Removing the Bur**

1. Turn the bottom section in the OPEN direction until it clicks.
2. Remove bur.

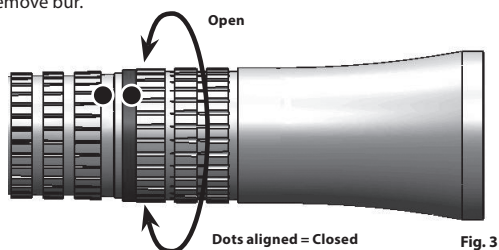
Fig. 3

**Caution**


- Make sure the black dots are aligned after inserting the bur. Improper alignment may cause heat generation and result in burns.
- Always insert the bur completely into chuck to prevent bur walk out.
- Never operate the straight attachment without a bur.

**Contra Angles (Fig. 4)**

**Swing latch and push button heads**

**Inserting the bur**

1. Press the push button or swing the latch to open the chuck.
2. Insert the notched end of latch bur into the head.
3. Slightly turn the bur until it engages with the latch mechanism.
4. Release the push button or swing the latch to the closed position.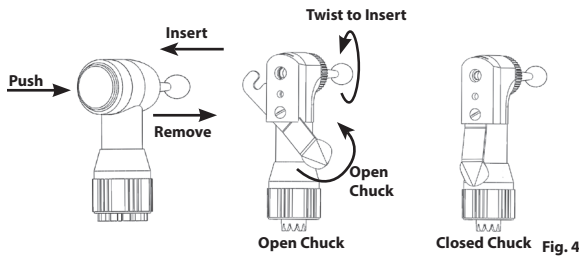

**Removing the bur**

1. Press the push button or swing the latch to open the chuck.
2. Remove the bur from the head.

**Caution**

- After the bur is seated, pull on the bur to make sure it is locked into place.
- Do not use bent, worn or damaged burs as they could break and cause personnel injury.
- Always keep bur shanks clean. Debris in the chuck can cause poor bur concentricity or low retention force.

**MAINTENANCE**

**Caution**

- Wear appropriate personal protective equipment during the maintenance process. This should include eye protection and gloves.
- Do not use ultrasonic bath, boiling water or chemicals to clean the handpiece.

**Cleaning**

1. Remove attachment from motor.
2. Remove any material and debris using a soft cloth or brush that has been moistened with alcohol or a soapy solution.
3. Wipe dry.

**Caution**

- Do not use a wire brush

**Lubrication (Fig. 5)**

1. Attach an E-Type spray adapter to the spray lubricant can nozzle.
2. Insert E-Type spray adapter into the motor end of the attachment and dispense lubricant for approximately 2-3 seconds.
3. Connect to motor and run attachment for approximately 10 seconds to expel excess lubricant.
4. Wipe off any excess lubricant with a soft cloth.

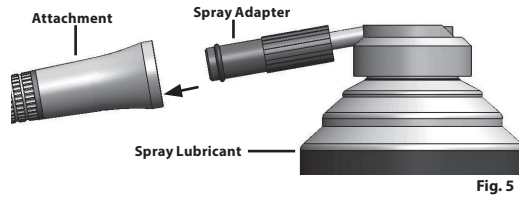
Fig. 5

**STERILIZATION**

**Caution**

- Sterilize by steam autoclave after each patient use.
- Sterilization is required before first use.
- Wear appropriate personal protective equipment during the sterilization process. This should include eye protection and gloves.
- Do not autoclave at temperatures that exceed 275° F (135° C).
- Do not autoclave together with other instruments.
- Do not use dry heat sterilization.
- Keep in sterilization pouch until ready to use.

1. Place the handpiece in a sterilization pouch and seal. Bag handpiece individually.
2. Steam autoclave for 25 minutes at 270° F (132° C).
3. Allow a minimum of 20 minutes for the drying cycle.

**SERVICE**

For items that require service, including warranty service, please send to Henry Schein ProRepair.

Henry Schein ProRepair  
620 S. Placentia Ave.  
Placentia, CA 92870 USA  
Phone # 714-579-0175  
Fax # 714-579-0186

**WARRANTY**

These products are designed for use in a dental office and this warranty is not applicable to other uses. The products are warranted against manufacturing defects in materials and workmanship for 6 months from date of purchase

Henry Schein<sup>®</sup> will repair or replace products at its sole discretion. This warranty shall be limited to replacement or repair of the products or their parts and shall not extend to any other claims, including but not limited to loss of profit, cost of removal or replacement, incidental or consequential damages or other similar claims arising from use of this product. Damage to products resulting from acts of God, faulty installation, misuse, tampering, accident, abuse, negligence, alterations, unauthorized repairs or problems unrelated to materials and workmanship are not covered by this warranty.

DISTRIBUTED BY:  
Distribuido por: Distribuído por:  
HENRY SCHEIN, INC.  
Melville, NY 11747 USA

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