Maxima™ by  HENRY SCHEIN®

---

# PRO 2

ⓔN Highspeed Handpiece

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## INSTRUCTIONS FOR USE

Read before use. Keep in a safe place for future reference.

**CAUTION:** Federal (US) law restricts this device to sale by or on the order of a licensed professional.

 **Caution**

- Do not run handpiece without bur locked in chuck.
- Do not disassemble, modify or alter handpiece.
- Do not press push-button end cap during use since this could result in OVERHEATING of handpiece head. Special caution must be exercised during use to keep cheek tissue away from push-button end cap. Contact with cheek tissue may cause push-button to depress and may result in burn injury to patient.
- Never attempt to remove bur while handpiece is running.
- If any abnormalities are found, stop using immediately and contact Henry Schein® ProRepair.

## SPECIFICATIONS

Description	Maxima PRO 2	Maxima PRO 2S	
Item Code	570-1072	570-1074	570-2095
Drive Air Pressure	38 – 42 psi (40 psi Recommended)	38 – 42 psi (40 psi Recommended)	38 – 42 psi (40 psi Recommended)
Water Pressure	12 – 29 psi (15 psi Recommended)	12 – 29 psi (15 psi Recommended)	9 – 15 psi (12 psi Recommended)
Spray Air Pressure	15 – 44 psi (22 psi Recommended)	15 – 44 psi (22 psi Recommended)	15 – 44 psi (22 psi Recommended)
Operating Speed	360,000 – 440,000 rpm	360,000 – 440,000 rpm	340,000 – 400,000 rpm
Connection Type	4-hole	MULTIflex® Non-Optic	MULTIflex® Non-Optic
Power	20 W	20 W	20 W
Weight	44 g	64 g	64 g
Head Height	12.5 mm	12.5 mm	12.5 mm
Head Diameter	10.5 mm	10.5 mm	10.5 mm

<b>Description</b>	<b>Maxima PRO 2L</b>		
<b>Item Code</b>	<b>570-1073</b>	<b>570-1451</b>	<b>570-2094</b>
Drive Air Pressure	38 – 42 psi (40 psi Recommended)	38 – 42 psi (40 psi Recommended)	38 – 42 psi (40 psi Recommended)
Water Pressure	12 – 29 psi (15 psi Recommended)	12 – 29 psi (15 psi Recommended)	9 – 15 psi (12 psi Recommended)
Spray Air Pressure	15 – 44 psi (22 psi Recommended)	15 – 44 psi (22 psi Recommended)	15 – 44 psi (22 psi Recommended)
Operating Speed	360,000 – 440,000 rpm	360,000 – 440,000 rpm	340,000 – 400,000 rpm
Connection Type	MULTIflex® Optic	Midwest® Optic	MULTIflex® Optic
Power	20 W	20 W	20 W
Weight	68 g	60 g	68 g
Head Height	12.5 mm	12.5 mm	12.5 mm
Head Diameter	10.5 mm	10.5 mm	10.5 mm

## Checking Handpiece Pressure

1. Insert the test multi-gauge manometer between the handpiece/coupler and delivery system tubing (Fig. 1).
2. Check the drive air pressure, water pressure, and the spray air pressure.

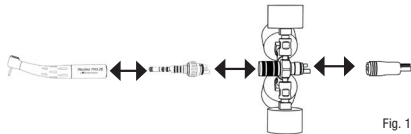
Fig. 1

- Exceeding recommended air pressure may result in premature cartridge failure.

## OPERATION

### Maxima PRO 2L / Maxima PRO 2S

#### Connecting Coupler (Fig. 2)

3. Align tubes and pins on coupler with holes in delivery system tubing.
4. Thread tubing nut to bottom of coupler.
5. Hold coupler and finger tighten nut.

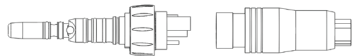
Fig. 2

### Connecting Handpiece (Fig. 3)

1. Slide handpiece onto coupler.
2. Press firmly until it snaps into place.

**Note:** Handpiece should swivel freely with slight tension and sit flush on coupler.

### Removing Handpiece (Fig. 3)

Hold ring on coupler and pull handpiece straight off.

**Note:** Do not hold or pull on hose.

Fig. 3

### Maxima PRO 2 (Fig. 4)

#### Connecting Handpiece

1. Align tubes in handpiece with delivery system tubing.
2. Thread tubing nut to bottom of handpiece and finger tighten.


#### Removing Handpiece

Unthread tubing nut and pull handpiece straight off.

Fig. 4

## Changing Burs (Fig. 5)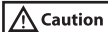

### Caution

- Only use ISO standard burs with shank diameter of .0628" - .0630" (1.595mm-1.600mm).
- Do not use bent, worn or damaged burs.
- Always keep burs clean. Dirt and debris on shank may cause clogging/sticking, loss of grip or premature wear of chuck.
- For standard head handpiece, use standard shank burs; to prevent bur breaking off in chuck assembly do not use short shank burs.

## Installation

1. Press push-button and insert bur into head until it stops.
2. Release pressure on button; bur should now be secure.
3. Firmly tug on bur to ensure that it is properly secured.

## Removal

1. Press push-button on end cap with thumb.
2. Pull to remove bur.

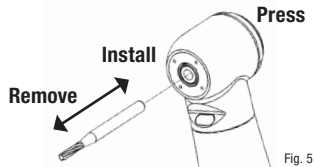
Fig. 5

## MAINTENANCE

- Wear appropriate personal protective equipment during maintenance process. This should include eye protection and gloves.

**Note:** For best performance and extended handpiece life, use the Maxima XTEND Handpiece Maintenance Machine to care for your handpieces.

### **Cleaning and Lubrication with Maxima XTEND Handpiece Maintenance Machine**

#### **Handpiece maintenance**

**Note:** Cleaning and lubrication is required before each sterilization cycle.

1. Remove bur.
2. Detach handpiece from delivery system.
3. Attach handpiece to appropriate connector in Henry Schein® Maxima XTEND® Maintenance Machine (570-0072).
4. Run maintenance cycle. See Maxima XTEND instructions for further information.
5. Remove handpiece from Maxima XTEND Maintenance Machine after cycle completes.
6. Wipe excess cleaner/lubricant from handpiece with dry, lint-free cloth.

## Chuck maintenance

**Note:** It is recommended that the push-button chuck be cleaned every 3 - 4 maintenance cycles.

1. Remove bur.
2. Place chuck opening against tapered end of chuck cleaning nozzle.
3. Press start. Keep handpiece in place until beep sounds indicating that cleaning cycle is completed. See Maxima XTEND instructions for additional information.

## Manual Cleaning and Lubrication

### Handpiece Maintenance

**Note:** Cleaning and lubrication is required before each sterilization cycle.

1. Remove bur.
2. Detach handpiece from delivery system.
3. Swivel Models Only: Fit spray appropriate adapter (112-6934) onto universal cap nozzle of Henry Schein® Maxima HP Spray (112-6928) (Fig. 6).
4. Insert nozzle into back of handpiece and hold it in place (Fig. 7 or Fig. 8).
5. Dispense spray for 1-2 seconds, then run handpiece to purge cleaner/lubricant.
6. Repeat steps 4 and 5 until cleaner/lubricant expelled from head is clear.
7. Wipe excess cleaner/lubricant from handpiece with dry, lint-free cloth.

Fig. 6

Fig. 7

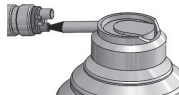
Fig. 8

## Chuck Maintenance

**Note:** It is recommended that the push-button chuck be cleaned every 3 - 4 maintenance cycles.

1. Remove bur.
2. Detach handpiece from delivery system.
3. Fit chuck adapter onto universal cap nozzle of Henry Schein® Maxima HP Spray.
4. Insert spray adapter into chuck and dispense spray for 1 - 2 seconds (Fig. 9).
5. Work chuck mechanism to distribute cleaner/lubricant by repeatedly pressing button.
6. With push button depressed, insert and remove bur several times to break-up remaining debris.
7. Wipe off excess cleaner/lubricant with a dry, lint-free cloth.

## Cleaning Water Spray Ports

**Note:** It is recommended that spray ports be periodically cleaned.

1. Carefully clean ports using cleaning wire supplied with handpiece (Fig. 10).
2. Connect handpiece and run with water flow activated to confirm ports have been fully cleared of debris.
3. Repeat steps 1 and 2 as necessary.

## Cleaning Optic Rod

1. Clear optic surfaces of particulates by spraying with air syringe.
2. Wipe optic surfaces with a soft cloth or cotton swab moist with alcohol.

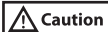
Fig. 9

Fig. 10

## Changing Coupler O-rings

**Caution**

- Improper care of o-rings may cause handpiece to malfunction.
- When changing/replacing coupler o-rings, always replace complete set of 5 pieces.

**Note:** Coupler o-rings may be lubricated with Henry Schein Maxima HP Spray.

1. Use an explorer to gently remove old/worn o-rings.
2. Slide on new o-rings using only fingers to not cause damage.

## STERILIZATION

**WARNING**

- Sterilize with autoclave after each patient use.
- Sterilization is required before first use.

 **Caution**

- Wear appropriate personal protective equipment during sterilization and lubrication process. This should include eye protection and gloves.
  - Do not use ultrasonic bath or cold sterile/disinfectant chemicals to clean handpiece.
  - Do not autoclave handpieces together with other instruments.
  - Do not use dry heat sterilization.
  - Keep handpiece in sterilization pouch until ready to use.
  - Do not cool handpiece manually as damage may occur.
  - Do not autoclave at temperatures that exceed 275° F (135° C).
1. Remove bur, then wipe external surface with dry paper towel to remove any excess lubricant and debris. Place paper towel in bio-hazard bin when finished.
  2. Place handpiece in sterilization pouch and seal. Bag handpieces individually; no more than 1 per pouch.
  3. Sterilize in a gravity type autoclave for a minimum of 15 minutes or in a pre-vacuum autoclave for a minimum of 4 minutes at 270° F (132° C). Let autoclave cycle completely. Do not remove before handpiece is completely cool and pouch is dry.

## **REPLACEMENT PARTS & ACCESSORIES**

570-1080	Maxima 4-Hole Non-optic Multiflex Coupler Chrome	112-5022	Maxima 5-Hole Optic Coupler Multiflex Chrome
112-5021	Maxima 6-Pin Halogen Multiflex Coupler Chrome	112-5482	Maxima 6-Pin LED Coupler Multiflex Chrome
112-6928	Maxima HP Spray	112-6934	Maxima HP Spray Nozzle Adapter Kit
570-0072	Maxima XTEND Handpiece Maintenance Machine	570-0140	Maxima Coupler O-ring Kit
570-0203	Midwest Coupler O-ring Kit	570-0078	Maxima Xtend Multiflex Service Adapter
570-0079	Maxima Xtend Midwest Highspeed Service Adapter		

## **SERVICE**

For items that require service, including warranty service, please send to Henry Schein® ProRepair.

Henry Schein ProRepair

620 S. Placentia Ave.

Placentia, CA 92870

714-579-0175

## **WARRANTY**

These products are designed for use in a dental office and this warranty is not applicable to other uses. The products are warranted against manufacturing defects in materials and workmanship for 1 year from date of purchase.

Henry Schein® will repair or replace products at its sole discretion. This warranty shall be limited to replacement or repair of the products or their parts and shall not extend to any other claims, including but not limited to loss of profit, cost of removal or replacement, incidental or consequential damages or other similar claims arising from use of this product. Damage to products resulting from acts of God, faulty installation, misuse, tampering, accident, abuse, negligence, alterations, unauthorized repairs or problems unrelated to materials and workmanship are not covered by this warranty.

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