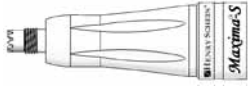
INSTRUCTIONS FOR USE

Maxima® - S

MOTOR TO ANGLE ADAPTOR

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Indication for use

The Henry Schein® *Maxima® - S* motor to angle adaptor is intended to be used with Star® brand low-speed motors for a variety of dental procedures.

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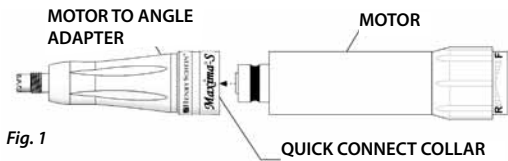
You will receive maximum use and value from your *Maxima® - S* motor to angle adaptor if you first familiarize yourself with these operating instructions.

Contents

- Installation
- Cleaning & Lubrication
- Sterilization
- Repair
- Warranty

Installation (Fig.1)

1. Assemble motor to angle adaptor to torque multiplier or motor by retracting the quick connect collar while placing motor to angle adaptor on motor; release the quick connect collar.
2. To remove motor to angle adaptor, reverse the above procedure. *Never use force to connect or disconnect the motor to angle adaptor.*
3. Thread contra angle head onto motor to angle adaptor by hand. Seat completely.

Caution If head is not seated completely on adaptor, gears are not meshing. Remove angle; spin motor to angle adaptor drive gear (Fig 2). Repeat Step 3.

MOTOR TO ANGLE ADAPTER DRIVE GEAR

Cleaning & Lubrication

Cleaning

1. Remove motor to angle adapter from motor.
2. Remove any adherent material using only a soft cloth, soft brush, gauze or cotton swab that has been moistened in a soapy solution.
3. Wipe and dry thoroughly.

Caution

- Do not use a wire brush.
- Do not use ultrasonic bath, nor immerse the adaptor in water or any solutions.

Lubrication

1. Lubricate by placing 3 drops of Henry Schein® Lubricant Oil (100-2037) into backend of motor to angle adaptor.
2. Reattach head if removed.
3. Run motor to angle adaptor until it reaches normal speed to ensure all internal parts are completely lubricated. Wipe off any excess lubricant on outside of attachment with dry gauze pad or tissue.

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Adaptor must be cleaned and sterilized between each patient use.

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1. Disconnect adaptor from motor.
2. Place adaptor in sterilization bag; use paper or paper/plastic bags only. Follow the sterilization guidelines as stated by the manufacturer of your sterilizer.
3. Steam sterilize at 135° C for a minimum of 10 minutes.
4. Remove adaptor from sterilizer after drying cycle and allow cooling in bag.

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Maxima® - S

STRAIGHT ATTACHMENT

Read before use. Keep in safe place for future reference.

WARNING Federal (US) law restricts the use of this device to licensed professionals.

Indication for use

This Henry Schein® Maxima-S Straight Attachment is intended to be used with Star® brand low-speed motors for a variety of dental procedures.

Operation and Maintenance Instructions

You will receive maximum use and value from your Maxima-S straight attachment if you first familiarize yourself with these operating instructions.

Contents


- Installation
- Operation
- Cleaning & Lubrication
- Sterilization
- Warranty

Installation (Fig. 1)


WARNING

To prevent injury, always remove bur or cutting tool from the straight attachment before installing or disconnecting from the motor.

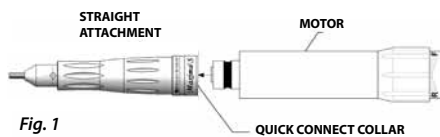1. Attach straight attachment to motor by retracting quick connect collar while placing straight attachment on motor then release quick connect collar.
2. To remove straight attachment, reverse above procedure. *Never use force to connect or disconnect straight attachment.*



Operation

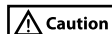
Always use ISO burs to prevent locking and slipping of burs in chuck. If a problem persists, contact Henry Schein® ProRepair.

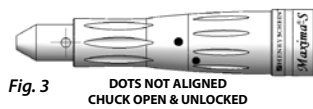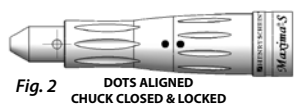
Engraved on the sleeve and chuck locking collar of the straight attachment are two dots. Aligned dots indicate chuck is closed and locked (Fig.2). When dots are not aligned, chuck is open and unlocked (Fig.3).

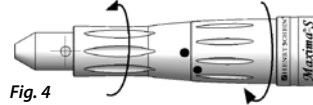To open chuck, grasp chuck locking collar with one hand, sleeve with the other, and twist (Fig.4) until it clicks in place.

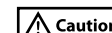With chuck open insert a .0925" (2.350 mm) shank bur. Close chuck and lock by reversing the opening procedure.

Operate the straight attachment with a bur, U-type or disposable angle secured in chuck. Operating with the chuck open or without a bur or angle installed will damage the chuck.

Cleaning & Lubrication

Cleaning

1. Remove straight attachment from motor.
2. Close chuck (dots aligned).
3. Remove any adherent material using only a soft cloth, soft brush, gauze or cotton swab that has been moistened in a soapy solution.
4. Wipe and dry thoroughly.

- Do not use a wire brush.
- Do not use ultrasonic bath, nor immerse the handpiece in water or any solutions.

Lubrication

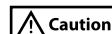
1. Lubricate by placing 3 drops of Henry Schein® Lubricant Oil (100-2037) into back end of attachment and 1 drop into chuck.
2. Insert Bur.
3. Run straight attachment until it reaches normal speed to ensure all internal parts are completely lubricated. Wipe off any excess lubricant on outside of attachment with dry gauze pad or tissue.

Sterilization

WARNING

Sterilization is required before first use.

Attachment must be cleaned and sterilized between each patient use.

- Wear appropriate personal protective equipment during the sterilization and lubrication process. This should include eye protection and gloves.
- Do not autoclave attachments together with other instruments.
- Do not use dry heat sterilization.
- Keep attachment in sterilization pouch until ready to use.

1. Disconnect attachment from motor.
2. Remove bur or angle attachment.
3. Place attachment in sterilization bag; use paper or paper/plastic bags only. Follow sterilization guidelines as stated by the manufacturer of your sterilizer.
4. Steam sterilize at 135° C for a minimum of 10 minutes.
5. Remove attachment from sterilizer after drying cycle and allow cooling in bag.

Repair

For items that require service, including warranty service, please send to Henry Schein® ProRepair.

Henry Schein® ProRepair

620 S. Placencia Ave.
Placencia, CA 92870 USA

Phone # 714-579-0175

Fax # 714-579-0186

Limited Warranty

This product is designed for use in a dental office and this warranty is not applicable to other uses. The unit is warranted against manufacturing defects in materials and workmanship for six (6) months from date of purchase; Henry Schein® will repair or replace the unit at its sole discretion. This warranty shall be limited to replacement or repair of the unit or its parts and shall not extend to any other claims, including but not limited to loss of profit, cost of removal or replacement, incidental or consequential damages or other similar claims arising from use of this product. Damage to the product resulting from acts of God, faulty installation, misuse, tampering, accident, abuse, negligence, alterations, unauthorized repairs or problems unrelated to materials and workmanship are not covered by this warranty.

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