XTEND

(EN) Handpiece Maintenance System

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INSTRUCTIONS FOR USE INSTRUCCIONES DE USO DIRECTIVES D'UTILISATION

Read before use. Keep in a safe place for future reference.

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ELECTRICAL SAFETY

- Never operate unit if cord is damaged or loose.
- Keep unit away from water or other liquid. If the unit falls into water, unplug it immediately; do not reach into the water or liquid. Send to Henry Schein® ProRepair for evaluation and repair.
- Do not use extension cord to operate.
- When unplugging unit, grasp plug with a dry hand. Never jerk cord.
- Do not use outdoors or in an extremely humid or damp environment.
- Never allow power cord to be pulled, twisted or severely bent.
- To avoid risk of electrical shock, equipment damage or fire during an electrical storm, turn machine off and do not touch machine or cord.
- This unit has a grounded plug as a safety feature. Never bypass.
- Keep unit away from excessive heat sources.
- Only use 115 V ac 60Hz input power.

GENERAL OPERATIONS/PRECAUTIONS

- For professional use only.
- Turn off unit before unplugging.
- Wear appropriate personal protective equipment during use and cleaning.
- Use only with adequate ventilation.
- Sterilize handpieces after each maintenance cycle.
- Keep workspace clean and free of debris.
- Only use accessories approved for use with this product.
- If any abnormalities are found, stop using immediately and contact Henry Schein® ProRepair.
- Do not expose to direct heat or open flame.

INDICATIONS FOR USE


The Maxima XTEND Handpiece Maintenance System is intended for cleaning and lubrication prior to sterilization of high-speed, low-speed, and electric handpieces as well as cartridge style sonic air scalers.

PARTS AND CONTROLS

<p>In the Box</p> <ol style="list-style-type: none"> 1. Main Unit 2. Air Tube 3. Power Cord 4. Oil Tray 5. 3-Pack Oil Tray Liners 6. Window Cover 7. MULTIflex Service Adapter 8. Maxima XTEND Spray 	
<p>Main Unit - External</p> <ol style="list-style-type: none"> 9. Control Panel 10. Front Door 11. Lubricant Door 12. Power Switch 13. Fuse Holder 14. Power Cord Connection 15. Air Connection 16. Air Regulator 	
<p>Main Unit - Internal</p> <ol style="list-style-type: none"> 17. Motorized E-type Connector 18. 4-hole Connectors 19. Chuck Nozzle 20. Internal Oil Suction 	
<p>Control Panel</p> <ol style="list-style-type: none"> 21. Handpiece Selector Lights 22. Handpiece Selector Buttons 23. LCD Display 24. Start Button 25. Empty Spray Can Warning Light 	

ASSEMBLY SETUP

Note: Leave adequate space around the Maxima XTEND for all connections. The Maxima XTEND should be positioned on a firm, flat, vibration-free surface.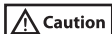

- Air supply must be clean and oil free.
- Make sure the valve for main air supply is not opened until set up is complete.
- Oil may leak if spray can is not properly seated. If can is loose, adjust can holder by unscrewing or turning to the left.
- Use only Maxima XTEND spray. Using other cleaners may damage unit.
- Make sure 4-hole connector nut is securely tightened when handpiece or service adapter is attached to avoid creating an air and/or oil leak.

1. Insert air tube completely into air connection on back of unit.
2. Connect other end of air tube to main air supply.

Note: Air Regulator dial is pre-set to 4.5 bars. Recommended air pressure is 4.5 bars. Air pressure outside 4 - 6 bars will result in an error code.

3. Connect one end of power cord to the power cord connection and plug other end into grounded electrical outlet.
4. If using service adapters, insert adapter into 4-hole connector & tighten nut (Fig. 1). One MULTIflex service adapter is included with Maxima XTEND.

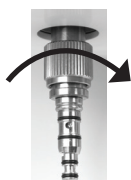
FIG. 1

FIG. 2

FIG. 3

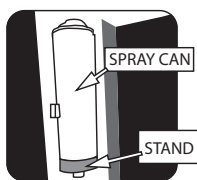
FIG. 4

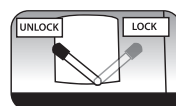
FIG. 5


5. Place 1 oil tray liner in oil tray and insert completely into lower front of unit (Fig. 2).
6. Press the "push" label on lubricant door to open (Fig. 3).

7. Move lever into unlocked position (Fig. 5), then place Maxima XTEND Spray (570-0073) on stand (Fig. 4).
8. Ensure can nozzle is properly seated, then move lever into locked position (Fig. 5).
9. Open main air valve to pressurize maintenance unit.
10. Turn power switch on.

OPERATIONAL SETUP

1. Press Start button for 2 seconds to enter set-up mode.
2. "Compressed air on" will appear on LCD display confirming compressed air is on.
3. Press Start button to show handpiece purge/dry time. The default time is set to 45 seconds. Press "1" to decrease or "3" to increase purge/dry time as needed.
NOTE: There are six time settings (30s/ 45s/ 60s/ 75s/ 90s/ 120s). Press Start button again to display the buzzer setting. If the buzzer is set to "off," press "1" to turn buzzer on.
4. Press Start button again to display the status of the motorized e-type connector motor. If the motor is set to "off," press "1" to activate the motor during maintenance cycles.
5. Press Start button again to display compressed air setting. The compressed air setting should show 4.5 bar. If LCD display does not show 4.5 bar, turn the Air Regulator counterclockwise to lower or clockwise to increase the air pressure until it is set at 4.5 bars.

OPERATION

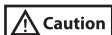- Wear appropriate personal protective equipment during use and cleaning.

Chuck maintenance

Note: Ensure bur is removed from handpiece before attempting this step. It is recommended that handpiece or attachment chuck be cleaned every 3 - 4 maintenance cycles.


1. Open front door.
2. Place chuck opening against tapered end of chuck cleaning nozzle (Fig. 6).
3. Press Start. LCD display will illuminate while in use.
4. Keep handpiece in place until beep sounds indicating that cleaning cycle is completed.

FIG. 6

Handpiece and attachment maintenance

Attaching high-speed air-driven handpieces

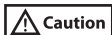- Ensure bur is removed from air-driven high-speed handpieces before using maintenance machine.
- Missing or improperly seated o-rings on service adapter may cause handpiece to become stuck on adapter or o-ring(s) may become damaged.

1. Open front door of maintenance machine.

For quick connect style handpieces only

2. Attach required service adapter by inserting back of adapter into 4-hole connector and finger tighten threaded nut.
3. Inspect service adapter to ensure that all o-rings are properly seated in their grooves.
4. Push handpiece onto applicable service adapter.

For fixed backend handpieces only

5. Insert fixed backend handpiece into 4-hole connector and finger tighten nut.

Attaching electric and air-driven straight attachments

Note: Ensure bur is inserted in straight attachment before attaching to motorized e-type connector or service adapter.

1. Open front door.
2. Attach required low-speed service adapter (if applicable) by inserting back of adapter into 4-hole connector and finger tighten threaded nut.
3. Push attachment onto applicable service adapter or motorized e-type connector until it clicks and locks in place.

Attaching electric and air-driven contra angle attachments

- Ensure bur is removed from contra angle attachments before attaching to motorized e-type connector or service adapter.

Note: It is recommended that electric contra angle attachments be maintained by connecting to motorized e-type connector to ensure

that gears and moving parts are properly cleaned and lubricated.

1. Open front door.
2. Attach required low-speed service adapter (if necessary) by inserting back of adapter into 4-hole connector and finger tighten threaded nut.
3. Push attachment onto applicable service adapter or motorized e-type connector until it clicks and locks in place.

Handpiece and attachment lubrication and internal cleaning

- Be sure to only select maintenance station(s) that have handpiece(s) installed.

Note: Do not depress buttons or open front door of Maxima XTEND while machine is running, this will cause maintenance cycle to be interrupted. If this occurs, open door completely to reset machine and then start lubrication and internal cleaning cycle again.

1. Close front door of maintenance machine.
2. Select maintenance station number(s) on the control panel by pressing the 1, 2 and/or 3 button(s) that correspond to stations with handpiece(s) attached. Indicator lights will illuminate when station is activated for use.
3. Press Start button. Machine is now operating. LCD display will illuminate while in use.
4. Machine will beep when maintenance cycle is complete.
5. Remove handpiece(s) and/or attachment(s) from maintenance machine. To disconnect attachment(s), press the lever on the motorized e-type connector or low-speed service adapter before removing.
6. Place handpiece(s) in sterilization pouch(es) and sterilize according to manufacturer recommendation.

MACHINE CLEANING AND MAINTENANCE

- Do not use hot water, solvents, chemicals or auto-sterilizer for cleaning machine or accessories.
- Wear appropriate personal protective equipment while cleaning unit.
- Only use Maxima XTEND oil tray liners (570-0081) in the oil tray. The use of any other substance may result in oil leakage or gassing of the machine.

Maintenance Unit

Note: It is recommended that the maintenance unit be wiped down every evening.

1. Turn off power to maintenance machine and unplug unit from wall.
2. Wipe down all internal and external surfaces with a soft damp cloth or soft damp paper towel.
3. Discard cloth or paper towel in biohazard bin.

Window Cover

Note: It is recommended that the window cover (570-0082) be replaced when it starts to yellow or change color.


1. Open front door and remove protective window cover.
2. Clean window cover by rinsing with lukewarm tap water or isopropyl alcohol and wipe dry with paper towel. Discard paper towel in biohazard bin.
3. Install clean protective window cover.

Oil Tray

Note: It is recommended that oil tray liners (570-0081) be replaced every 2 weeks.

1. Remove oil tray from bottom of machine.
2. Remove old oil tray liner and discard in biohazard bin.
3. Clean tray by rinsing with lukewarm tap water or isopropyl alcohol and wipe dry with paper towel. Discard paper towel in biohazard bin.
4. Place 1 new oil tray liner into tray and insert oil tray into machine.

Service Adapters

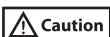- Missing or improperly seated o-rings on service adapter may cause handpiece to become stuck on adapter or o-ring(s) may become damaged.

Note: It is recommended that service adapters be cleaned weekly.

The motorized e-type connector is fixed to the unit and cannot be removed.

1. Remove service adapter from threaded connector.
2. Clean service adapter by rinsing with lukewarm tap water or isopropyl alcohol and wipe dry with paper towel. Discard paper towel in biohazard bin.
3. Inspect service adapter to ensure that all o-rings are properly seated in their grooves.

4. Attach service adapter following instructions in Step 4 in ASSEMBLY SET UP.

Maxima XTEND Spray

FIG. 7

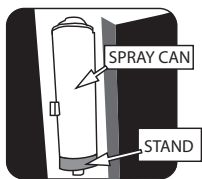
FIG. 8

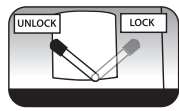
FIG. 9

1. Turn off power to maintenance machine and unplug unit from wall.
2. Press the “push” label on lubricant door to open (Fig. 7).
3. Move lever into unlocked position (Fig. 9) while holding spray can to help guide it down.
4. Remove empty spray can and dispose according to local regulations.
5. Place Maxima XTEND Spray (570-0073) on stand (Fig. 8).
6. Ensure can nozzle is properly seated, then move lever into locked position (Fig. 9).

Fuses

1. Turn off power to maintenance machine and unplug unit from wall.
2. Insert small flathead screwdriver into divot on side of fuse holder to press tab inward and lift out fuse holder (Fig 10).
3. Pull out fuse(s).
4. Install new fuse(s) (250 V / 2 A slow blow) into holder.
5. Place fuse holder back into unit.

FIG. 10

ACCESSORIES

570-0073 Maxima XTEND Spray

570-0081 10-pack Oil Tray Liners

570-0082 Window Covers

Service Adapters

570-0078 MULTIflex

570-0079 Midwest High-speed

570-0080 Midwest Low-speed

570-0083 A-dec / W&H

570-0084 Star

570-0085 NSK Swivel

570-0086 E-type

TROUBLESHOOTING

Note: The Maxima XTEND Maintenance System will beep continuously any time there is an operation error. Before attempting any troubleshooting solutions, power off machine.

PROBLEM	CAUSE	SOLUTION
Unit will not operate; lights and LCD display not on.	Unit not powered on.	Check unit power cord connections at outlet and back of unit. Make sure power outlet is live. Inspect and change fuse(s).
Unit will not operate when Start button pressed; LCD display shows “DOOR.”	Front door open.	Close front door.
Unit will not operate when Start button pressed; LCD display shows “CHECK CAN” and Empty Spray Can light is blinking.	Maxima XTEND Spray can empty. Maxima XTEND Spray can not properly installed.	Replace Maxima XTEND Spray can. Reinstall Maxima XTEND Spray can.
Unit will not operate when Start button pressed; LCD display shows “Low air pressure. Code 04.”	Compressed air is not connected. Air pressure set below 4 bars.	Properly connect compressed air. Enter OPERATIONAL SETUP for air pressure and use air regulator to adjust to 4 - 6 bars.
Unusual odor	Oil tray liners saturated.	Replace oil tray liners and clean oil tray.
Spray can will not install.	Spray nozzle is bent. Not using Maxima XTEND Spray.	Replace spray can. Use only Maxima XTEND Spray.

ERROR CODE & TYPE	DESCRIPTION	SOLUTION
Error 01 Keyboard Error	Control panel malfunction	Contact Henry Schein ProRepair
Error 02 Oil supply motor error	Oil inlet valve malfunction	Contact Henry Schein ProRepair
Error 03 Auxiliary rotation	Motorized e-type connection malfunction	Contact Henry Schein ProRepair
Error 04 Pressure input too low	Air pressure too low	Check that air supply pressure is set to at least 4 bars as explained in the OPERATIONAL SETUP section
Error 05 Pressure input too high	Air pressure too high	Check that air supply pressure is set to less than 6 bars as explained in the OPERATIONAL SETUP section
Error 06 Function memory access error	Electromagnetic switch malfunction	Contact Henry Schein ProRepair
Error 07 Oil empty	Maxima XTEND Spray can empty	Replace spray can
Error 08 Electromagnetic valve error	Oil not being distributed to handpieces/ attachments	Contact Henry Schein ProRepair
Error 09 Circuit is over temperature	Circuit board malfunction	Contact Henry Schein ProRepair
Error 10 Circuit detect error	Circuit board malfunction	Contact Henry Schein ProRepair

SPECIFICATIONS

Power:	AC 100-240 V 50/60 Hz
Fuse:	250V 2A Slow Blow Type
Air Pressure:	58 - 87 psi max (4 - 6 bar)
Recommended Pressure:	65.3 psi (4.5 bar)
Air Consumption:	40 L _n /min
Weight:	19.8 lbs (9 kg)
Dimensions:	295 (W) x 385 (H) x 295 (D)
IP Code:	IP20
Classification (EN60601):	Class B
Operating Temperature:	59 ~ 104°F (15° ~ 40°C)
Transport & Storage Temperature:	-4 ~ 158F (-20° ~ 70°C)
Operating Humidity:	25 to 90%
Transport & Storage Humidity:	5 to 95%
Operating Altitude:	Below 6500 ft (2000 m)
Transport & Storage Pressure:	700 to 1060 hPa

DISPOSAL OF MEDICAL DEVICES

Any medical devices which could possibly be contaminated must first be decontaminated by the responsible doctor or medical institution and then be disposed of by an agent licensed and qualified to handle medical and industrial waste.

INFORMATION ON ELECTROMAGNETIC COMPATIBILITY

Guidance and manufacturer's declaration-electromagnetic emissions		
The Maxima XTEND is intended for use in the electromagnetic environment specified below. The customer or the user of the Maxima XTEND should assure that it is used in such an environment.		
Emission test	Compliance	Electromagnetic environment-guidance
RF emissions CISPR 11	Group 1	The Maxima XTEND uses RF energy only for its internal function. Therefore, its RF emissions are very low and are not likely to cause any interference in nearby electronic equipment.
RF emissions CISPR 11	Class B	The Maxima XTEND is suitable for use in all establishments, including domestic establishments, and those directly connected to the public low-voltage power supply network that supplies buildings used for domestic purposes.

Harmonic emissions IEC 61000-3-2	Class A	
Voltage fluctuations /flicker emissions IEC 61000-3-3	Compliance	

Immunity test	IEC 60601 test level	Compliance level	Electromagnetic environment-guidance
Conducted RF IEC 61000-4-6	3 Vrms 150 KHz to 80 MHz	3 Vrms	Portable and mobile RF communications equipment should be used no closer to any part of the Maxima XTEND including cables, than the recommended separation distance calculated from the equation applicable to the frequency of the transmitter. Recommended separation distance: $d = 1,2 \sqrt{P}$ $d = 1,2 \sqrt{P}$ 80MHz to 800 MHz $d = 2,3 \sqrt{P}$ 800MHz to 2,5 GHz Where P is the maximum output power rating of the transmitter in watts (W) according to the transmitter manufacturer and d is the recommended separation distance in meters (m). Field strengths from fixed RF transmitters, as determined by an electromagnetic site survey, ^a should be less than the compliance level in each frequency range. ^b Interference may occur in the vicinity of equipment marked with the following symbol: 
Radiated RF IEC 61000-4-3	3 V/m 80MHz to 2,5 GHz	3 V/m	

NOTE 1: At 80 MHz and 800 MHz, the higher frequency range applies.

NOTE 2: These guidelines may not apply in all situations. Electromagnetic propagation is affected by absorption and reflection from structures, objects, and people.

^a Field strengths from fixed transmitters, such as base stations for radio (cellular/cordless) telephones and land mobile radios, amateur radio, AM and FM radio broadcast, and TV broadcast cannot be predicted theoretically with accuracy. To assess the electromagnetic environment due to fixed RF transmitters, an electromagnetic site survey should be considered. If the measured field strength in the location in which the Maxima XTEND is used exceeds the applicable RF compliance level above, the Maxima XTEND should be observed to verify normal operation. If abnormal performance is observed, additional measures may be necessary, such as re-orienting or relocating the Maxima XTEND.

^b Over the frequency range 150 kHz to 80 MHz, field strengths should be less than 3 V/m.

SERVICE

For items that require service, including warranty service, please send to Henry Schein® ProRepair.

Henry Schein ProRepair
 620 S. Placentia Ave.
 Placentia, CA 92870 USA
 Phone # 714-579-0175
 Fax # 714-579-0186

WARRANTY

This product is designed for use in a dental office and this warranty is not applicable to other uses. This product is warranted against manufacturing defects in materials and workmanship for 2 years from date of purchase.

Henry Schein® will repair or replace product at its sole discretion. This warranty shall be limited to replacement or repair of the product or its parts and shall not extend to any other claims, including but not limited to loss of profit, cost of removal or replacement, incidental or consequential damages or other similar claims arising from use of this product. Damage to product resulting from acts of God, faulty installation, misuse, tampering, accident, abuse, negligence, alterations, unauthorized repairs or problems unrelated to materials and workmanship are not covered by this warranty.

Distributed by (in US only)
HENRY SCHEIN INC.
 135 DURYEA ROAD
 MELVILLE, NY 11747 USA

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