# Essentials

## LED CURING LIGHT 2

570-1739  
Pink  
570-1740  
Black  
570-1741  
Silver

INSTRUCTIONS FOR USE  
INSTRUCCIONES DE USO  
DIRECTIVES D'UTILISATION

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
 **CAUTION:** Federal (US) law restricts this device to sale by or on the order of a licensed professional

### Caution

- For professional use only.
- Light guide can become hot during use, avoid contact with it.
- Never look directly into the LED light source without eye protection.
- Sterilize light guide and anti-glare cup prior to first use.
- To avoid damage to the LED handpiece, do not drop or subject it to excessive shock.
- Keep away from water and other liquids.
- If charging base falls into water, unplug immediately. Do not reach into water or liquid.
- If either the charger or the LED handpiece come into contact with water or other liquids, stop using immediately and send to Henry Schein® ProRepair for evaluation.
- Keep away from heat and direct sunlight.
- If any abnormalities are found, stop using immediately and contact Henry Schein® ProRepair.

## SPECIFICATIONS

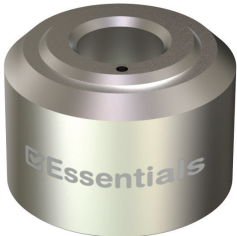	
<b>CHARGING BASE</b>	<b>LED HANDPIECE</b>
Input power: ac 100 - 240V 50/60 Hz	Light intensity: 1300 mW/cm <sup>2</sup>
Output power: 5V 2A 5W	Wavelength: 420 - 490 nm
Weight: 185 g	Weight: 120 g

## PARTS AND CONTROLS

### LED Handpiece

1. Light Guide 8 mm
2. On/Off Button
3. Mode Selection Button
4. Set Time Button
5. Display
6. Control Module
7. Battery Module

### Charger

1. Charging Base
2. Charge Indicator Light
3. DC Adapter

### Accessories

1. Glare Shield
2. Anti-glare Cups

## SETUP

1. Assemble handpiece by connecting control and battery modules. Hand tighten securely.

2. Insert the light guide into the handpiece using gentle pressure and a twisting motion until it is fully seated. There should be no gap between handpiece and light guide.

3. Place the orange anti-glare cup onto the light guide and adjust according to preference.

4. Slide glare shield over light guide until it is seated firmly in place.

5. Connect the DC adaptor to the charging base and then plug into outlet.

**Note:** When first plugged in, the charging light will not activate.

6. Place the handpiece in the charger, as shown, to begin charging. While in the base, the charge indicator light will be red and turn green when fully charged.


**Note:** Allow the handpiece to fully charge. The battery should be charged for 3 hours prior to first use.


STEP 1

STEP 2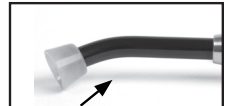

STEP 3

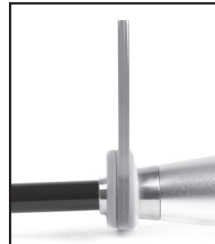STEP 4

STEP 6

## OPERATION



### Caution

- Wear protective eyewear and ensure glare shield is secured in place before use.
- Light guide can become hot during use, avoid contact with it.

### On / Off

1. Press the "On/Off" button (A) once to turn on the display panel. The digital display will show the curing time from the previous cycle.
2. Press the "On/Off" button to begin or stop the curing cycle.

**Note:** A beep will sound when the cycle starts. A final beep will sound at the end of the cycle.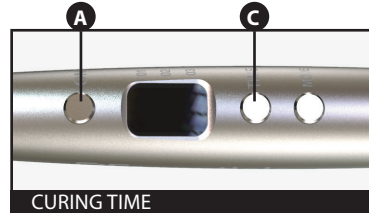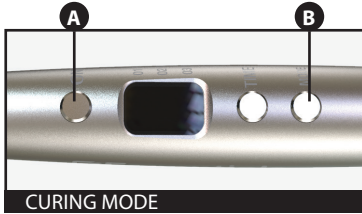

### Set curing mode

1. Press "Mode" button (B) to display active mode setting.
2. Press the "Mode" button to cycle through the 3 mode options.

**01 FULL** - Continuous full power.

**02 RAMP** - Starts at 70% power and gradually increases to full power.

**03 PULSE** - Alternates between full and 70% power in 1/2 second increments.

### Setting curing time

1. Press "Time" button (C) to display curing time.
2. To change the active curing time, push the "Time" button to increase the time in 1-second increments. The default curing time for all modes is 6 seconds..
3. Press the "On/Off" button (A) to begin curing cycle.

## Charging

### Caution


- Ensure handpiece is completely dry before placing in charger.
- Please charge and store the battery in a cool environment.
- If the battery is warm after charging, wait for the battery to cool before using to maximize battery life.
- Avoid storing the battery in a fully charged state, as it will shorten the battery lifespan. If the battery is fully charged, emit the unit for about 20 seconds before storing.
- Avoid storing the battery when nearly empty as self-consumption will result in excessive discharge and prevent further charging. If the battery is empty, charge for 10-20 minutes before storing.
- When storing the battery, do not place on the charger stand to avoid overcharge.
- Please charge/discharge the battery every 2 months when not used.
- Please discard the battery according to manufacturer's instructions and recycle the battery.
- Do not dispose in fire to prevent injury or burns.
- Avoid the battery from contacting any metal objects or it will short circuit.
- Remove and reconnect the battery module if any short circuiting occurs.
- Do not disassemble.
- Do not crush.
- Keep the battery dry.
- Keep the battery away from the heat source.

**Note:** If the battery capacity is considerably reduced and cannot be fully charged even after a long time, the battery is depleted. If the battery is depleted or damaged, please contact your dealer to buy a new battery.

**Charger ready:** No light (without handpiece in base)

**Charging in progress:** Steady red light

**Charging complete:** Steady green light (while handpiece in base)

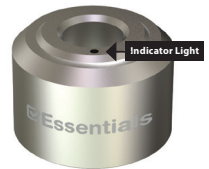## CHANGING LI-ION BATTERY

### Caution

- After battery replacement, all user settings will revert back to the factory settings.
- Prior to using a new battery, please charge for a minimum of 3 hours to ensure maximum battery life.
- There is danger of explosion if the battery is incorrectly replaced. Replace only with the same or equivalent battery type as recommended by the manufacturer.
- Medical device is used Li-ion Battery, and replace the battery only with service personnel

1. Unscrew the battery module counter clockwise from control module.
2. Turn battery module upside-down and allow battery to slide out.
3. Insert new battery. Battery orientation does not matter.
4. Recycle old battery.

## CLEANING AND STERILIZATION

- Sterilize light guide and anti-glare cup before first use.
- Sterilize light guide and anti-glare cup after each patient use.
- Keep in sterilization pouch until ready to use.
- Wear appropriate personal protective equipment during the cleaning and sterilization process. This should include eye protection and gloves.
- Do not run or submerge handpiece in water.
- Do not sterilize handpiece body.

1. Remove light guide, place in sterilization pouch, and seal. Bag light guides separately.
2. Wipe handpiece surface with a soft cloth soaked in warm water or a mild detergent mixture.
3. Dry handpiece completely before returning it to the charger.

## REPLACEMENT PARTS

570-1918 8mm Light Guide

570-1919 Li-ion Battery

## TROUBLESHOOTING

PROBLEM	CAUSE	SOLUTION
Display is blank after pressing the "On/Off" button	The battery is drained	Charge handpiece
	Defective battery	Replace battery
	Battery not properly connected	Reinstall into handpiece
	A metal object has short circuited the battery	Remove object, reinstall into handpiece
	Integrated circuit board is defective	Contact Henry Schein® ProRepair
"bL" appears on display	The battery is low	Charge handpiece
	Defective battery	Replace battery
"HH" appears on display	Unit has overheated	Allow handpiece to cool down
	Defective temperature sensor	Contact Henry Schein® ProRepair
Low light intensity	Light guide is not installed properly	Ensure light guide is seated flush with handpiece (no gap)
	Cracked, chipped or damaged light guide	Replace light guide
	Resin affixed to light guide	Clean light guide to remove any debris
	Defective LED module	Contact Henry Schein® ProRepair
Continuous beeping of the handpiece	Battery not properly connected	Reinstall into handpiece

PROBLEM	CAUSE	SOLUTION
Handpiece doesn't charge	DC adapter is not properly connected	Check outlet and charger base connections
	Poor connection between handpiece and charger	Check handpiece and base charging contacts to make sure they are clean and debris free
	Dead or defective battery	Replace battery
	Defective charging base	Contact Henry Schein® ProRepair
Red signal on charger stand when empty	There may be debris or moisture in the stand	Clean out and remove any liquid or object inside the charging stand
	Defect in charger	Contact Henry Schein® ProRepair
Charger stand status light indicator flickers	Battery not properly connected	Reinstall into handpiece
	Defective battery	Replace battery
	Defective charging base	Contact Henry Schein® ProRepair

## SERVICE

For items that require service, including warranty service, please send to Henry Schein® ProRepair.

Henry Schein® ProRepair  
620 S. Placentia Ave.  
Placentia, CA 92870 USA  
Phone # 714-579-0175  
Fax # 714-579-0186

## WARRANTY

This product is designed for use in a dental office and this warranty is not applicable to other uses. This product is warranted against manufacturing defects in materials and workmanship for one (1) year from date of purchase.

Henry Schein® will repair or replace product at its sole discretion. This warranty shall be limited to replacement or repair of the product or its parts and shall not extend to any other claims, including but not limited to loss of profit, cost of removal or replacement, incidental or consequential damages or other similar claims arising from use of this product. Damage to product resulting from acts of God, faulty installation, misuse, tampering, accident, abuse, negligence, alterations, unauthorized repairs or problems unrelated to materials and workmanship are not covered by this warranty.

# Essentials

## LED CURING LIGHT 2

INSTRUCTIONS FOR USE  
INSTRUCCIONES DE USO  
DIRECTIVES D'UTILISATION

Read before use. Keep in a safe place for future reference.

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