MASTER TORQUE *ST*

by  HENRY SCHEIN®

112-5997

Read before use. Keep in safe place for future reference.

 **CAUTION:** Federal (US) law restricts this device to sale by or on the order of a licensed professional

Caution

- For professional use only.
- Before each use, check for vibration, noise, and overheating. If any abnormalities are observed, stop using immediately and contact Henry Schein ProRepair.
- Do not disassemble or alter the handpiece.
- Never operate the handpiece without a bur, prophy angle or contra angle attachment (instruments) installed.
- The user is responsible for correct operation, maintenance, and inspection.
- Sterilization is required before first use.
- Do not drop, hit or subject to excessive shock.
- Do not wipe or immerse in acidic solutions.
- Do not exceed the recommended air pressure. Excessive air pressure may cause premature failure.

SPECIFICATIONS

Motor

| ITEM | SPEED RANGE | DRIVE AIR PRESSURE | BURS |
|---------------------------------------|-------------------|--------------------|--------------------|
| Master Torque ST Handpiece (112-5997) | 2,000 - 5,500 rpm | 15 - 35 psi | Ø 2.35 (ISO1797-1) |


CONNECTION

Caution

- Air supply must be dry, clean, and uncontaminated. Moist, contaminated air leads to wear and premature failure.
- Damaged or missing connection gasket can cause air leaks resulting in poor performance.

1. Insert 4-hole backend into tubing
2. Hand-tighten union nut clockwise.

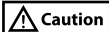**Caution**

- Improper connection to the tubing may result in the handpiece disconnecting while in use. Before each use, ensure the handpiece is securely fastened to the tubing.

OPERATION**Forward and reverse rotation (Fig. 1)****Forward**

Turn the forward/reverse control ring until the alignment screw is over the F position.

**Reverse**

Turn the forward/reverse control ring until the alignment screw is over the R position.

**Stop Position**

When the forward/reverse control ring is in the "O" position, the drive air is interrupted and the motor will not rotate.

Note: Rotation speed can be adjusted by setting the control ring between the F and O or R and O position.

Inserting / removing instruments

Inserting (Fig. 2)

1. Turn the chuck locking ring until the black dot on the ring is aligned with the red dot (open position).
2. Insert instruments

Note: If inserting prophylagy angle or contra angle attachment, align notch on angle or attachment with alignment screw on handpiece and push on until it stops

3. Turn chuck locking ring until the black dot on the ring is aligned with the black dot (closed position).

Removing

1. Turn the chuck locking ring until the black dot on the ring is aligned with the red dot (open position).
2. Remove instrument.


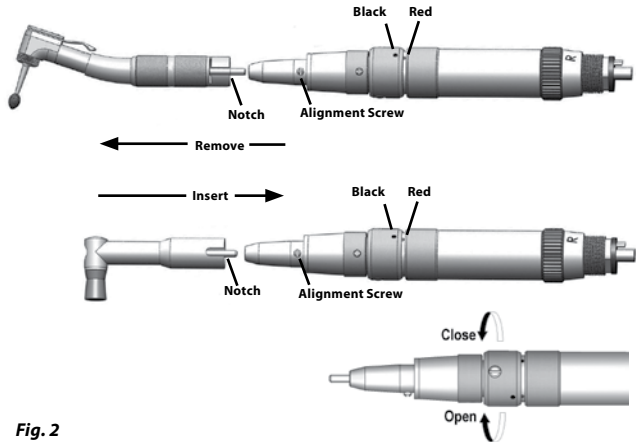
Fig. 2

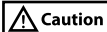- To avoid chuck damage do not close the chuck (black dots aligned) unless an instrument is installed.
- Always insert instruments completely into chuck to prevent walk out.
- When installing prophy angle and contra angle attachments make sure they sit flush on handpiece. If not, repeat inserting process.
- After instrument is installed, pull on it slightly to make sure it is locked into place.
- Make sure the black dots are aligned after inserting instrument. If the chuck is not properly locked, instrument may dislodge or overheat and result in burns.
- Do not install or remove instruments until motor has stopped completely.
- Loose instruments cause excessive noise and vibration and may result in patient injury.
- Do not re-use disposable prophy angles as they could break and cause injury.

Drive Air Pressure

- For prophylactic procedures, suggested drive air pressure is 15 psi.
- For cutting procedures, suggested drive air pressure is 35 psi.

MAINTENANCE


Caution

- Wear appropriate personal protective equipment during the maintenance process. This should include eye protection and gloves.


1. Disconnect handpiece from air supply connector.
2. Dispense handpiece cleaner/lubricant (Henry Schein Spray & Clean 102-8721) for 2 – 3 seconds into handpiece drive air line (Fig. 3).
3. Reconnect handpiece to air supply connector and operate in forward and reverse until excess cleaner/lubricant is expelled.

Caution

- Hold handpiece firmly while lubricating to prevent movement resulting from spray pressure.
- Always hold spray can upright.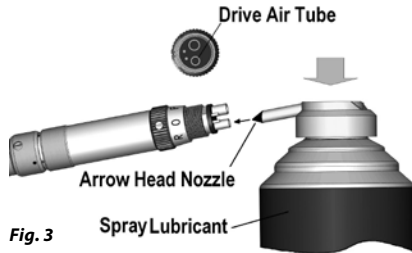

STERILIZATION

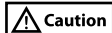- Sterilization is required before first use.
- Wear appropriate personal protective equipment during sterilization process, including eye protection and gloves.
- Do not autoclave at temperatures that exceed 275° F (135° C).
- Do not use dry heat sterilization.
- Sterilize by steam autoclave after each patient use.
- Do not use ultrasonic bath, boiling water or chemicals to clean the handpiece.
- Do not autoclave together with other instruments.
- Keep in sterilization pouch until ready to use.

1. Wipe handpiece with a damp cloth to remove any debris.
2. Clean/lubricate handpiece as described in the Maintenance section.
3. Place the handpiece in a sterilization pouch and seal. Bag handpieces individually.
4. Steam autoclave for 25 minutes at 270° F (132° C).
5. Allow a minimum of 20 minutes for the drying cycle.

TROUBLESHOOTING

If problems still occur after following these instructions, contact Henry Schein ProRepair at 1-800-367-3674.

| PROBLEM | SOLUTION |
|---|---|
| Handpiece does not run | <ol style="list-style-type: none">1. Check air pressure.2. Check forward/reverse control ring to makes sure that it is not in the stop "O" position. |
| Handpiece sluggish | <ol style="list-style-type: none">1. Check air pressure.2. Check for faulty or missing gasket.3. Perform maintenance procedure.4. Check location of the forward/reverse control ring |
| Air leak at connector | <ol style="list-style-type: none">1. Make sure tubing connection is tight.2. Check for faulty or missing gasket. |
| Handpiece runs but instrument does not rotate | <ol style="list-style-type: none">1. Make sure instrument is fully inserted in chuck.2. Make sure chuck is locked in closed position (block dots aligned). |
| Instrument vibrates excessively | Faulty instrument. Install another one. |

SERVICE

For items that require service, including warranty service, please send to Henry Schein ProRepair.

Henry Schein ProRepair
620 S. Placentia Ave.
Placentia, CA 92870 USA
Phone # 714-579-0175
Fax # 714-579-0186

WARRANTY

These products are designed for use in a dental office and this warranty is not applicable to other uses. The products are warranted against manufacturing defects in materials and workmanship for 1 year from date of purchase

Henry Schein® will repair or replace products at its sole discretion. This warranty shall be limited to replacement or repair of the products or their parts and shall not extend to any other claims, including but not limited to loss of profit, cost of removal or replacement, incidental or consequential damages or other similar claims arising from use of this product. Damage to products resulting from acts of God, faulty installation, misuse, tampering, accident, abuse, negligence, alterations, unauthorized repairs or problems unrelated to materials and workmanship are not covered by this warranty.

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