Maxima™ by  HENRY SCHEIN®

PRO 2

① EN Dual Use Highspeed Handpiece

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INSTRUCTIONS FOR USE

Read before use. Keep in a safe place for future reference.

CAUTION: Federal (US) law restricts this device to sale by or on the order of a licensed professional.

 **Caution**

- Do not run handpiece without bur locked in chuck.
- Do not disassemble, modify or alter handpiece.
- Do not press push-button end cap during use since this could result in OVERHEATING of handpiece head. Special caution must be exercised during use to keep cheek tissue away from push-button end cap. Contact with cheek tissue may cause push-button to depress and may result in burn injury to patient.
- Never attempt to remove bur while handpiece is running.
- If any abnormalities are found, stop using immediately and contact Henry Schein Handpiece & Small Equipment Solutions.

SPECIFICATIONS

Item Code	570-3815	570-3814
Operating Air Pressure	35 – 40 psi	35 – 40 psi
Operating Speed	415,000-440,000 rpm	415,000-440,000 rpm
Connection Type	Star® Non-Optic	Star® Optic
Lubrication Environment	Dual Use	Dual Use
Power	18 W	18 W
Weight	50 g	52 g
Head Height	12.5 mm	12.5 mm
Head Diameter	10.5 mm	10.5 mm

AIR PRESSURE

Set air pressure at handpiece connection to 35 - 40 psi.

Caution

- Exceeding recommended air pressure may result in premature cartridge failure.

OPERATION

Connecting Coupler (Fig. 1)

1. Align tubes and pins on coupler with holes in delivery system tubing.
2. Thread tubing nut to bottom of coupler.
3. Hold coupler and finger tighten nut.

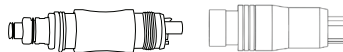
Fig. 1

Connecting Handpiece (Fig. 2)

1. Slide handpiece onto coupler.
2. Press firmly until it snaps into place.

Note: Handpiece should swivel freely with slight tension and sit flush on coupler.

Removing Handpiece (Fig. 2)

Hold ring on coupler and pull handpiece straight off.

Note: Do not hold or pull on hose.

Fig. 2

Changing Burs (Fig. 3)

- Only use ISO standard burs with shank diameter of .0626" - .0630" (1.59-1.60 mm).
- Do not use bent, worn or damaged burs.
- Always keep burs clean. Dirt and debris on shank may cause clogging/sticking, loss of grip or premature wear of chuck.
- For standard head handpiece, use standard shank burs; to prevent bur breaking off in chuck assembly do not use short shank burs.

Installation

1. Press push-button and insert bur into head until it stops.
2. Release pressure on button; bur should now be secure.
3. Firmly tug on bur to ensure that it is properly secured.

Removal

1. Press push-button on end cap with thumb.
2. Pull to remove bur.

DUAL USE MAINTENANCE

Caution

- Wear appropriate personal protective equipment during maintenance process. This should include eye protection and gloves.

Note: The turbine for this Dual Use Handpiece contains a special bearing design that requires no lubrication and free to be maintained in a lubricated or unlubricated environment. Although if it gets lubricated even once, it **MUST** then continue to be lubricated for the rest of its life.

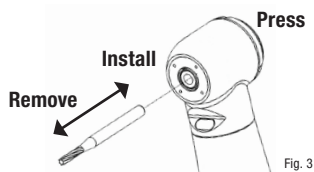
Fig. 3

Handpiece Maintenance For Lubricated Environments

Cleaning and Lubrication with Maxima XTEND Handpiece Maintenance Machine

Note: Cleaning and lubrication is required before each sterilization cycle.

1. Remove bur.
2. Detach handpiece from delivery system.
3. Attach handpiece to appropriate connector in Henry Schein® Maxima XTEND® Maintenance Machine (570-0072).
4. Run maintenance cycle. See Maxima XTEND instructions for further information.
5. Remove handpiece from Maxima XTEND Maintenance Machine after cycle completes.
6. Wipe excess cleaner/lubricant from handpiece with dry, lint-free cloth.

Manual Cleaning and Lubrication

Note: Cleaning and lubrication is required before each sterilization cycle. For best performance and extended handpiece life, use the Maxima XTEND Handpiece Maintenance Machine to care for your handpieces.

1. Remove bur.
2. Detach handpiece from delivery system.
3. Swivel Models Only: Fit spray appropriate adapter (112-6934) onto universal cap nozzle of Henry Schein® Maxima HP Spray (112-6928) (Fig. 4).
4. Insert nozzle into intake air tube in service adapter and hold it in place (Fig. 5).
5. Dispense spray for 1-2 seconds, then run handpiece to purge cleaner/lubricant.
6. Repeat steps 4 and 5 until cleaner/lubricant expelled from head is clear.
7. Wipe excess cleaner/lubricant from handpiece with dry, lint-free cloth

Fig. 4

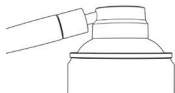
Fig. 5

Handpiece Maintenance For Unlubricated Environments

Manual Cleaning

Note: If a Dual Use Handpiece gets lubricated even once, it **MUST** then continue to be lubricated for the rest of its life.

1. Remove bur.
2. Detach handpiece from delivery system.
3. Dampen lint-free cloth with alcohol & wipe head/body removing any debris.

Chuck Maintenance

With Maxima XTEND Handpiece Maintenance Machine

Note: It is recommended that the push-button chuck be cleaned every 3 - 4 maintenance cycles.

1. Remove bur.
2. Place chuck opening against tapered end of chuck cleaning nozzle.
3. Press start. Keep handpiece in place until beep sounds indicating that cleaning cycle is completed. See Maxima XTEND instructions for additional information.

Manual Chuck Maintenance With Maxima Handpiece Spray

Note: It is recommended that the push-button chuck be cleaned every 3 - 4 maintenance cycles.

1. Remove bur.
2. Detach handpiece from delivery system.
3. Fit chuck adapter onto universal cap nozzle of Henry Schein® Maxima HP Spray.
4. Insert spray adapter into chuck and dispense spray for 1 - 2 seconds (Fig. 6).
5. Work chuck mechanism to distribute cleaner/lubricant by repeatedly pressing button.
6. With push button depressed, insert and remove bur several times to break-up remaining debris.
7. Wipe off excess cleaner/lubricant with a dry, lint-free cloth.

Fig. 6

Cleaning Water Spray Ports

Note: It is recommended that spray ports be periodically cleaned.

1. Carefully clean ports using cleaning wire supplied with handpiece (Fig. 8).
2. Connect handpiece and run with water flow activated to confirm ports have been fully cleared of debris.
3. Repeat steps 1 and 2 as necessary.

Fig. 6

Cleaning Optic Rod

1. Clear optic surfaces of particulates by spraying with air syringe.
2. Wipe optic surfaces with a soft cloth or cotton swab moist with alcohol.

Changing Coupler O-rings

- Improper care of o-rings may cause handpiece to malfunction.
- When changing/replacing coupler o-rings, always replace complete set of 2 pieces.

Note: Coupler o-rings may be lubricated with Henry Schein Maxima HP Spray.

1. Use an explorer to gently remove old/worn o-rings.

2. Slide on new o-rings using only fingers to not cause damage.

STERILIZATION

WARNING

- Sterilize with autoclave after each patient use.
- Sterilization is required before first use.

Caution

- Wear appropriate personal protective equipment during sterilization and lubrication process. This should include eye protection and gloves.
- Do not use ultrasonic bath or cold sterile/disinfectant chemicals to clean handpiece.
- Do not autoclave handpieces together with other instruments.
- Do not use dry heat sterilization.
- Keep handpiece in sterilization pouch until ready to use.
- Do not cool handpiece manually as damage may occur.
- Do not autoclave at temperatures that exceed 275° F (135° C).

1. Remove bur, then wipe external surface with dry paper towel to remove any excess lubricant and debris. Place paper towel in bio-hazard bin when finished.
2. Place handpiece in sterilization pouch and seal. Bag handpieces individually; no more than 1 per pouch.
3. Sterilize in a gravity type autoclave for a minimum of 15 minutes or in a pre-vacuum autoclave for a minimum of 4 minutes at 270° F (132° C). Let autoclave cycle completely. Do not remove before handpiece is completely cool and pouch is dry.

REPLACEMENT PARTS & ACCESSORIES

112-6928 Maxima HP Spray

112-6934 Maxima HP Spray Nozzle Adapter Kit

570-0072 Maxima XTEND Handpiece Maintenance Machine

570-0205 Star Coupler O-ring Kit

570-0084 Maxima XTend Star Service Adapter

SERVICE

For items that require service, including warranty service, please send to Handpiece & Small Equipment Solutions.

Handpiece & Small Equipment Solutions

620 S. Placentia Ave.

Placentia, CA 92870 USA

Phone # 800-235-6337

WARRANTY

These products are designed for use in a dental office and this warranty is not applicable to other uses. The products are warranted against manufacturing defects in materials and workmanship for 1 year from date of purchase.

Henry Schein® will repair or replace products at its sole discretion. This warranty shall be limited to replacement or repair of the products or their parts and shall not extend to any other claims, including but not limited to loss of profit, cost of removal or replacement, incidental or consequential damages or other similar claims arising from use of this product. Damage to products resulting from acts of God, faulty installation, misuse, tampering, accident, abuse, negligence, alterations, unauthorized repairs or problems unrelated to materials and workmanship are not covered by this warranty.

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