PRECISION-TORQUE 2

ⓔ EN Highspeed Handpiece

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INSTRUCTIONS FOR USE

Read before use. Keep in a safe place for future reference.

CAUTION: Federal (US) law restricts this device to sale by or on the order of a licensed professional

**Caution**

- Do not run handpiece without a bur locked in chuck.
- Do not disassemble, modify or alter handpiece.
- Do not press push-button end cap during use since this could result in OVERHEATING of handpiece head. Special caution must be exercised during use to keep cheek tissue away from push-button end cap. Contact with cheek tissue may cause push-button to depress and may result in burn injury to patient.
- Never attempt to remove bur while handpiece is running.
- If any abnormalities are found, stop using immediately and contact Henry Schein® ProRepair.

SPECIFICATIONS

| Description | Precision-Torque 2 | Precision-Torque 2L | Precision-Torque 2Cr | Precision-Torque 2CrL |
|------------------------|----------------------------------|----------------------------------|----------------------------------|----------------------------------|
| Item Code | 570 - 0401 | 570 - 0402 | 570 - 0741 | 570 - 0740 |
| Operating Air Pressure | 38 – 42 psi / 0.26 - 0.29 MPa | 38 – 42 psi / 0.26 - 0.29 MPa | 38 – 42 psi / 0.26 - 0.29 MPa | 38 – 42 psi / 0.26 - 0.29 MPa |

| | | | | |
|-----------------|-----------------------|-----------------------|-----------------------|-----------------------|
| Operating Speed | 405,000 – 420,000 rpm | 405,000 – 420,000 rpm | 405,000 – 420,000 rpm | 405,000 – 420,000 rpm |
| Connection Type | 4-Hole Fixed | 5-Hole Fixed | 4-Hole Fixed | 5-Hole Fixed |
| Power | 16.5 W | 16.5 W | 16.5 W | 16.5 W |
| Weight | 27 g | 29 g | 45 g | 47 g |
| Head Height | 11 mm | 11 mm | 11 mm | 11 mm |
| Head Diameter | 10.2 mm | 10.2 mm | 10.2 mm | 10.2 mm |

AIR PRESSURE

Set air pressure at handpiece connection to 38 - 42 psi.

Caution

- Exceeding recommended air pressure may result in premature cartridge failure.

OPERATION

Handpiece Connection

1. Align tubes on handpiece with holes in delivery system tubing (Fig.1).
2. Thread tubing nut to bottom of handpiece and tighten finger tight (Fig 2).

Removing handpiece

1. Unscrew tubing nut from bottom of handpiece (Fig.2).

Note: Do not hold or pull on hose.

Changing Burs

Caution

- Only use ISO burs with shank diameter of .0626" - .0630" (1.59-1.60 mm).
- Do not use bent, worn or damaged burs.
- Always keep burs clean. Dirt and debris on shank may cause clogging/sticking, loss of grip or premature wear of chuck.

Note: For use with standard and short shank burs.

Installation

1. Press push-button and insert bur into head until it stops (Fig. 3).
2. Release pressure on button; bur should now be secure.
3. Firmly tug on bur to ensure that it is properly secured.

Removal

1. Press push-button on end cap with thumb (Fig. 3).
2. Pull to remove bur.


MAINTENANCE

- Wear appropriate personal protective equipment during maintenance process. This should include eye protection and gloves.

Note: For best performance and extended handpiece life, use the Maxima XTEND Handpiece Maintenance Machine to care for your handpieces.

Cleaning and Lubrication with Maxima XTEND Handpiece Maintenance Machine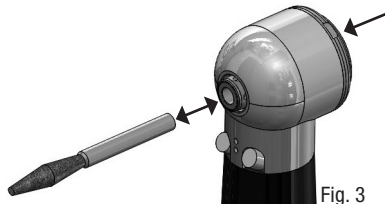

Handpiece maintenance

Note: Cleaning and lubrication is required before each sterilization cycle.

1. Remove bur.
2. Detach handpiece from delivery system tubing.
3. Open front door of Henry Schein® Maxima XTEND® Maintenance Machine (570-0072).
4. Insert fixed backend handpiece into connector and finger tighten nut. Run maintenance cycle. See Maxima XTEND instructions for further information.
5. Remove handpiece from Maxima XTEND Maintenance Machine after cycle completes.
6. Wipe excess cleaner/lubricant from handpiece with dry, lint-free cloth.

Chuck maintenance

Note: It is recommended that the push-button chuck be cleaned every 3 - 4 maintenance cycles.

1. Remove bur.
2. Place chuck opening against tapered end of chuck cleaning nozzle.
3. Press start. See Maxima XTEND instructions for additional information.

Manual Cleaning and Lubrication

Handpiece Maintenance

Note: Clean and lubricate with Henry Schein Spray & Clean (102-8721) before each sterilization cycle. Cleaning and lubrication is required before each sterilization cycle.

1. Disconnect the handpiece from air supply.
2. Insert the nozzle into the handpiece drive air tube (Fig. 4) and spray for approximately 3 seconds.
3. Attach the handpiece to an air supply or purge station and run for approximately 20 seconds to remove debris and excess lubricant. A spray cover or paper towel should be used to contain expelled lubrication.

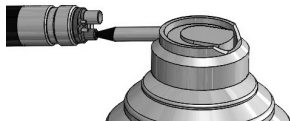
Fig. 4

Chuck Maintenance

Note: It is recommended that the push-button chuck be cleaned every 3 - 4 maintenance cycles.

1. Remove bur.
2. Detach handpiece from air supply.
3. Insert spray nozzle tip of Henry Schein Spray & Clean (102-8721) into chuck and dispense spray for 1 - 2 seconds (Fig. 5).
4. Work chuck mechanism to distribute cleaner/lubricant by repeatedly pressing button.
5. With push button depressed, insert and remove bur several times to break-up remaining debris.
6. Wipe off excess cleaner/lubricant with a dry, lint-free cloth.

Cleaning Water Spray Ports

Note: It is recommended that spray ports be periodically cleaned.

1. Carefully clean ports using cleaning wire supplied with handpiece (Fig. 6).
2. Connect handpiece and run with water flow activated to confirm ports have been fully cleared of debris.
3. Repeat steps 1 and 2 as necessary.

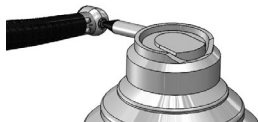
Fig. 5

Fig. 6

Cleaning Fiber Optics (Fig 7)

1. Clear optic surfaces of particulates by spraying with air syringe.
2. Wipe optic surfaces with a soft cloth or cotton swab moist with alcohol.

STERILIZATION

⚠ WARNING

- Sterilize by autoclave after each patient use.
- Sterilization is required before first use.

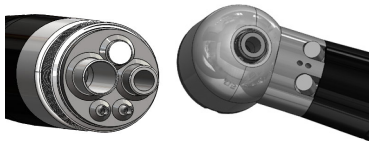
Fig. 7

⚠ Caution

- Wear appropriate personal protective equipment during sterilization and lubrication process. This should include eye protection and gloves.
- Do not use ultrasonic bath or cold sterile/disinfectant chemicals to clean handpiece.
- Do not autoclave handpieces together with other instruments.
- Do not use dry heat sterilization.
- Keep handpiece in sterilization pouch until ready to use.
- Do not cool handpiece manually as damage may occur.

1. Remove bur, then wipe external surface with dry paper towel to remove any excess lubricant and debris. Place paper towel in biohazard bin when finished.
2. Place handpiece into sterilization pouch and seal. Bag handpieces individually; no more than 1 per pouch.
3. Steam autoclave for a minimum of 15 minutes at 270° F (132° C). Let autoclave cycle completely. Do not remove before handpiece is completely cool and pouch is dry.

REPLACEMENT PARTS & ACCESSORIES

570-0403 Replacement Turbine Precision-Torque 2 /
Precision-Torque 2L

102-8721 Henry Schein Spray & Clean

570-0404 Replacement End Cap Precision-Torque 2 /
Precision-Torque 2L

570-0072 Maxima XTEND Handpiece Maintenance Machine

SERVICE

For items that require service, including warranty service, please send to Henry Schein® ProRepair®.

Henry Schein ProRepair
620 S. Placentia Ave.
Placentia, CA 92870 USA
Phone # 714-579-0175
Fax # 714-579-0186

WARRANTY

These products are designed for use in a dental office and this warranty is not applicable to other uses. The products are warranted against manufacturing defects in materials and workmanship for 1 year from date of purchase.

Henry Schein® will repair or replace products at its sole discretion. This warranty shall be limited to replacement or repair of the products or their parts and shall not extend to any other claims, including but not limited to loss of profit, cost of removal or replacement, incidental or consequential damages or other similar claims arising from use of this product. Damage to products resulting from acts of God, faulty installation, misuse, tampering, accident, abuse, negligence, alterations, unauthorized repairs or problems unrelated to materials and workmanship are not covered by this warranty.

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Henry Schein Inc.