Maxima[®] PRO 45L

by  HENRY SCHEIN[®]

45° Fiber-optic handpiece

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Henry Schein Inc.

Melville, NY 11747 USA

Resistant to disinfection with heat.

Resistant to steam sterilization up to 135°C (275° F).

 Caution

Maxima® PRO 45L handpiece is a medical device and is intended for dental treatment by a licensed dental professional. This handpiece must be used in accordance with these operating instructions and any misuse or modification to the product is prohibited and may cause harm.

It is the responsibility of the user to:

- Sterilize handpiece before beginning the treatment of each patient.
- Ensure that the device is in proper working condition.
- Ensure that the use will be for the correct purpose.
- Protect patients and third parties from all dangers.
- Avoid cross contamination by the product.

Discontinue use immediately and contact Henry Schein® ProRepair® if you experience:

- Unusual noise or vibration.
- Excessive bur wobble or insufficient bur retention.
- Overheating of handpiece head or push-button.


To avoid damage to the handpiece and harm to the patient, do not:

- Drop or subject the handpiece to excessive shock.
- Press the push-button during use.

 Important Remarks


The Maxima[®] PRO 45L handpiece is intended to be used for cavities and crown preparations, removal of fillings and prosthetic restorations, and for removal of carious tissue. It is designed for use by a trained professional in the field of general dentistry.

It is the responsibility of the user to only use a device that is in proper working condition and ensure that it is used for the correct purpose in order to protect patients and third parties from all dangers. Do not direct cooling water or air into open wounds during surgery as this may increase the risk of an air embolism or cutaneous emphysema.

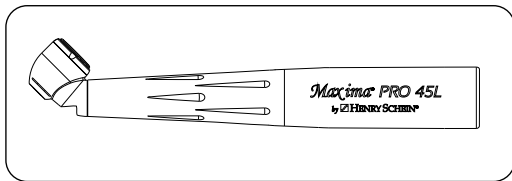Use of non-wear hard metal guide bushing.

Maxima[®] PRO 45L - ref.: 112-6772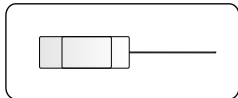

Water port cleanout tool
Ref.: 112-6845


Specification	Maxima [®] PRO 45L (112-6772)
Head Size	Standard
Connection Type	MULTIflex Coupling
Air Pressure	40 psi (2.8 bar)
Operating Speed	350,000 ~ 420,000 rpm
Power	≥ 19 W
Air Consumption	45 ~ 55 NI/min
Torque	≥ 0.0017 N.m

For Handpiece Connection

Caution

- Air supply must be dry, clean, and uncontaminated. Moist, contaminated air leads to wear and premature failure.
- Damaged or missing MULTIflex coupler o-rings can cause water and air leaks resulting in poor performance and premature failure.
- Insufficient cooling water can cause overheating and damage the tooth.
- Improper connection can cause the handpiece to be release during treatment or cause damage to the fiber-optic rod.


Note: At the beginning of each work day and before each patient, the water and air channels should be rinsed for approx. 20 to 30 seconds.

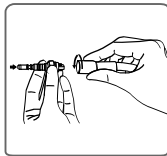Connect to Air Supply

For proper operation, set air pressure at 40 psi (2.8 bar) and check with a pressure gauge. Screw the MULTiflex coupling to a hose and tighten with a wrench.

Obs: You need a MULTiflex coupling for proper operation. If your hose is not MULTiflex, consult your dealer for the correct coupling adaptor.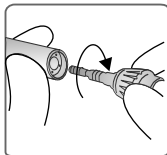

Connect to Coupler

1. Check MULTiflex coupling to make sure that the 5 o-rings are not missing or damaged.
2. Snap the handpiece onto the MULTiflex coupling and push until you hear it click into place. Then pull to make sure it is securely connected.

Disconnect from Coupler

Hold the coupling firmly and pull the handpiece while twisting.

Cooling water

Caution


For the cooling, a minimum of 50ml /min (pressure 14.5 ± 1.5 psi (1 ± 0.1 bar)) of water is needed. Insufficient water can cause overheating of the used bur and damage the tooth and/or bone.

Changing Burs

Caution

- Check the function of the chuck by pulling on the bur. Use gloves or another type of finger protection to insert, remove, and test burs in order to avoid injury and infection.
- Do not depress the push-button while handpiece is running. Actuation during operation will result in overheating and damage to the chuck.
- Avoid contact between soft tissue and the push-button. In the event of handpiece damage, heating of the head and associated burns are possible.
- Remove burs after use.

Inserting and Removing Burs

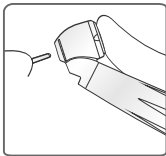Inserting Bur

Press the push-button firmly with thumb and insert the bur as far as it will go.

Removing Bur

After bur stops, press the push-button firmly and remove bur.

Bur Types

Caution

- Ensure that the burs are used for their intended purpose only and in compliance with the manufacturer's instructions.
- Do not use bent, irregularly worn or damaged burs.
- Burs that deviate from the following criteria must not be used.

Bur Types

Only use burs that correspond to DIN EN ISO 1797-1 type 3, that are made of steel or hard metal and meet the following criteria:

Shaft diameter: 1.59 to 1.60 mm (0.0626 in to 0.0629 in)

Overall length: max. 25 mm (0.9842 in)

Shaft clamping length: min. 11 mm (0.4330 in)

Cutting diameter: max. 2 mm (0.0787 in).

Changing Coupling O-rings


- Squeeze the O-ring between your fingers to form a loop.
- Push the O-ring forward and remove.
- Install new O-rings into the coupler grooves.

Caution

Improper care of O-rings may cause the handpiece to malfunction. Do not apply Vaseline[®], grease or oil.

Note

Coupler O-rings should be lubricated with handpiece lubricant applied with a cotton swab.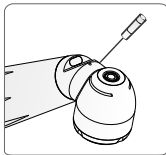

Unclogging Cooling Holes

Insufficient cooling water may be due to the clogging of a cooling port. Clean the port by using the water port cleanout tool in combination with air pressure.

Preparation of Site for Use

Caution

Non-sterile medical devices, such as handpieces, may cause infection.


- Observe suitable personal protective measures.
- Remove residual cement, composite or blood from handpiece surfaces.
- Handpiece must be dry before moving to maintenance area.
- Handpiece should be cleaned and sterilized shortly after each use.
- Remove burs from the handpiece after use.

Cleaning

Caution

- Do not clean in an ultrasonic unit.
- Clean handpiece manually.

Manual Exterior Cleaning

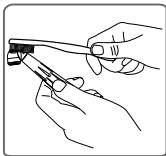- Tap water 30°C (86°F) $\pm$ 5°C (41°F).
- Brush with a medium-hard toothbrush.
- Brush under flowing tap water.
- Remove visible debris with a damp cloth or sponge.

A steam sterilization in an autoclave is required before the first use and after each use!

Interior Cleaning

The interior of the handpiece should be cleaned automatically. Do not clean interior manually.

Cleaning and maintenance of handpieces before sterilization is required to prevent premature failure.

Manual Drying

- Dry the outside and inside of the handpiece using compressed air until no water drops are visible.

 Caution

Bur/sharp tool in the handpiece.

To avoid injury remove burs before handling.

Improper maintenance of the handpiece can lead to premature wear and failure.


- Regularly service the handpiece per these instructions.

Lubrication

Lubricate the handpiece after each cleaning and before each sterilization cycle. Henry Schein® recommends using Henry Schein® Spray & Clean (102-8721), KaVo Spray or an equivalent lubricant spray.


1. Insert spray nozzle into drive air line and spray for approximately 3 seconds. A spray cover or paper towel should be used to contain expelled lubrication.
2. Attach handpiece without bur to tubing or air station, run for 20 seconds and then wipe off excess lubricant.

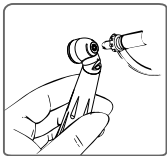Chuck Maintenance

It is recommended to clean the chuck once a week. Spray lubricant into bur opening using lubrication nozzle.

Care Products and Systems – Automatic Maintenance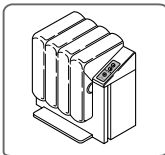

Henry Schein® recommends servicing the handpiece after each cleaning and before each sterilization cycle:

- Remove the bur.
- Operate automatic maintenance system per manufacturer's instructions.

Sterilization is required before the first use and after each use!

Sterilization Preparation

Note

- Do not use undersized sterilization pouches.
- Sterilization pouches must meet applicable industry standards.
- Properly seal the handpiece in the sterilization pouch.

Sterilize

Use a sterilizer that is compliant with DIN EN 13060 standard.

Caution

Premature failure or malfunction may be caused by not properly following these care instructions.

- Handpiece must be maintained with Henry Schein[®] Spray & Clean (102-8721), KaVo Spray or an equivalent lubricant spray.


Caution

Exposure to moisture may cause corrosion.

- Remove handpiece immediately from sterilizer after the sterilization cycle is complete.
- Do not let the handpiece cool down in the sterilizer after the drying cycle has been completed. This may increase the chance of exposure to moisture.
- Use caution when handling hot or warm handpieces in order to avoid burns.
- Do not handle wet packages as this may compromise sterility.

 Caution

- A steam sterilization in an autoclave is required before first use and after each patient.
- Wear appropriate personal protective equipment during the sterilization process. This protection equipment should protect eyes (protection glasses), hands (gloves), breathing (mask) and general clothing (lab coat).
- Do not autoclave handpieces together with other instruments.
- Do not use dry heat sterilization.
- Keep handpiece in sterilization pouch until the time of use.

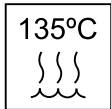Place the handpiece individually in a sterilization pouch, and seal it carefully.
Sterilize in a gravity autoclave in accordance with manufacturer's instructions.

a - Sterilize for a minimum of 10 minutes at $134^{\circ}\text{C} \pm 1^{\circ}\text{C}$ ($273^{\circ}\text{F} \pm 1.8^{\circ}\text{F}$).
b - Sterilize for a minimum of 60 minutes at $121^{\circ}\text{C} \pm 1^{\circ}\text{C}$ ($250^{\circ}\text{F} \pm 1.8^{\circ}\text{F}$).
Drying time: 30 min.

Accessories:

Item Code	Product Description
112-6773	Maxima [®] PRO 45 cartridge with cap wrench
112-5022	Maxima [®] PRO MULTIflex 5-hole coupler
112-5021	Maxima [®] PRO MULTIflex 6-pin coupler
112-5482	Maxima [®] PRO MULTIflex 6-pin LED coupler
112-6845	Water port cleanout tool
112-6846	Maxima [®] PRO 45 push-button end cap
102-8721	Henry Schein [®] Spray & Clean

Service:

For items that require service, including warranty service, please send to Henry Schein® ProRepair®.

Henry Schein ProRepair

620 S. Placentia Ave.

Placentia CA 92870 - USA

Phone: 714-579-0175 | Fax: 714-579-0186

Email: prorepair@henryschein.com

Warranty

This product is designed for use in a dental office and this warranty is not applicable to other uses. The unit is warranted against manufacturing defects in materials and workmanship for one (1) year from date of purchase; Henry Schein® will repair or replace the unit at its sole discretion. This warranty shall be limited to replacement or repair of the unit or its parts and shall not extend to any other claims, including but not limited to loss of profit, cost of removal or replacement, incidental or consequential damages or other similar claims arising from use of this product. Damage to the product resulting from acts of God, faulty installation, misuse, tampering, accident, abuse, negligence, alterations, unauthorized repairs or problems unrelated to materials and workmanship are not covered by this warranty.

Maxima[®] PRO 45L

by  HENRY SCHEIN[®]

45° Fiber-optic handpiece

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CAUTION: Federal (US) law restricts this device to sale by or on the order of a licensed professional

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