REF 570-4342 with Foot Pedal  
REF 570-4343 without Foot Pedal

# ACCLEAN®

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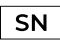
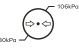
by  HENRY SCHEIN®

## INDICATION FOR USE

The Acclean Cordless Hygiene Handpiece is a cordless prophylaxis handpiece featuring control buttons and equipped with a battery pack and a wireless foot control. It is designed for use with disposable prophylaxis angles in hygiene operatory to perform cleaning and polishing procedures on teeth surface and fillings.

## GRAPHICAL SYMBOL EXPLANATION

 Consult Instructions for Use	 Catalogue Number
 Caution	 Serial Number
 Atmospheric Pressure Limits	 Date of Manufacture
 Temperature Limits	 Type BF Applied Part
 Humidity Limits	

## ESSENTIAL PERFORMANCE

The Acclean Cordless Hygiene Handpiece is intended to clean and polish teeth by maintaining consistent operational speeds of 500 to 3000 RPM and deliver sufficient torque to prevent stalling during normal clinical use. These parameters are critical to achieving the intended purpose of cleaning and polishing teeth without posing an unacceptable risk to the patient or dental professional.

## DEVICE DESCRIPTION

The Acclean Cordless Hygiene Handpiece system is a cordless handpiece which is intended for use by dental professionals for cleaning and polishing teeth. This is a general hygiene procedure that is performed on people of all ages in a professional dental operatory. The Cordless Hygiene Handpiece system is comprised of a cordless, battery-powered handpiece, a removable and autoclavable nosecone, an AC powered battery charging station, a battery-powered wireless foot control(optional) and an AC adapter. Additionally, the handpiece of Cordless Hygiene Handpiece must be used with a disposable sleeve and disposable prophylaxis angle. The handpiece features a removable nosecone that is to be cleaned and steam sterilized prior to first use and after each patient's use.

## WARNINGS

- People with cardiac pacemakers, defibrillators and other active implanted medical devices are cautioned that some types of electronic equipment might interfere with the operation of the device. Dental professionals should contact the device manufacturer or the patient's physician for specific recommendations.
- Do not use in the presence of flammable anesthetic mixture with air, oxygen, or nitrous oxide.
- Do not modify the Acclean Cordless Hygiene Handpiece. Modifications will invalidate the warranty and may endanger the patient and/or operator.
- The handpiece, wireless foot pedal, charging base, battery pack, and power supply cord are not waterproof. To prevent damage to the equipment do not immerse any of these components in water or chemical solution.
- The handpiece, battery pack, foot pedal, and charger base cannot be sterilized. Do not attempt to sterilize these components. Only the handpiece nosecone should be sterilized.
- Always operate the handpiece with the fitted nosecone.

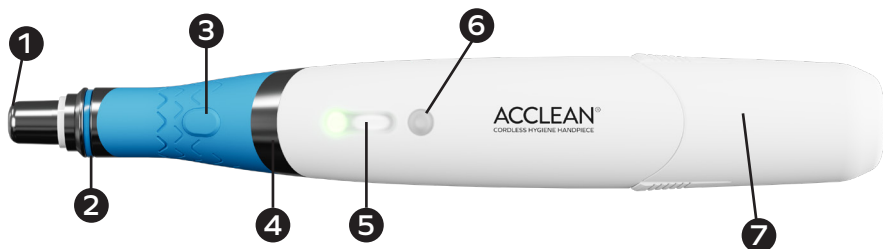
## GENERAL OPERATION/PRECAUTIONS

- For professional use only.
- Use personal protective wear during use.
- Sterilize nosecones upon first use.
- Perform a test run outside the patient's oral cavity prior to every treatment. If overheating or any abnormalities occur, stop using immediately and contact Henry Schein Handpiece & Small Equipment Solutions at 800-235-6337.
- Keep away from heat and direct sunlight.
- Always install a handpiece barrier sleeve before use to prevent contamination of the handpiece.
- Long term use of the equipment may result in overheating of the motor handpiece, thus it should be left to cool for next use. If the motor handpiece is overheating frequently contact Henry Schein Handpiece & Small Equipment Solutions at 800-235-6337.

## PARTS & CONTROLS

### CONTENT

1. Handpiece
2. 3 Removable Nosecones
- 2 Battery Packs
4. Charging Base
5. Power Cord & Adapter
6. Acclean Barrier Sleeve Sample Pack (100 pcs)
7. Acclean Disposable Prophy Angle Sample Pack (6 pcs)
8. (Optional) Wireless Foot Pedal

### CONTROLS

1. Disposable Prophy Angle Connection Point
2. Color Band
3. Motor On/Off Button
4. Autoclavable Nosecone
5. LED Indicator
6. Speed Mode Button
7. Removable Battery Pack

### OPERATION SETTINGS - HANDPIECE

SETTINGS	INDICATIONS	RPM RANGES
Handpiece Setting 6	LED 1 On, LED 2 On, LED 3 On	3000 RPM
Handpiece Setting 5	LED 1 On, LED 2 On, LED 3 Flashing	2500 RPM
Handpiece Setting 4	LED 1 On, LED 2 On, LED 3 Off	2000 RPM
Handpiece Setting 3	LED 1 On, LED 2 Flashing, LED 3 Off	1500 RPM
Handpiece Setting 2	LED 1 On, LED 2 Off, LED 3 Off	1000 RPM
Handpiece Setting 1	LED 1 Flashing, LED 2 Off, LED 3 Off	500 RPM

Battery Life (100% to 25%)	Green Solid LED
Battery Life (25% to 0%)	Amber Solid LED

## OPERATION SETTINGS - FOOT PEDAL

SETTINGS	INDICATIONS	RPM RANGES
Foot Pedal Mode	Manually Adjusted with Foot Pedal	Variable Speeds
Cruise Control	Locks in the Desired Speed	Desired Speeds

### AUTO POWER OFF

- When approximately 5 minutes have elapsed with no operation of the ON/OFF button, speed button, or foot pedal the power is automatically turned off to save energy and prevent accidental operation.

### LAST MEMORY

- The current setting (last memory) is always memorized when turning off the motor handpiece.

## SETUP

1. Sterilize removable nosecones by autoclave before first use and after each patient.
2. Place the charging base on a level surface and locate the power cord.
3. Connect the power cord & adapter (USB-C to USB-C) and then plug it into the back of the charging base. Plug the adapter into a power outlet.
4. Insert the two battery packs into the charging base. A full charge takes around 65 minutes to complete. Solid white ring light means battery is fully charged.
5. Charge the foot pedal by either plugging it into the same cord as the charging base (USB-C to USB-C) or by taking a (USB-A to USB-C) cord, plugging it into the back of the charging base and then into the foot pedal.

### BARRIER SLEEVE

1. Always install a handpiece barrier sleeve before use to prevent contamination of the handpiece. The barrier sleeve does not replace cleaning and disinfection of the handpiece. Please clean and disinfect the handpiece after each patient.
2. Ensure handpiece motor is turned off and nosecone is removed
3. Place the disposable barrier sleeve over the internal shoulder of the handpiece.
4. Place the removeable nosecone over the disposable barrier sleeve.
5. The nosecone should assemble with minimal effort. If excessive effort is need for assembly, the barrier sleeve should be inserted further down the handpiece.
6. Remove the barrier sleeve after each patient and discard.

## DISPOSABLE PROPHY ANGLE

1. Any disposable prophy angle will work with the Acclean Cordless Hygiene Handpiece.
2. Align the disposable prophy angle notch with the raised knob on the handpiece nosecone.
3. Slide into place and check that it is securely in place and completely flush with the nosecone.
4. Remove the disposable prophy angle after each patient and discard.

## PAIRING (OPTIONAL) FOOT PEDAL

For your convenience the handpiece comes paired with the foot pedal if bought together. If bought separately or if either the handpiece or foot pedal are replaced you will need to pair the foot pedal to the handpiece.

1. Press and hold the speed button for 5 seconds.
2. The handpiece enters pairing mode and the blue LED begins flashing.
3. If the foot pedal is already paired to a handpiece press the reset button on the bottom of the foot pedal.
4. Tap the foot pedal to enter pairing mode on the foot pedal.
5. Once the pairing is successful the handpiece's blue LED will remain on.

## PRINCIPLE OF OPERATION

### OPERATION

1. Insert a new barrier sleeve, sterilized nosecone, and new disposable prophy angle according to the instructions above.
2. Check that the handpiece battery life LED is green. If the LED is amber, remove the attached battery pack and replace it with secondary battery pack.
3. Perform a test run outside the patient's oral cavity prior to every treatment to look for overheating or abnormalities.

### HANDPIECE OPERATION

4. Tap the speed button to cycle through and select the desired starting speed.
5. Load the polishing paste to the disposable prophy angle.
6. Bring the disposable prophy angle close to the patient's mouth before taping the on/off button.
7. Begin treatment. Tap on/off button to turn off the motor when reapplying polishing paste to the disposable prophy angle.

### FOOT PEDAL (OPTIONAL) OPERATION

8. Check that foot pedal is paired by looking for the blue LED on the handpiece and taping the foot pedal.

9. Load the polishing paste to the disposable prophylaxis angle.
10. Bring the disposable prophylaxis angle close to the patient's mouth stepping on the foot pedal.
11. Begin treatment. Pressing on the foot pedal controls the rotation speed of the disposable prophylaxis angle.

## FOOT PEDAL CRUISE CONTROL MODE

12. While operating the foot pedal tap the on/off button on the handpiece to enter cruise control mode by locking the current speed. After entering cruise control mode the rotation speed will remain locked even when the foot pedal is no longer being pressed.
13. To exit cruise control mode either tap the foot pedal, press the on/off button, or press the speed button

## MAINTENANCE & CLEANING

### CLEANING, DISINFECTING, AND STERILIZATION

Perform the following steps after each patient use to ensure the Acclean Cordless Hygiene Handpiece is cleaned, disinfected, and sterilized properly prior to being used on the next patient.

1. Disassemble by removing the disposable prophylaxis angle from the nosecone, then the nosecone from the handpiece and then the barrier sleeve from the handpiece. Discard the disposable prophylaxis angle and barrier sleeve.

### NOSECONE

*Caution: Do not use disinfectant solution on sterilizable nosecone.*

2. Using a lint-free cloth dampened with an enzymatic detergent, thoroughly wet the nosecone with the prepared detergent. Using a soft bristled brush, brush the nosecone until all visible soil is removed.
3. Rinse the nosecone under running tap water for a minimum of 30 seconds to remove detergent residue.
4. Dry the nosecone using a clean lint-free cloth.
5. Visually inspect each nose cone for visible soil. If soil is seen, repeat the cleaning procedure.
6. Sterilization
  - 6-1. Ensure that the device is thoroughly dry prior to sterilization.
  - 6-2. Enclose the device into an appropriately sized sterilization pouch intended for steam sterilization; **no more than 1 per pouch.**
  - 6-3. Place the pouched device into the sterilizer per the manufacturer's instructions.
  - 6-4. Sterilize the device according to one of the recommended methods listed below.

**Sterilize metal and plastic components in separate pouches to avoid damage to the components. DO NOT exceed 273°F (134°C).**

**Gravity-displacement steam sterilization:** place in autoclave in accordance with: ANSI/AAMI ST55 at 132°C for 15 minutes and drying duration is 30 minutes. **Dynamic-air-removal steam sterilization:** place in autoclave in accordance with EN13060 at 134°C for 4 minutes and drying duration is 15 minutes.

7. To maintain sterility, the nosecone should remain bagged until ready to use.

## HANDPIECE, BATTERY PACK, CHARGING BASE, FOOT PEDAL

*Caution: The handpiece, battery pack, charging base, foot pedal, and power cords are not sterilizable by autoclave.*

*Caution: Do not apply disinfecting solution directly to Acclean Cordless Hygiene Handpiece system.*

8. Clean work area surfaces and Acclean Cordless Hygiene Handpiece system (including handpiece, battery packs, power cords, foot pedal, and charging base) with an enzymatic instrument cleaning solution.
9. Generously spray cleaning solution on a clean cloth or use a disposable lint-free cloth. Wipe the outer surfaces of the handpiece, battery pack, charging base, foot pedal, and power cords. Discard used cloth.
10. Allow cleaning solution to air dry.

## ACCESSORIES

570-4803	Nosecone
570-4804	Battery Pack
570-4805	Foot Pedal
570-4806	Replacement Handpiece
570-4807	Charging Base
570-4808	Power Cord & Adapter
570-4809	Color Band Blue (10/Bag)
570-4810	Color Band Green (10/Bag)
570-4811	Color Band Orange (10/Bag)
570-4812	Color Band Purple (10/Bag)
570-4802	Color Band Red (10/Bag)

## TROUBLESHOOTING

PROBLEM	SOLUTION
<b>DPA is not revolving.</b>	1. Make sure the nose cone and the DPA are securely connected.
	2. Confirm that the cordless handpiece main body and nose cone are properly snapped together.
	3. Check that the cordless handpiece is turned on and fully charged. If the lights don't come on, replace the battery.

PROBLEM	SOLUTION
<b>DPA is not revolving.</b>	4. If you're using the optional wireless foot pedal and it's not connected to the Power Supply, ensure the foot pedal is switched on or charged. Alternatively, plug the wireless foot pedal into the Power Supply so it runs off that power. Once the pedal is powered, check that it's synchronized with the cordless handpiece (The foot pedal will work while charging).
	5. Check the DPA can spin freely.
	6. Inspect the nose cone for damage by removing the cordless handpiece main body but leaving the DPA connected, then spin the DPA cup between your fingers. If it doesn't spin easily, add 1-2 drops of FDA-approved lubricant to the nose cone and try again. If the cup still won't spin, contact Technical Support. If you use lubricant, sterilize the nose cone before use (follow the guidelines in Section 8 on Infection Control Procedures).
	7. If the DPA still doesn't spin, remove the nose cone from the cordless handpiece main body and check if the main body motor spins when turned on via pedal or power button.
	8. Contact Technical Support for further assistance.
<b>Excessive noise or vibration during operation.</b>	<ol style="list-style-type: none"> <li>1. Ensure the nose cone is properly aligned with the main body of the cordless handpiece.</li> <li>2. Try using a new disposable prophylaxis angle (DPA).</li> <li>3. Check for any damaged, worn, or broken parts.</li> <li>4. Contact Technical Support for further assistance.</li> </ol>
<b>Optional Wireless Foot pedal does not charge.</b>	<ol style="list-style-type: none"> <li>1. Make sure the power cord is firmly connected to both the wall outlet and the optional wireless foot pedal.</li> <li>2. Ensure the LED under the USB connection is illuminated.</li> <li>3. Contact Technical Support for further assistance.</li> </ol>
<b>System intermittently shuts down.</b>	<ol style="list-style-type: none"> <li>1. The handpiece may shut down if exposed to strong non-compliant Radio Frequency (RF) interference from nearby devices.</li> <li>2. Contact Technical Support for further assistance.</li> </ol>
<b>Difficulty removing nosecone from cordless handpiece Main Body.</b>	<ol style="list-style-type: none"> <li>1. Contact Technical Support for further assistance.</li> </ol>
<b>Cordless Handpiece does not have the correct speed control.</b>	<ol style="list-style-type: none"> <li>1. Inspect the components for any visible debris.</li> <li>2. Contact Technical Support for further assistance.</li> </ol>
<b>Charging base for cordless handpiece does not drain liquids.</b>	<ol style="list-style-type: none"> <li>1. Clear any debris from the hole near the battery connector on the cordless handpiece charging base.</li> <li>2. Contact Technical Support for further assistance.</li> </ol>
<b>Battery does not charge.</b>	<ol style="list-style-type: none"> <li>1. Clean the charging contacts on both the battery and its charging base using one of the approved cleaning solutions</li> <li>2. Unplug and reconnect the power supply to the cordless handpiece charging base, then ensure that the LEDs are illuminated.</li> <li>3. Confirm that the Battery is properly seated in the charging base and that there are no obstructions.</li> <li>4. The battery temperature is too high.</li> <li>5. The rechargeable battery is fully charged or nearly full.</li> <li>6. The battery's temperature is too low.</li> <li>7. Contact Technical Support for further assistance.</li> </ol>

PROBLEM	SOLUTION
Battery does not hold charge.	1. Contact Technical Support for further assistance.
Power supply overheating.	1. Immediately unplug the unit and contact Technical Support.
Power cord is frayed or damaged in any way.	1. Discontinue use and contact Technical Support.
The motor does not rotate.	1. Unplug and reconnect the battery, ensuring it is properly connected. 2. Contact Technical Support for further assistance.

## TECHNICAL SPECIFICATIONS

### HANDPIECE SPECS

Rated Input	3.7 [V] 5 [A]
Rated Output	15 [W]
Speed	500 RPM - 3000 RPM
Charging Time	65 [min]
Dimensions (with battery) Dimensions (without battery)	179.32 x 27.12 x 27.12 [mm] 138.22 x 27.12 x 27.12 [mm]
Weight of handpiece	65.1 [gr] or 2.3 [oz]
Weight of nosecone	8.1 [gr] or 0.285 [oz]
Weight of battery	38.5 [gr] or 1.36 [oz]
Weight of handpiece with nosecone and battery	112.7 [gr] or 3.98 [oz]
Battery	3.7 [V] / 1200 [mAh]

### CHARGING BASE SPECS

Rated Input	11 [V] 2.25 [A]
Dimensions	131.03 x 62.1 x 56.31 [mm]
Weight	186 [gr] or 6.54 [oz]

### FOOT PEDAL SPECS

Rated Input	5 [V] and 1 [A]
Charging Time	60 [min]
Dimensions	47.3 x 118 x 118 [mm]
Weight	207 [gr] or 7.31 [oz]

## AC ADAPTOR SPECS

Rated Input	USB-C connector, Must contain PD & 25 [W]
Dimensions	44 x 39 x 26.8 [mm]
Weight	49.8 [gr]

	Temperature	Humidity	Atmospheric Pressure
Use Environment	+5°C to +40°C	25% RH ~ 85% RH	80kPa ~ 106kPa
Transportation & Storage Environment	-20°C ~ +50°C	25% RH ~ 80% RH	80kPa ~ 106kPa

## WIRELESS COMMUNICATIONS

Frequency	2400 to 2484 MHz
Power	1mW
Channels	126 RF channels

## SERVICE

For items that require service, including warranty service, please send to Henry Schein Handpieces & Small Equipment Solutions.

Henry Schein Handpieces & Small Equipment Solutions  
620 S. Placentia Ave.  
Placentia, CA 92870 USA  
Phone # 800-235-6337

## WARRANTY

This product is designed for use in a dental office and this warranty is not applicable to other uses. This product is warranted against manufacturing defects in materials and workmanship for 2 years from date of purchase. Henry Schein Handpieces & Small Equipment Solutions will repair or replace product at its sole discretion. This warranty shall be limited to replacement or repair of the product or its parts and shall not extend to any other claims, including but not limited to loss of profit, cost of removal or replacement, incidental or consequential damages or other similar claims arising from use of this product. Damage to product resulting from acts of God, faulty installation, misuse, tampering, accident, abuse, negligence, alterations, unauthorized repairs or problems unrelated to materials and workmanship are not covered by this warranty.

## EMC DECLARATIONS AND STATEMENTS

### DECLARATION - ELECTROMAGNETIC EMISSIONS

Emissions test	Compliance	Electromagnetic environment - guidance
<b>RF emissions CISPR 11</b>	<b>Group1 Class A</b>	The Cordless Hygiene Handpiece uses RF energy only for its internal function. Therefore, its RF emissions are very low and are not likely to cause any interference in nearby electronic equipment.
<b>Harmonic emissions IEC 61000-3-2</b>	<b>Class A</b>	The Cordless Hygiene Handpiece is suitable for use in all establishments other than domestic, and may be used in domestic establishments and those directly connected to the public low-voltage power supply network that supplies buildings used for domestic purposes, provided the following warning is heeded: Warning: This equipment/system is intended for use by healthcare professionals only. This equipment/system may cause radio interference or may disrupt the operation of nearby equipment. It may be necessary to take mitigation measures, such as re-orienting or relocating the Cordless Hygiene Handpiece or shielding the location.
<b>Voltage Fluctuations And Flicker IEC 61000-3-2:2013</b>	<b>Complies</b>	

### DECLARATION - ELECTROMAGNETIC IMMUNITY

IMMUNITY test	IEC 60601 test level	Compliance level	Electromagnetic environment - guidance
<b>Electrostatic discharge (ESD) IEC 61000-4-2</b>	8 kV contact 2, 4, 8, 15kV air	8 kV contact 2, 4, 8, 15kV air	Floors should be wood, concrete or ceramic tile. If floors are covered with synthetic material, the relative humidity should be at least 30 %.
<b>Electrical fast transient/burst IEC 61000-4-4</b>	2 kV for power supply lines 1 kV for input/output lines	2 kV for power supply lines N/A	Mains power quality should be that of a typical commercial or hospital environment.
<b>Surge IEC 61000-4-5</b>	1 kV line(s) to line(s) 2 kV line(s) to earth 2 kV Signal input/output) to earth	1 kV line(s) to line(s) N/A  N/A	Mains power quality should be that of a typical commercial or hospital environment.
<b>Voltage dips, short interruptions and voltage variations on power supply input lines IEC 61000-4-11</b>	0% UT; 0.5cycle at 0°, 45°, 90°, 135°,180°, 225°, 270° and 315° 0% UT; 1cycle and 70% UT; 25/30 cycles Single phase at 0° 0% UT; 250/300 cycle	0% UT; 0.5cycle at 0°, 45°, 90°, 135°,180°, 225°, 270° and 315° 0% UT; 1cycle and 70% UT; 25/30 cycles Single phase at 0° 0% UT; 250/300 cycle	Mains power quality should be that of a typical commercial or hospital environment. If the user of the Cordless Hygiene Handpiece requires continued operation during power mains interruptions, it is recommended that the Cordless Hygiene Handpiece be powered from an uninterruptible power supply or a battery.

<b>Power frequency (50/60 Hz) magnetic field IEC 61000-4-8</b>	30 (A/m)	30 (A/m)	Power frequency magnetic fields should be at levels characteristic of a typical location in a typical commercial or hospital environment.
<b>Immunity from proximity frequency magnetic field</b>	65 A/m @ 134.2 kHz; PM 2.1kHz 50%; 7.5 A/m @ 13.56 MHz; PM 50kHz 50%	65 A/m @ 134.2 kHz; PM 2.1kHz 50%; 7.5 A/m @ 13.56 MHz; PM 50kHz 50%	
<b>NOTE UT</b> is the a.c. mains voltage prior to application of the test level.			

## DECLARATION - ELECTROMAGNETIC IMMUNITY

IMMUNITY test	IEC 60601 test level	Compliance level	Electromagnetic environment - guidance
<b>Conducted RF IEC 61000-4-6</b>	3 V 0,15 MHz – 80 MHz	3 V 0,15 MHz – 80 MHz	<p>Portable and mobile RF communications equipment should be used no closer to any part of the Cordless Hygiene Handpiece, including cables, than the recommended separation distance calculated from the equation applicable to the frequency of the transmitter.</p> <p>Recommended separation distance</p> $d = \left[ \frac{3.5}{E_1} \right] \sqrt{P}$ $d = \left[ \frac{12}{E_1} \right] \sqrt{P}$ $d = \left[ \frac{12}{E_1} \right] \sqrt{P} \quad 80 \text{ MHz to } 800 \text{ MHz}$ $d = \left[ \frac{23}{E_1} \right] \sqrt{P} \quad 800 \text{ MHz to } 2.5 \text{ GHz}$ <p>where P is the maximum output power rating of the transmitter in watts (W) according to the transmitter manufacturer and d is the recommended separation distance in metres (m). Field strengths from fixed RF transmitters, as determined by an electromagnetic site survey, should be less than the compliance level in each frequency range. D Interference may occur in the vicinity of equipment marked with the following symbol:</p> 
<b>Radiated RF IEC 61000-4-3</b>	6 V in ISM bands between 0,15 MHz and 80 MHz 80 % AM at 1 kHz  3V/m from 0.15 to 2.7MHz 80MHz; 80 % AM at 1 kHz	6 V in ISM bands between 0,15 MHz and 80 MHz 80 % AM at 1 kHz e)  3V/m from 0.15 to 2.7MHz, 80 % AM at 1 kHz	

<b>Immunity from proximity frequency magnetic field</b> <b>Section 8.11</b>	65 A/m @ 134.2 kHz; PM 2.1kHz 50%; 7.5 A/m @ 13.56 MHz; PM 50kHz 50%	65 A/m @ 134.2 kHz; PM 2.1kHz 50%; 7.5 A/m @ 13.56 MHz; PM 50kHz 50%	
--	---	---	--

<b>Test specifications for ENCLOSURE PORT IMMUNITY to RF wireless communications equipment</b>						
Test Frequency (MHz)	Band (MHz)	Service	Modulation	Maximum power (W)	Distance (m)	Immunity test level (V/m)
385	380 - 390	TETRA 400	Pulse modulation 18Hz	1.8	0.3	27
450	430 - 470	GMRS 460, FRS 460	FM $\pm$ 5kHz deviation 1KHz sine	2	0.3	28
710 745 780	704 - 787	LTE Band 13, 17	Pulse modulation 217Hz	0.2	0.3	9
810 870 930	800 - 960	GSM 800/900, TETRA 800, iDEN 820, CDMA 850, LTE band 5	Pulse modulation 18Hz	2	0.3	28
1720 1845 1970	1700 - 1990	GSM 1800, CDMA 1900, DECT, LTE band 1,3,4,25 UMTS	Pulse modulation 217 Hz	2	0.3	28
2450	2400 - 2570	Bluetooth, WLAN, 802.11 b/g/n, RFID 2450, LTE band 7	Pulse modulation 217 Hz	2	0.3	28
5240 5500 5785	5100 - 5800	WLAN 802.11 a/n	Pulse modulation 217 Hz	2	0.3	9

**Recommended separation distances between portable and mobile RF communications equipment and the Cordless Hygiene Handpiece**

Rated maximum output power of transmitter <b>W</b>	Separation distance according to frequency of transmitter <b>m</b>			
	150 kHz to 80 MHz outside ISM bands	150 kHz to 80 MHz in ISM bands	80 MHz to 800 MHz	800 MHz to 2,5 GHz
	$d = \left[ \frac{3,5}{V_1} \right] \sqrt{P}$	$d = \left[ \frac{12}{V_2} \right] \sqrt{P}$	$d = \left[ \frac{12}{E_1} \right] \sqrt{P}$	$d = \left[ \frac{23}{E_1} \right] \sqrt{P}$
0.01	0.12	0.2	0.2	1
0.1	0.7	0.64	1.3	2.6
1	1.17	2	4	8
10	3.7	6.4	13	26
100	11.7	20	40	80

“WARNING: Use of this equipment adjacent to or stacked with other equipment should be avoided because it could result in improper operation. If such use is necessary, this equipment and the other equipment should be observed to verify that they are operating normally.”

“WARNING: Use of accessories, transducers and cables other than those specified or provided by the manufacturer of this equipment could result in increased electromagnetic emissions or decreased electromagnetic immunity of this equipment and result in improper operation.”

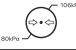
“WARNING: Portable RF communications equipment (including peripherals such as antenna cables and external antennas) should be used no closer than 30 cm (12 inches) to any part of the Cordless Hygiene Handpiece, including cables specified by the manufacturer. Otherwise, degradation of the performance of this equipment could result.”

“NOTE The EMISSIONS characteristics of this equipment make it suitable for use in industrial areas and hospitals (CISPR 11 class A). If it is used in a residential environment (for which CISPR 11 class B is normally required) this equipment might not offer adequate protection to radio-frequency communication services. The user might need to take mitigation measures, such as relocating or re-orienting the equipment”

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