
# *Maxima*<sup>®</sup>

by  HENRY SCHEIN<sup>®</sup>

## **INSTRUCTIONS FOR USE**

Read before use. Keep in safe place for future reference.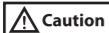

- Do not modify or alter coupler.
- If handpiece leaks air or water and or does not run up to speed, check o-ring seals on coupler.
- Never attempt to remove the coupler while running handpiece.
- If any abnormalities are found, stop using immediately and contact Henry Schein<sup>®</sup> ProRepair.

## SPECIFICATIONS

Item Code	Description	Weight	Finish	Light Source
570-0110	Maxima MULTIflex Non-Optic 4-Hole Coupler	24 g	Titanium	N/A
570-0111	Maxima MULTIflex Fiber-Optic 5-Hole Coupler	26 g	Titanium	Glass Rod Transfer
570-0109	Maxima MULTIflex Fiber-Optic 6-Pin LED Coupler	25 g	Titanium	LED
570-1080	Maxima MULTIflex Non-Optic 4-Hole Coupler	24 g	Chrome	N/A
112-5022	Maxima MULTIflex Fiber-Optic 5-Hole Coupler	26 g	Chrome	Glass Rod Transfer

112-5021	Maxima MULTIflex Fiber-Optic 6-Pin Coupler	25 g	Chrome	Halogen Bulb
112-5482	Maxima MULTIflex Fiber-Optic 6-Pin LED Coupler	25 g	Chrome	LED

### **AIR PRESSURE**

Set air pressure at handpiece connection to 35 - 38 psi.

- Exceeding recommended air pressure may result in premature cartridge failure.

## OPERATION


### Coupler Connection (Fig. 1)

1. Align tubes and pins on coupler with holes in delivery system tubing.
2. Thread tubing nut to bottom of coupler.
3. Hold coupler with provided wrench and finger tighten nut.

*Fig. 1*

### Handpiece Connection (Fig. 2)

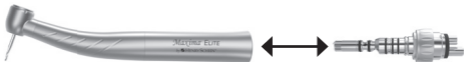*Fig. 2*

### Attaching handpiece

1. Slide handpiece onto MULTIflex® coupler.
2. Press firmly until it snaps into place.

**Note:** Handpiece should swivel freely with slight tension and sit flush on coupler.

## Removing handpiece

Hold ring on coupler and pull handpiece straight off.

**Note:** Do not hold or pull on hose.

## MAINTENANCE

- Wear appropriate personal protective equipment during maintenance process. This should include eye protection and gloves.

## Cleaning

1. Clear fiber-optic surfaces of particulates by spraying with air syringe.
2. Wipe fiber-optic surfaces with a soft cloth or cotton swab damp with alcohol.

## Changing O-Rings

- Improper care of o-rings may cause handpiece to malfunction.
- When changing/replacing coupler o-rings, always replace complete set of 5 pieces.


**Note:** Coupler o-rings may be lubricated with Henry Schein® Maxima HP Spray.

1. Use an explorer to gently remove old/worn o-rings.
2. Slide on new o-rings using only fingers to not cause damage.

### Changing Coupler LED Bulb (Fig. 3)

**Note:** LED bulb is polarity sensitive. If bulb does not light after changing, repeat steps 1 and 2, then rotate bulb 180° before reinstalling and replacing cap.

1. Remove LED bulb cap by turning it counterclockwise
2. Remove bulb.
3. Install new bulb and replace cap, by finger tightening only.

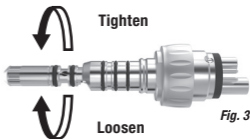### REPLACEMENT PARTS & ACCESSORIES

570-0140	O-Ring Kit Maxima Coupler (All)
570-0142	LED Replacement Bulb
570-0141	Halogen Replacement Bulb
100-5290	Replacement Gasket 4-Hole (Non-Optic)
100-5354	Replacement Gasket 5-Hole (Fiber-Optic)
570-0143	Replacement Gasket 6-Pin (LED)

## **SERVICE**

For items that require service, including warranty service, please send to Henry Schein® ProRepair®.

Henry Schein ProRepair  
620 S. Placentia Ave.  
Placentia, CA 92870 USA  
Phone # 714-579-0175  
Fax # 714-579-0186

## **WARRANTY**

These products are designed for use in a dental office and this warranty is not applicable to other uses. The products are warranted against manufacturing defects in materials and workmanship for 1 year from date of purchase.

Henry Schein® will repair or replace products at its sole discretion. This warranty shall be limited to replacement or repair of the products or their parts and shall not extend to any other claims, including but not limited to loss of profit, cost of removal or replacement, incidental or consequen-

tial damages or other similar claims arising from use of this product. Damage to products resulting from acts of God, faulty installation, misuse, tampering, accident, abuse, negligence, alterations, unauthorized repairs or problems unrelated to materials and workmanship are not covered by this warranty.

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