PRO 2-M LOWSPEED ATTACHMENTS

- ① EN Lowspeed Attachment
- ② ES Cabeza de baja velocidad
- ③ FR Tête basse

INSTRUCTIONS FOR USE

Read before use. Keep in a safe place for future reference.

Caution: Federal (US) law restricts this device to sale by or on the order of a licensed professional

- For professional use only.
- Sterilization is required before first use.
- Do not disassemble or alter the motor, attachments or heads.
- Before each use check for vibration, noise and overheating. If any abnormalities are observed, stop using immediately and contact Henry Schein ProRepair.
- Do not drop, hit or subject to excessive shock.
- The user is responsible for correct operation, maintenance, and inspection.
- Do not wipe or immerse in acidic solutions.

- Never operate the straight attachment without a bur.
- Do not exceed the recommended air pressure. Excessive air pressure may cause premature failure.

INTENDED USE

The Henry Schein PRO 2-M Straight & Contra Angle Attachments are intended to be used on Midwest® brand low-speed motors.

SPECIFICATIONS

Attachments

Item	Gear Ratio	Speed	Burs
Straight attachment (570-2864)	1:1	30,000 rpm max.	Ø 2.35 (ISO1797-1)
Contra angle sheath (570-2865)	1:1	30,000 rpm max.	N/A

CONNECTING / DISCONNECTING ATTACHMENTS (FIG. 1)

- Make sure attachment sits flush on top of motor. If not, repeat connecting process.
- Do not connect or disconnect attachments until drive motor has stopped completely.
- Loose and / or improper attachment causes excessive noise and vibration and may result in patient injury.

Connecting

To connect attachments to motor, slide attachment onto nose of motor until it clicks in place.

Disconnecting

To remove, depress flush mounted latch (Shorty® motor), or black latch ring on red indicator (Rhino® XP motor) and remove attachment from motor.

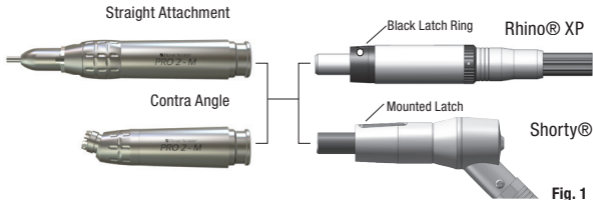
Fig. 1

INSERTING AND REMOVING THE BUR

Straight Attachment (Fig. 2)

- Make sure 2 black dots are aligned after inserting bur. Improper alignment may cause heat generation and result in burns.
- Always insert bur completely into chuck to prevent bur walk out.
- Never operate straight attachment without a bur.

Inserting the Bur

1. Twist the bottom section in the OPEN direction until it clicks.
2. Insert bur.
3. Turn the bottom section back until it clicks and the black dots align.

Removing the Bur

1. Turn the bottom section in the OPEN direction until it clicks.
2. Remove bur.

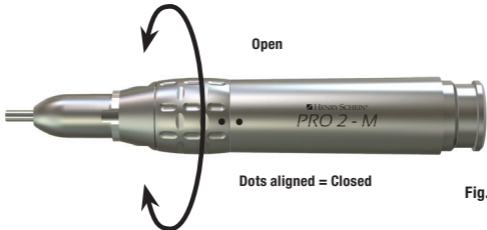
Fig. 2

Installing U-Style Angles (Fig. 3)


Fig. 3

The Straight Attachment accepts universal (U-Style) contra angles and prophy angles.

1. Twist bur locking ring until it stops to open the chuck.
2. Insert the shaft of U-Style angle into nose of the straight attachment and push until it is completely seated. Make sure that attachment index tab is seated into notch on the angle.

3. Close chuck by returning bur locking ring to its original position until it clicks in place and the two dots line up (Fig. 2).

Contra Angle Sheath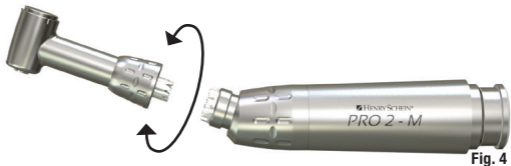

Mounting Angle Heads (Fig. 4)


1. Match castellated fitting on base of angle head with castellated fitting on top of contra angle sheath in the desired head position.

2. If swing style latch head, open bur latch.
3. Turn union nut in direction indicated in Fig. 4 until finger tight.

Removing Angle Heads (Fig. 4)

1. If swing style latch head, open bur latch.
2. Turn union nut in direction indicated in Fig. 4 to release head.
3. Remove angle head from contra angle sheath.

MAINTENANCE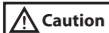

- Wear appropriate personal protective equipment during maintenance process. This should include eye protection and gloves.
- Do not use ultrasonic bath, boiling water or chemicals to clean hand-piece. Do not use a wire brush.

CLEANING & LUBRICATION (FIG. 5)

Note: Cleaning and lubrication is required before each sterilization cycle.

1. Remove attachment from motor.
2. Remove any material and debris using a soft cloth or brush that has

been moistened with alcohol or a soapy solution.

3. Wipe dry.
4. Attach E-Type spray adapter to Henry Schein Maxima HP Spray (112-6928) nozzle.
5. Insert into motor end of attachment and dispense lubricant for approximately 2-3 seconds.
6. Connect to motor and run attachment for approximately 10 seconds to expel excess lubricant.
7. Wipe off any excess lubricant with a soft cloth.


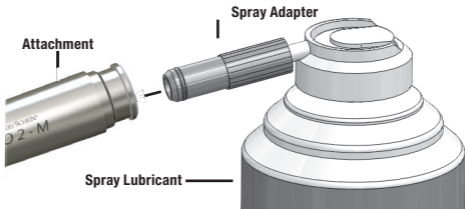
Fig. 5

STERILIZATION

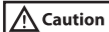- Sterilize by steam autoclave after each patient use.
- Sterilization is required before first use.
- Wear appropriate personal protective equipment during the sterilization process. This should include eye protection and gloves.
- Do not autoclave at temperatures that exceed 275° F (135° C).
- Do not autoclave together with other instruments.
- Do not use dry heat sterilization.
- Keep in sterilization pouch until ready to use.

1. Remove bur, then wipe external surface with dry paper towel to remove any excess lubricant and debris. Place paper towel in bio-hazard bin when finished.
2. Place handpiece in sterilization pouch and seal. Bag handpieces individually; no more than 1 per pouch.
3. Sterilize in a gravity type autoclave for a minimum of 15 minutes or in a pre-vacuum autoclave for a minimum of 4 minutes at 270° F (132° C). Let autoclave cycle completely. Do not remove before handpiece is completely cool and pouch is dry.

ACCESSORIES

112-6928 Maxima HP Spray

112-6934 Maxima HP Spray Nozzle Adapter Kit

SERVICE

For items that require service, including warranty service, please send to Henry Schein ProRepair.

Henry Schein ProRepair
620 S. Placentia Ave.
Placentia, CA 92870 USA
Phone # 714-579-0175
Fax # 714-579-0186

WARRANTY

These products are designed for use in a dental office and this warranty is not applicable to other uses. The products are warranted against manufacturing defects in materials and workmanship for 1 year from date

of purchase.

Henry Schein® will repair or replace products at its sole discretion. This warranty shall be limited to replacement or repair of the products or their parts and shall not extend to any other claims, including but not limited to loss of profit, cost of removal or replacement, incidental or consequential damages or other similar claims arising from use of this product. Damage to products resulting from acts of God, faulty installation, misuse, tampering, accident, abuse, negligence, alterations, unauthorized repairs or problems unrelated to materials and workmanship are not covered by this warranty.

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