PRO 2-S LOWSPEED ATTACHMENTS

- ① EN Lowspeed Attachment
- ② ES Cabeza de baja velocidad
- ③ FR Tête basse

INSTRUCTIONS FOR USE

Read before use. Keep in a safe place for future reference.

Caution: Federal (US) law restricts this device to sale by or on the order of a licensed professional

- For professional use only.
- Sterilization is required before first use.
- Do not disassemble or alter the motor, attachments or heads.
- Before each use check for vibration, noise and overheating. If any abnormalities are observed, stop using immediately and contact Henry Schein ProRepair.
- Do not drop, hit or subject to excessive shock.
- The user is responsible for correct operation, maintenance, and inspection.
- Do not wipe or immerse in acidic solutions.

- Never operate the straight attachment without a bur.
- Do not exceed the recommended air pressure. Excessive air pressure may cause premature failure.

INTENDED USE

The Henry Schein PRO 2-S Straight & Contra Angle Attachments are intended to be used on Star® brand low-speed motors.

SPECIFICATIONS

Attachments

| Item | Gear Ratio | Speed | Burs |
|-----------------------------------|------------|-----------------|-----------------------|
| Straight attachment (570-2868) | 1:1 | 20,000 rpm max. | Ø 2.35 (ISO1797-1) |
| Contra angle sheath (570-2869) | 1:1 | 20,000 rpm max. | N/A |

CONNECTING / DISCONNECTING ATTACHMENTS (FIG. 1)

- Make sure attachment sits flush on top of motor. If not, repeat connecting process.
- Do not connect or disconnect attachments until drive motor has stopped completely.
- Loose and / or improper attachment causes excessive noise and vibration and may result in patient injury.

Connecting

1. To connect attachments to motor, pull back release ring while sliding attachment onto nose of motor.

2. Release ring to secure.

Disconnecting

1. To remove, pull back release ring while sliding attachment off nose of motor.

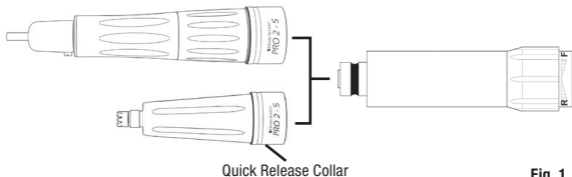
Fig. 1

INSERTING AND REMOVING THE BUR Straight Attachment (Fig. 2)

- Make sure 2 black dots are aligned after inserting bur. Improper alignment may cause heat generation and result in burns.
- Always insert bur completely into chuck to prevent bur walk out.
- Never operate straight attachment without a bur.


Inserting the Bur

1. Twist the bottom section in the OPEN direction until it clicks.
2. Insert bur.
3. Turn the bottom section back until it clicks and the black dots align.


Removing the Bur

1. Turn the bottom section in the OPEN direction until it clicks.

2. Remove bur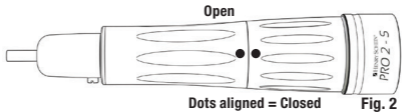

Contra Angle Sheath

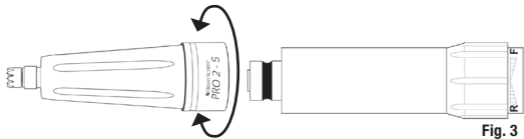Mounting Angle Heads (Fig. 3)

1. Thread head onto contra angle in direction indicated in Fig. 3 until finger tight.

Removing Angle Heads (Fig. 3)

1. Turn head in direction indicated in Fig. 3 to remove.

MAINTENANCE

- Wear appropriate personal protective equipment during maintenance process. This should include eye protection and gloves.
- Do not use ultrasonic bath, boiling water or chemicals to clean hand-piece. Do not use a wire brush.

NOTE: These attachments DO NOT require lubrication.

1. Disconnect handpiece and coupler from air supply tubing Leave the coupler in the handpiece.
2. Dispense handpiece cleaner into drive air tube of handpiece or coupler (Fig. 4).
3. Reconnect handpiece and run for 10 seconds or until handpiece runs smoothly.

Fig. 4

STERILIZATION

- Sterilize by steam autoclave after each patient use.
- Sterilization is required before first use.
- Wear appropriate personal protective equipment during the sterilization process. This should include eye protection and gloves.
- Do not autoclave at temperatures that exceed 275° F (135° C).
- Do not autoclave together with other instruments.
- Do not use dry heat sterilization.
- Keep in sterilization pouch until ready to use.

1. Remove bur, then wipe external surface with dry paper towel to remove

any excess lubricant and debris. Place paper towel in bio-hazard bin when finished.

2. Place handpiece in sterilization pouch and seal. Bag handpieces individually; no more than 1 per pouch.
3. Sterilize in a gravity type autoclave for a minimum of 15 minutes or in a pre-vacuum autoclave for a minimum of 4 minutes at 270° F (132° C). Let autoclave cycle completely. Do not remove before handpiece is completely cool and pouch is dry.

ACCESSORIES

112-6928 Maxima HP Spray

112-6934 Maxima HP Spray Nozzle Adapter Kit

SERVICE

For items that require service, including warranty service, please send to Henry Schein ProRepair.

Henry Schein ProRepair
620 S. Placentia Ave.
Placentia, CA 92870 USA
Phone # 714-579-0175
Fax # 714-579-0186

WARRANTY

These products are designed for use in a dental office and this warranty is not applicable to other uses. The products are warranted against manufacturing defects in materials and workmanship for 1 year from date

of purchase.

Henry Schein® will repair or replace products at its sole discretion. This warranty shall be limited to replacement or repair of the products or their parts and shall not extend to any other claims, including but not limited to loss of profit, cost of removal or replacement, incidental or consequential damages or other similar claims arising from use of this product. Damage to products resulting from acts of God, faulty installation, misuse, tampering, accident, abuse, negligence, alterations, unauthorized repairs or problems unrelated to materials and workmanship are not covered by this warranty.

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