
maxima™

ADVANCED

by HENRY SCHEIN®

Ultrasonic Cleaner MA100

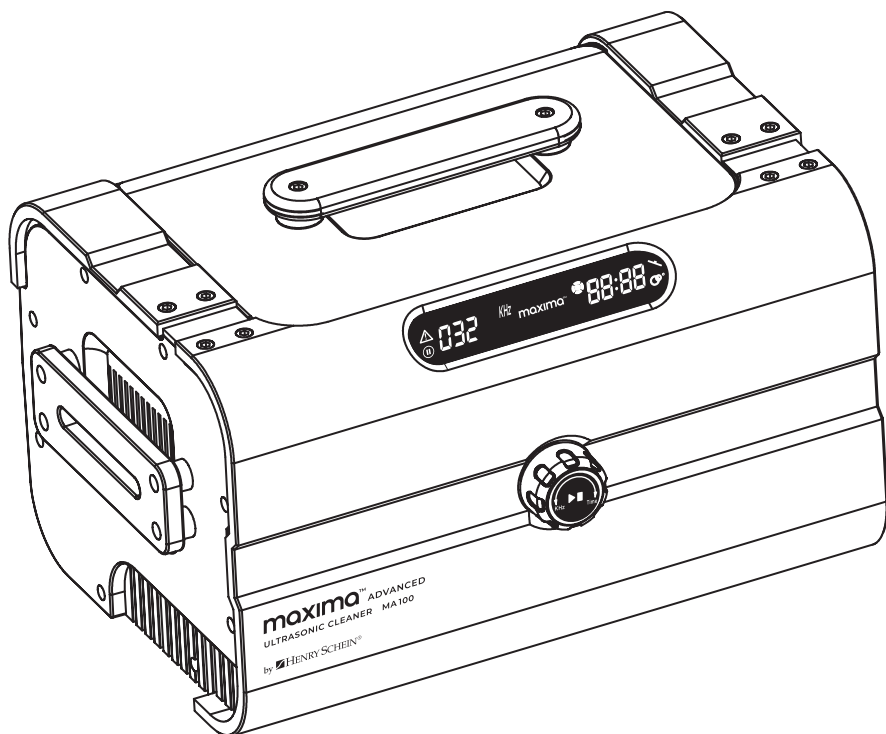REF 570-4851

INSTRUCTIONS FOR USE

Read before use. Keep in safe place for future reference.

INDICATIONS FOR USE

The Maxima Advanced Ultrasonic Cleaner MA100 is designed to thoroughly wash (remove blood, tissue, and small particles) instruments prior to disinfection. The system is suitable for use in dental offices to clean dental instruments.

INTENDED USE

The Maxima Advanced Ultrasonic Cleaner MA100 is intended for cleaning dental instruments by the emission of high frequency soundwaves.

INTRODUCTION

Principles of ultrasonic cleaning:

Millions of tiny air bubbles are generated inside the liquid by high frequency vibration. The air bubbles burst inside the liquid and shake off the debris to achieve the cleaning effect.

Purified water or distilled water has the same cleaning effect as regular tap water for ultrasonic cleaning. Therefore using tap water is sufficient.

Capability Of Maxima Advanced Ultrasonic Cleaner MA100

This dual-frequency ultrasonic cleaner uses 32 kHz for fast cleaning of surface stains (though with higher noise) and 48 kHz for deeper, more precise cleaning of crevices with less noise.

Features

- Tank capacity 4.0 L / 1.06 gallon. Tank size 24.8x17.4x10.3 cm/ 9.8"x6.9"x4.1"
- Dual frequency. 32 kHz for strong surface cleaning, 48 kHz for gentle precision cleaning
- Fast and quiet cleaning. Normal cleaning is 5 minutes per layer of instruments or cassettes. No more than 10-15 minutes per load. Quieter than most ultrasonic cleaners with similar power and capacity.
- Digital control. LED display, 5 timer settings, 2 frequency selections
- Two layers of overheat protection. A 30-minute cleaning time limit with a 10-minute enforced pause, and an internal heat sensor
- Single rotary control knob. Large single control knob for easy and intuitive operation
- Open lid warning. Warning indicator flashes for 15 seconds, followed by a 5-second warning beep.
- Settings memory. Remembers previous settings for easy repeated operation
- Auto degassing. Auto 2-minute degassing cycle after power on
- Moisture-resisting PCBs and anti-interference IC. Incorporates moisture protection for the PCBs and interference resistance for the ICs
- Two dual-bearing high speed cooling fans. Enhanced cooling system help prolong the service life of the machine
- Aluminum-wrapped plastic housing. The aluminum-wrapped plastic housing provides superior sturdiness and durability

Read the Manual First

The manual should be carefully reviewed before starting to use the device. Warnings should be observed carefully. Please follow the manual for operations.

TABLE OF CONTENTS

Safety Precautions	3
Important Safety Instructions	4
Product Structure And Accessories	5
Control Panel And Operations	6
Exploded View	7
Common Cleaning Methods	9
Operation Guidelines	10
Care and Maintenance	12
Specifications	13

SAFETY PRECAUTIONS

Keep it away from children!

- Please store the ultrasonic cleaner where it is not reachable by children.
- Danger to children! Danger for death through suffocation! Keep the packaging material away from children.
- This appliance shall not be used by children. Keep the appliance and its cord out of reach of children.

To prevent life-threatening electrical shock, please observe the following:

Danger of electrical shock! Do not use while bathing. Never immerse the device or the power cord in water or other liquid.

- **Danger of electrical shock!** Never touch the power plug with wet hands, especially when inserting or removing the plug.
- **Danger of electrical shock!** If the unit has fallen into water during operation, do not touch the unit. Remove the power plug from the socket first.
- **Danger of electrical shock!** Do not spray water or liquid over the device.
- Never operate the device unattended.
- Follow the manual to operate the device.
- Do not use components unapproved by the manufacturer.
- When removing the power cord from the socket, grab the power plug not the cord.
- To protect the power cord from damage, do not cause it to get caught by things such as a cupboard door or a chair leg; do not drag across a hot surface.
- If there is damage to the power plug, cord, housing, or other parts of the device, do not use the device.
- Do not disassemble the device, except by professionals.
- If the unit is damaged, non-operational or has fallen into water, take it to a qualified service provider.
- Remove the power plug from the socket
 - if malfunction occurs
 - before cleaning the device
 - if the device is not going to be used for prolonged period
 - after each use (recommended)
- The installation of an earth leakage circuit breaker with a rated tripping current of no

more than 30 mA provides further protection against an electrical shock. The installation should only be carried out by a trained electrician.

To prevent fire hazards, please observe the following:

- Never block the vents on the device. Keep the vents free from lint, hair and other materials.
- Do not place the device on a soft surface, such as a bed or a couch, where the vents could be blocked.
- Turn off the power switch at the end of the day.
- Observe the other warnings in the previous section.
- If the supply cord is damaged, it must be replaced by the manufacturer or its service agent or a similarly qualified person in order to avoid a hazard.

SAFETY PRECAUTIONS

Other observations

- Do not operate the product without filling the tank with water. Running dry will damage the unit.
- Do not fill the tank above the Max line after adding the instruments to avoid overspill.
- Do not use solution containing abrasive substances or strong corrosive chemical solution not recommended by the manufacturer or the supplier.
- Place the device on a dry and flat surface for operation.
- When the device is subjected to severe electromagnetic interference, it may malfunction, stop operating or lose control functions. If this happens, unplug the power cord then reinsert it to restart the device.

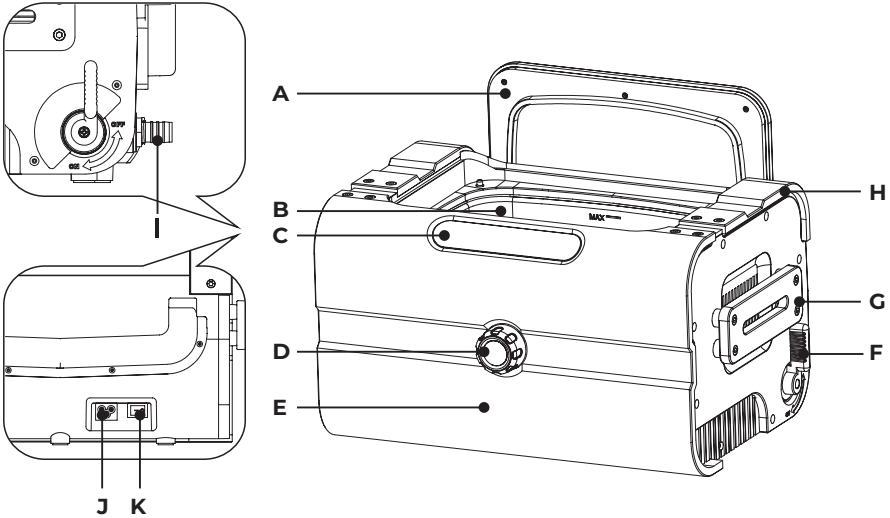
IMPORTANT SAFETY INSTRUCTIONS

When using electric appliances, basic precautions should always be followed, including the following:

- Read all the instructions before using the appliance.
- To reduce the risk of injury, close supervision is necessary when an appliance is used near children.
- Only use attachments recommended or sold by the manufacturer.
- Do not use outdoors.
- To disconnect, turn all controls to the off (0) position, then remove plug from outlet. Do not unplug by pulling on the cord. To unplug, grasp the plug, not the cord. Unplug from outlet when not in use and before servicing or cleaning.
- To reduce the risk of electrical shock, do not put the appliance in water or other liquid. Do not place or store appliance where it can fall or be pulled into a tub or sink.
- All servicing of this product, including transducer replacement, is to be conducted by qualified service personnel.
- Do not operate any appliance with a damaged cord or plug, or after the appliance malfunctions or is dropped or damaged in any manner.
- Return appliance to the nearest authorized service facility for examination, repair, or electrical or mechanical adjustment.

PRODUCT STRUCTURE AND ACCESSORIES

- A.** Lid
- B.** Stainless steel cleaning tank
- C.** Digital display
- D.** Set frequency/timer dial
- E.** Aluminum front panel
- F.** Drainage valve handle
- G.** Aluminum handle
- H.** Aluminum panel
- I.** Drainage spout
- J.** Power socket
- K.** Power switch

Standard Accessories

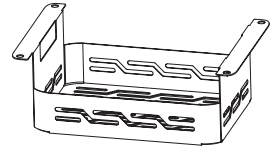
Power cord
(570-5432)

Drain hose with elbow
(570-3805)

Drain Strainer


Stainless steel basket
(570-5433)

Optional Accessories

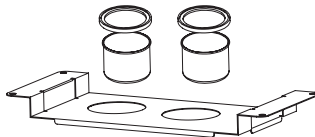
Stainless steel removable cleaning chamber
(570-5407)

Stainless steel beaker (2pcs/set) and holder
(570-5406)

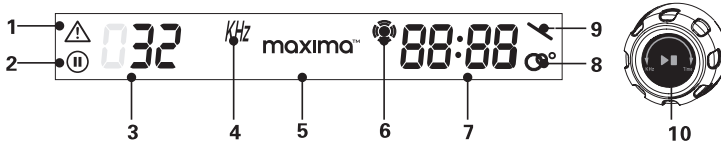CONTROL PANEL AND OPERATIONS

1. Overheating indicator

A flashing overheating indicator signals internal overheating or other malfunctions, automatically pausing operation. To resume, the internal temperature must drop below the system's set limit. Power cycling will not override this safety feature.

2. Pause indicator

A flashing pause indicator signals a mandatory 10-minute pause. While power cycling can override this, doing so repeatedly may damage the system.

32

3. Frequency indicator.

Indicates current operating ultrasonic frequency

KHz

4. KHZ frequency symbol

maxima™

5. Brand label

After 3 minutes of inactivity, the control panel enters power save mode

6. Ultrasonic cleaning indicator.

Indicates active ultrasonic cleaning

10:00

7. Cleaning time

A count down clock indicates the remaining cleaning time

8. Degas indicator

Indicates degassing mode in operation.

9. Open lid indicator

Indicates an open lid during normal operation.

10. Control knob

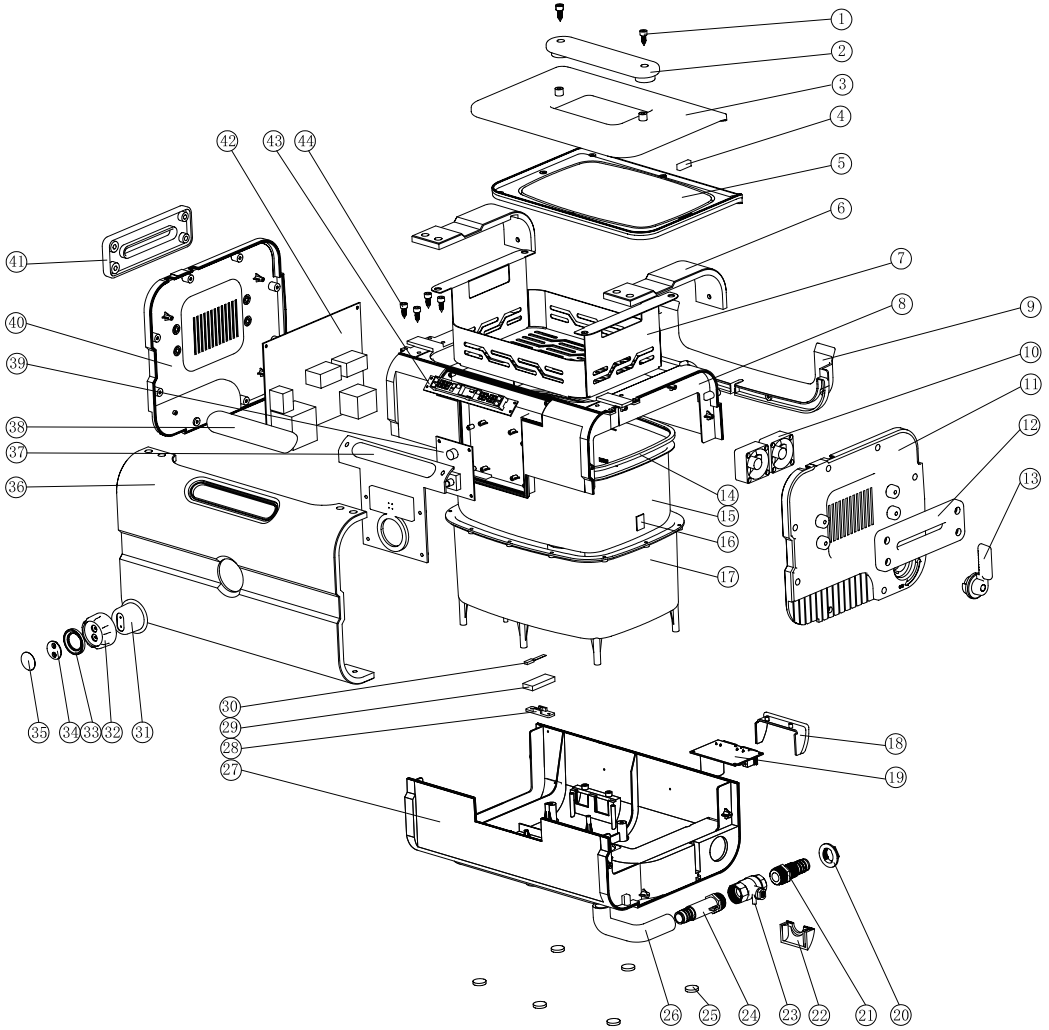
a. Counter-clockwise - Turn the knob counter-clockwise to select the ultrasonic frequency (32 or 48kHz) and push the knob in to set it before starting the cleaning cycle. The frequency cannot be changed during operation.

b. Clockwise - Turn the knob clockwise to select from 5 cleaning times.

→ 05:00 → 07:30 → 10:00 → 12:30 → 15:00 ↵

Press the knob to start or stop cleaning


EXPLODED VIEW

NO.	ITEM	MATERIAL
1	Stainless steel screw	SUS 304
2	Aluminum lid handle	ABS
3	Lid-top	ABS
4	Magnet	NdFeB magnet
5	Lid-under	ABS
6	Aluminum corner guards	AL6063
7	Stainless steel basket	SUS 304
8	Top housing	ABS
9	Lid holder	ABS
10	High-speed cooling fan	Standard parts
11	Right panel	ABS
12	Right handle	ABS
13	Drainage valve handle	ABS+ copper
14	Silicone gasket	Silicone
15	Stainless steel tank	SUS 304
16	Open-lid inductive PCB	94V0
17	Inner tank housing	ABS
18	Power socket canopy	ABS
19	Power socket and filter PCB	94V0
20	Locking Nut	ABS
21	Drain spout	PP
22	Drain spout holder	ABS
23	Stainless steel drain valve	SUS 304
24	Drain fitting	PP
25	Silicone grommets	Silicone
26	Rubber elbow	NBR
27	Bottom housing	ABS
28	Temperature sensor holder	PA+30% glass fiber
29	Foam	EVA
30	Temperature sensor	Standard parts
31	Inner knob	ABS
32	Outer knob cover	ABS
33	Stainless steel ornament	SUS 304
34	Locking cover of knob	ABS
35	Sticker	PC
36	Aluminum front panel	AL6063
37	Control panel cover	AS
38	Display window	PETG
39	Control PCB	94V0
40	Left panel	ABS
41	Left panel handle	ABS
42	Main PCB	94V0
43	Display panel PCB	94V0
44	Stainless steel screw	SUS 304

COMMON CLEANING METHODS

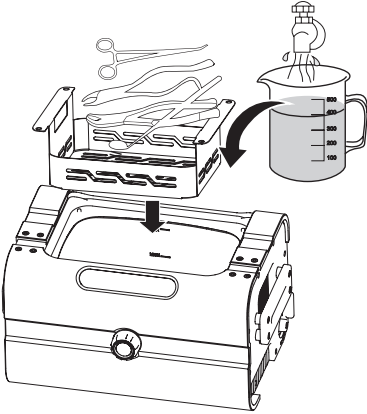REGULAR CLEANING

Cleaning Method:

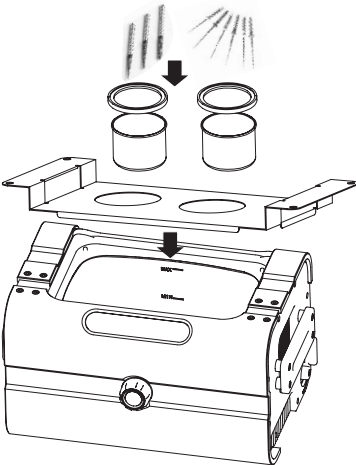
1. Add approximately 2.5 to 3 liters of enzymatic cleaning solution to the tank, ensuring the level is between the MIN and MAX lines.
2. Place the items to be cleaned inside the stainless steel basket. Carefully place the basket inside the tank.
3. Close the lid. Turn the power switch on. Rotate the dial counter clock wise to select the frequency then push in to set it. Turn the knob clockwise to select cleaning time. For a single layer of items, use 5 minutes. Push the dial to start ultrasound. To pause it, push the dial again.

For 2 layers of items, choose 10 minutes. For 3 or more layers of items, choose 15 minutes.

Do not run it more than 20 minutes within 30 minutes to avoid overheating. Do not keep on adding instruments and extend the cleaning time again to avoid overheating.

The lid must remain closed during cleaning cycle to prevent the spread of aerosols. The red open lid indicator will flash if the lid is not closed.

INDIRECT CLEANING


Cleaning Method:

1. Fill the main tank with water to between the MIN and MAX lines
2. Place small items such as burs and files in the stainless steel beakers (Use a different beaker for different sources.) Add enzymatic or other solution inside the beakers to submerge the items inside. Place the clear lids over the beakers if needed.
3. Close the lid then turn on the power. Turn the knob clockwise to select cleaning time and counter-clockwise to select operating frequency. Select 5 to 10 minutes then push in the knob to start the cleaning cycle.

OPERATION GUIDELINES

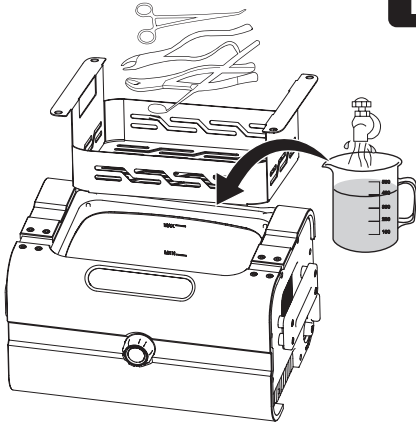1

Open the lid. Add approximately 2.5 to 3 liters of enzymatic cleaning solution to the tank, ensuring the level is between the MIN and MAX lines. Place the items to be cleaned inside the stainless steel basket. Carefully place the basket inside the tank.

Do not turn on the unit without cleaning solution in the tank. Dry running for over 30 seconds will damage the unit and severely reduce the life of the unit.

2

Close the lid. Turn on the power. Turn the knob counter-clockwise to select the desired operating frequency: 32 kHz or 48 kHz. Push in the knob to choose the frequency.

32 kHz for strong surface cleaning, rougher and quicker cleaning with higher noise, suitable for cleaning large instruments. 48 kHz for gentle precise cleaning, deeper cleaning with lower noise, mainly for sharp tools.

Turn the dial clockwise to select cleaning time.

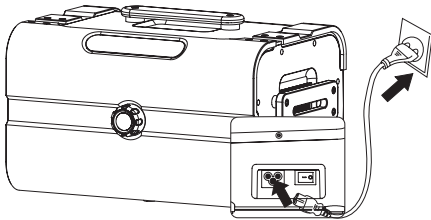
→05:00→07:30→10:00→12:30→15:00

Select the cleaning time and then push the dial to start or stop the unit.


For 1 layer of items, choose 5 minutes.
For 2 layers of items, choose 10 minutes.
For 3 layers of items, choose 15 minutes. Number of layers affects cleaning result.

Buzzing noise can be heard during cleaning cycle. Close the lid to reduce the noise level.

OPERATION GUIDELINES

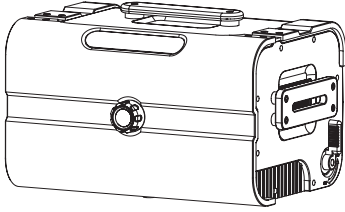
3

The timer on the LED display counts down to show the remaining cleaning time. To pause cleaning, push in the dial again

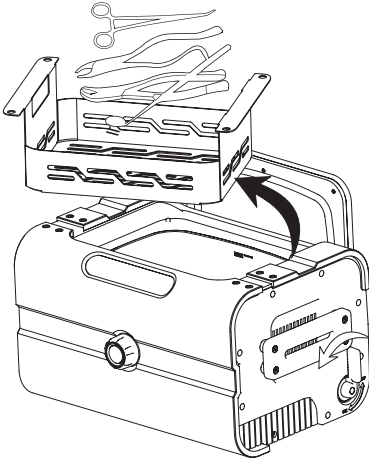
If the lid is opened during operation, the open-lid warning symbol will illuminate and then flash for 15 seconds. Additionally, warning beeps will sound if the unit is in a cycle. Closing the lid will return the unit to normal operation.


4

After cleaning is done, open the lid and retrieve the basket and items. Turn off the power, open the drainage valve to drain the solution out. Clean and wipe the tank dry then close the drainage valve.

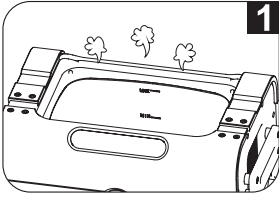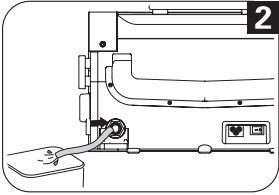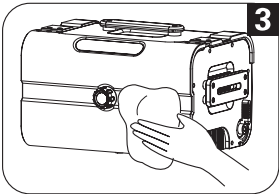
CARE AND MAINTENANCE

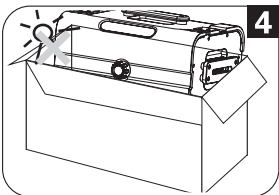
Do not turn on the unit without water in the tank.
Dry running for over 30 seconds will damage the unit and severely reduce the life of the unit.

Avoid leaving liquid in the tank for extended periods.
At the end of the day, connect the hose as shown, open the drain valve to discharge the dirty solution. Wipe the tank clean.

Do not spread disinfectant over the unit, only wipe it.
 Many disinfectants and bleach products contain sodium hypochlorite with strong oxidation. They will darken the aluminum housing and the stainless steel tank.
Disinfectants could cause corrossions inside the unit so do not spread it towards the unit. Only wipe it.
Wipe the housing with clean water and then dry off both the tank and housing thoroughly with paper towels.

Do not leave it under direct sunlight.
Leave it in a cool, dry and ventilated area.

SPECIFICATIONS

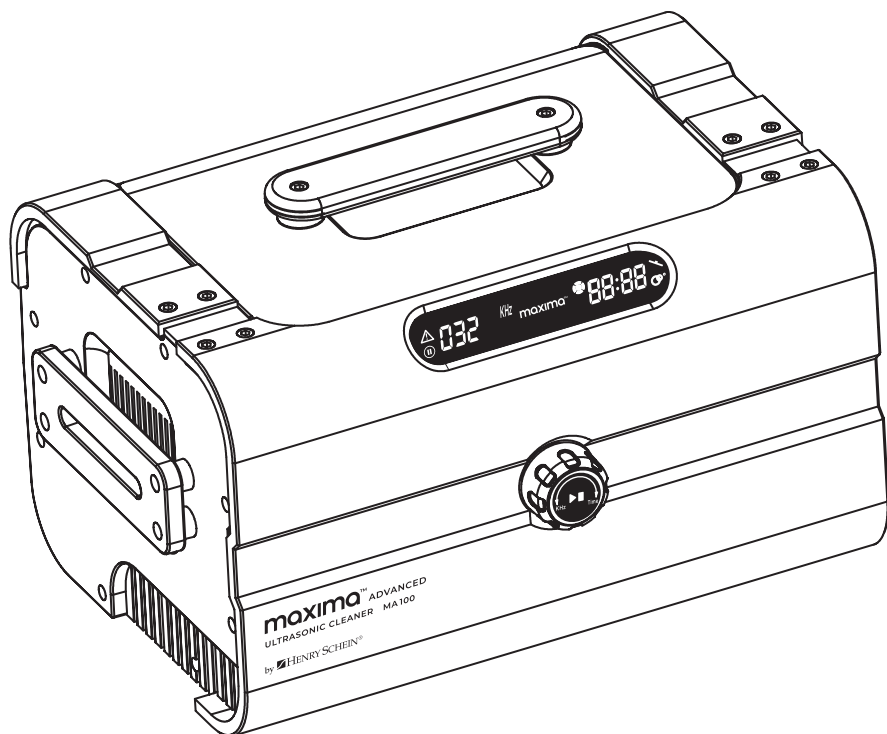
Description	Dual frequency Ultrasonic Cleaner	
Model	MA100	
Tank Capacity	4.0L / 1.06 gallon (US)	Max. 3.65 L / 1.0 gallon (US) Min. 1.8 L / 0.5 gallon (US)
Tank Size	24.8x17.4x10.3 cm / 9.8"x6.9"x4.1"	
Longest Item Fits inside Tank	26.5 cm / 10.4"	
Power Supply	CS4.0B (NH) 90W (AC120V 50/60Hz)	
Timer Settings	5min → 7.5min → 10min → 12.5min → 15min	
Drainage	Drainage valve	
Ultrasonic Frequency (Dual Frequency)	Low frequency: 32KHz High frequency: 48KHz	
Tank Material	Stainless Steel SUS304	
Housing Material	Aluminum+ABS	
Net Weight	7.2 kg / 15.9lb	
Gross Weight	9.2 kg / 20.3lb	
Unit Size	39.5x29.8x23.6 cm / 15.6"x11.7"x9.3"	
Inner Carton Size	48.5x36.5x32 cm / 19.1"x14.4"x12.6"	
Qty per Master Carton	1 pc / ctn	
Master Carton Size	51x38x35.5 cm / 20.0"x15.0"x14.0"	

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Henry Schein, Inc.